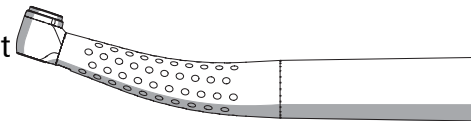
Midwest Stylus[®] Plus Air-Driven-High-Speed Handpiece / Turbine / Turbina

for Sirona, KaVo, NSK MachLite and Midwest Connection
pour raccord Sirona, KaVo, NSK MachLite et Midwest
para conexión Sirona, KaVo, NSK MachLite y Midwest

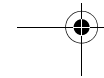
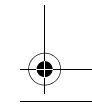
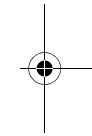
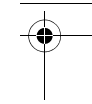
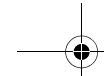
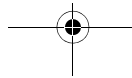
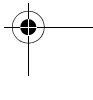
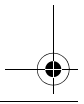
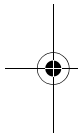
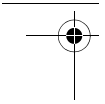
Operating Instructions
Notice d'utilisation
Instrucciones de uso

English / Français / Español


6607290_Titel.fm Seite 2 Montag, 5. September 2016 12:44 12

Midwest Stylus[®] Plus Air-Driven-High-Speed Handpiece

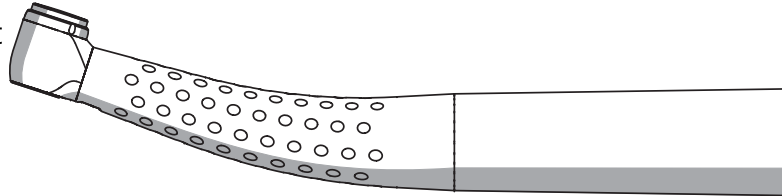for Sirona, KaVo,
NSK MachLite and Midwest
connection

English

Operating Instructions



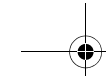
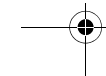
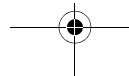
Table of Contents






1	Before you begin	6
1.1	Structure of the document	7
1.2	Service life	10
1.3	Trade names / registered trade marks of the manufacturers	11
2	Safety information	12
3	Technical description	16
3.1	Task	16
3.2	Structure of Midwest Stylus® Plus	17
3.3	Technical data	18
3.4	Handpiece connection types available	23
3.5	Product labeling	25

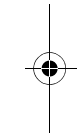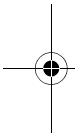4	Preparation	26
4.1	Initial start-up and longer breaks in use	26
4.2	Prior to starting the work day.....	26
4.3	Before each patient	26
5	Operation	28
5.1	Connecting the Sirona quick coupling to the supply hose.....	29
5.2	Replacing the handpiece.....	30
5.3	Inserting and removing burs.....	33
5.4	Adjusting the cooling spray	35
6	Follow-up	36
6.1	After each treatment session.....	36
6.2	At the end of the work day.....	37

English

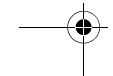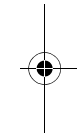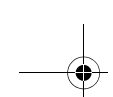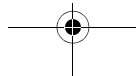
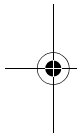
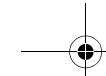
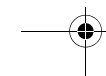
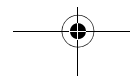
Table of Contents

Operating Instructions Midwest Stylus® Plus Air-Driven-High-Speed Handpiece

7	Conditioning	38
	7.1 Conduct pre-disinfection.....	38
	7.2 Automated cleaning and disinfecting	39
	7.3 Manual cleaning and disinfection	41
	7.4 Manual maintenance	43
	7.5 Sterilizing.....	47
8	Maintenance	51
	8.1 Maintenance of cooling spray nozzles	51
	8.2 Testing the FG clamping system.....	52
	8.3 Servicing Sirona quick coupling	54



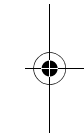
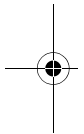
Operating Instructions Midwest Stylus® Plus Air-Driven-High-Speed Handpiece

Table of Contents


9	Spare parts and consumables	59
10	Storage and transport conditions	63
11	Disposal	65

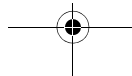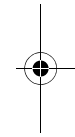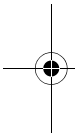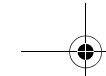English

1 Before you begin ...

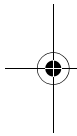Intended Use

6

1 Before you begin ...

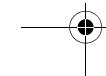
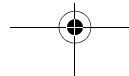
The Midwest Stylus® Plus handpiece corresponds with the provisions of state-of-the-art technology. The Midwest Stylus® Plus handpiece complies with the ISO 14457 and ISO 9168 standards.

The units, the product is connected to, must comply with the requirements of IEC 60601-1.

1. Read the operating instructions before using the Midwest Stylus® Plus handpiece.
2. Use the Midwest Stylus® Plus handpiece only for the purposes described in the operating instructions.
3. Follow the hygiene regulations, work safety regulations, and accident prevention measures that apply to the Midwest Stylus® Plus handpiece.

The handpieces of the Midwest Stylus® Plus Handpiece family are intended for:

6607290 D3673.201.01.01.02

Operating Instructions Midwest Stylus® Plus Air-Driven-High-Speed Handpiece

- Preparation of cavities and crowns
- Removal of carious material
- Removal of fillings
- Processing of tooth and restoration surfaces
- Reducing hard tooth structure

1 Before you begin ...

English

Contraindications

None

Target group

This product is intended only for use by trained dental personnel in dental practices and laboratories.

Scope of validity of operating instructions

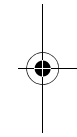
These operating instructions are applicable for the following handpieces:

- Midwest Stylus® Plus

1.1 Structure of the document

1.1.1 Labeling of information


- To prevent injuries, please observe all warnings.

Warnings


6607290 D3673.201.01.01.02

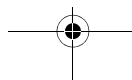7


1 Before you begin ...

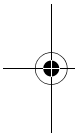Instructions for use

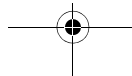8

Operating Instructions Midwest Stylus® Plus Air-Driven-High-Speed Handpiece

Warnings are labeled as follows:

⚠ DANGER! indicates a danger **leading** to death or serious injury if not avoided.

⚠ WARNING! indicates a danger that **may lead** to death or serious injury if not avoided.

⚠ CAUTION! indicates a danger that **may lead** to injury if not avoided.

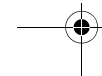
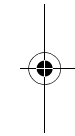
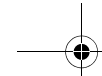
➤ To prevent material damage and additional expenses, please observe all instructions for use.

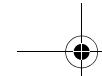
Instructions for use are labeled as follows:

NOTICE! indicates measures for the prevention of material damage.

IMPORTANT! indicates information on the avoidance of additional expenses and other important information.

Tip: indicates information for facilitating work.





1.1.2 Formats and symbols

The formats and symbols used in this document have the following meaning:

✓ Prerequisite	Requests you to do something.
1. First action step	
2. Second action step	
or	
➤ Alternative action	
↙ Result	
➤ Individual action step	

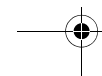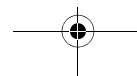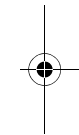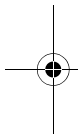English


1 Before you begin ...

Operating Instructions Midwest Stylus® Plus Air-Driven-High-Speed Handpiece

Use of formats and symbols [→ 9].	Identifies a reference to another text passage and specifies its page number.
• List	Identifies a list.

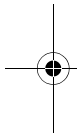
1.2 Service life

When used as intended:


- Non-moving parts have a typical service life of 5 years
- Moving parts have a typical service life of 3 years

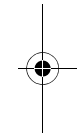
No warranty claim can be inferred here, as wear may occur earlier or later than indicated above depending on use, frequency of sterilization, and frequency of maintenance.

10

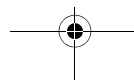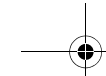6607290 D3673.201.01.01.02

1.3 Trade names / registered trade marks of the manufacturers

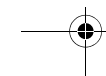
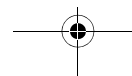
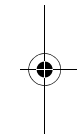
The designations used in this operating instruction with reference to the instruments are trade names or registered trade names of the manufacturers:

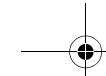
KaVo, Biberach, Germany:

- MULTiflex LUX.

NSK, Tochigi-ken, Japan:

- MachLite


2 Safety information


Obligations of the user

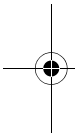- Use only fault-free materials that do **not** deviate from the specified data [→ 18].
- Protect yourself, patients, and others against any foreseeable dangers. To do this, follow the safety information.
- Comply with the Intended use of the equipment.
- You should always keep these operating instructions within reach for further reference.



Preventing the spread of infections and cross contamination

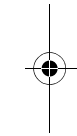Prevent the spread of infections and cross contamination between patients, users, and third parties. Sterilize equipment after each patient. Take the appropriate hygiene measures, e.g. wear protective gloves.

Prevention of eye damage

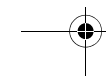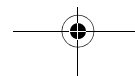The LED is in risk class 2 according to the IEC 62471:2006 standard. The LED emits optical radiation that is potentially hazardous and may be harmful to the eyes! Potential damage to the retina from the blue light emission. Do not stare at the LED for longer periods of time while in operation.


Malfunction or damage

Discontinue use immediately in case of malfunction, unusual or different sounds or damage. Damaged handpieces may cause injury. Notify the dental dealer or the manufacturer.

**Potential explosion hazard
Repair**




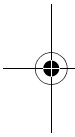
Do **not** use this product in areas subject to explosion hazards.
Do **not** attempt to repair the handpiece yourself.

Spare and accessory parts

The replacement of bearings can lead to sudden failures and injuries to patients due to the different levels of wear of the newer and older components. Safe operation is **not** guaranteed when repairs are performed incorrectly. For repairs, contact the dental dealer or the manufacturer.

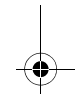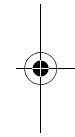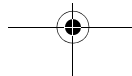
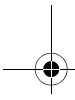
Use only original Midwest parts or parts approved by Midwest. Safe operation is **not** guaranteed for parts that have not been approved by Midwest.

2 Safety information

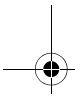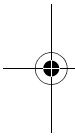First aid measures in the case of lubricant accidents

14

Operating Instructions Midwest Stylus® Plus Air-Driven-High-Speed Handpiece

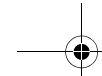
- General information: Immediately remove any clothing soiled by the product.
- After inhalation: Supply fresh air; consult doctor in case of complaints.
- After skin contact: If skin irritation continues, consult a doctor.
- After eye contact: Rinse opened eye for several minutes under running water.
- After swallowing: If symptoms persist consult doctor.

For details download the "*Midwest® Plus Aerosol Spray*" Material Safety Data Sheet from the Midwest homepage:
www.dentsplysirona.com







If you have any questions, please contact your dental dealer or the manufacturer.

English

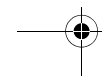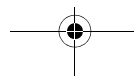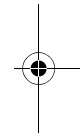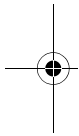
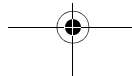
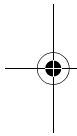
3 Technical description


3.1 Task

The Midwest Stylus® Plus handpiece is used to power dental instruments with an FG shank.

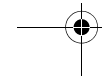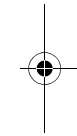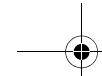The quick coupling is used for media supply and connection to the supply hose and features convenient swivel.

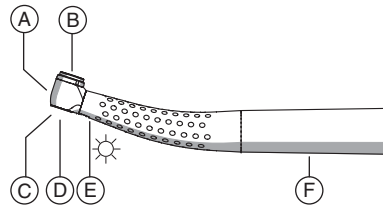3.2 Structure of Midwest Stylus® Plus

A	Handpiece head
B	Push button
C	Cooling spray outlet
D	Opening of chuck system
E	Light aperture
F	Handpiece sleeve




English

3.3 Technical data

Handpiece Midwest Stylus® Plus

	Midwest Stylus® Plus
Light function	x
Back-suction stop in the head housing	x
Anti-retraction valve in the spray water area	x
Pushbutton cover clamping system	FG

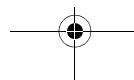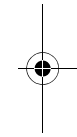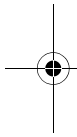English




Handpiece head

	Midwest Stylus® Plus
No-load speed in rpm	400 000 ± 10 %
Maximum torque in Nmm	~ 1.6

Friction grip bur

	Midwest Stylus® Plus
Shank standard	ISO 1797-1
Shank diameter in mm	1.59 - 1.60


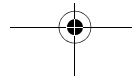
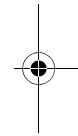
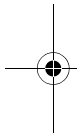
3 Technical description


Operating Instructions Midwest Stylus® Plus Air-Driven-High-Speed Handpiece

	Midwest Stylus® Plus
Maximum total length in mm	21
Maximum working diameter in mm (ISO 2157)	2.1

Treatment center

	Midwest Stylus® Plus
Driving air pressure in bar (psi) (flowing, dry, and clean)	2.7 - 3.0 (39 - 43.5)
Driving air consumption in NI/min	48 ± 5
Return air pressure in bar	< 0.3

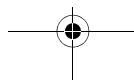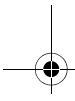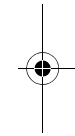
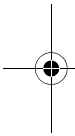
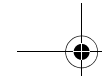
English

	Midwest Stylus® Plus
Spray air pressure in bar (psi)	~ 2.7 (39)
Spray air flow in NI/min	> 2.5
Spray water pressure in bar (psi)	~ 2.0 (29)
Water content in spray	> 50 ml/min

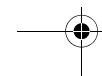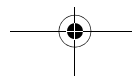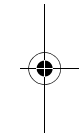The indicated pressures can be checked by a service technician with a star tester that is placed between the supply hose and the quick coupling.


Quick couplings (Sirona)

	Quick coupling F	Quick coupling B
Connection	6-hole	4-hole
Standard	ISO 9168	ISO 9168
Driving air return	without return airflow	without return airflow
Spray water control	on the coupling	-
Anti-retraction valve	in the spray water area	in the spray water area
Light function	x	-
Lamp voltage (lamp with dark brown socket)	3.6 V	-
Lamp voltage (LED)	3.6 V	-

3.4 Handpiece connection types available

	Connection type available for	Midwest Stylus® Plus
	Quick coupling F (Sirona)	x
	Quick coupling B (Sirona)	x
	Multiflex LUX (KaVo) quick coupling	x

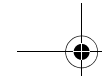English



3 Technical description

Operating Instructions Midwest Stylus® Plus Air-Driven-High-Speed Handpiece

	Connection type available for	Midwest Stylus® Plus
	MachLite quick coupling (NSK)	x
	Midwest quick coupling	x

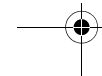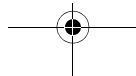3.5 Product labeling

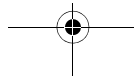
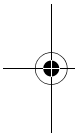
Sterilizable in a steam sterilizer (autoclave) at the temperature specified

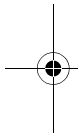
Manufacturing date

Serial Number

4 Preparation

4.1 Initial start-up and longer breaks in use

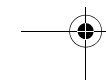
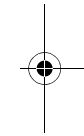
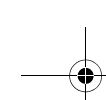
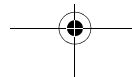
- ✓ The handpiece is only able to reach full power if indicated operating pressures are set [→ 18]. If necessary have the pressures checked by a service technician with a star tester that is placed between the supply hose and the quick coupling.
- Sterilize the handpiece and accessory parts prior to startup.
- Clean and maintain the handpiece after longer breaks in use.

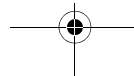
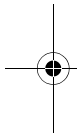
4.2 Prior to starting the work day

- Purge the water and air channels for 30 seconds.


4.3 Before each patient

1. Purge the water and air channels for 30 seconds.
2. Attach the turbine [→ 30].




English

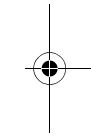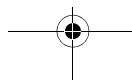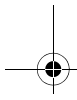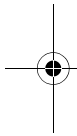3. Insert the bur instrument [→ 33].
 4. Set a sufficient amount of cooling water (> 50 ml/min) [→ 35].
 5. Use filtered water only (< 50 µm).
 6. Check the nozzles for blockages and lime deposits, for example, and clean the nozzles if necessary [→ 51].
- ⚠ CAUTION!** Insufficient cooling leads to overheating of the preparation site and damage to the tooth substance. Ensure that the water content is > 50ml/min.



5 Operation

NOTICE! Use only burs and diamond polishers that are sharp and undamaged. Use clean burs and diamond polishers to avoid dirt in the clamping system.

⚠ CAUTION! A loose or partially removed bur instrument can detach itself from the head or break off. This may cause injury! Therefore, use the handpiece only if the bur instrument is inserted at least 10 mm and is clamped securely in place.

⚠ CAUTION! Insufficient cooling leads to overheating of the preparation site and damage to the tooth substance. Ensure that the water content is > 50ml/min.

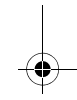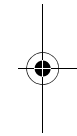⚠ CAUTION! Do not pull the patient's cheek back with the handpiece while the handpiece is rotating. This would actuate the pushbutton, thus creating a risk of burning the patient's oral mucosa.



⚠ CAUTION! Potentially hazardous optical radiation may cause harm to the eyes. Do **not** stare at the LED for longer periods of time while in operation.


IMPORTANT: For application details and operating data of the different preparation tools, please refer to the information provided by the manufacturer.

5.1 Connecting the Sirona quick coupling to the supply hose

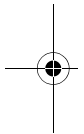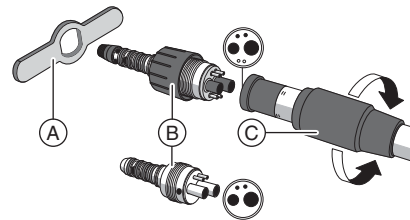1. Attach the quick coupling (B) to the supply hose, noting the various tube diameters.
2. Screw the cap nut (C) tight. Secure the quick coupling using the key (A).



6607290 D3673.201.01.01.02





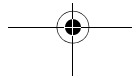
5 Operation

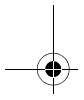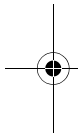30

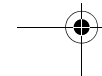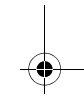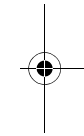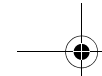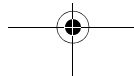5.2 Replacing the handpiece

⚠ CAUTION! Do not remove the handpiece while the handpiece is running.

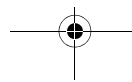
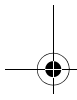
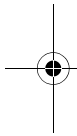
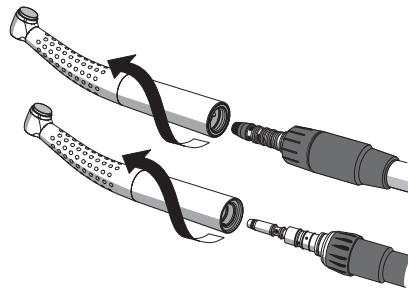
5.2.1 Handpiece with Sirona or KaVo connection

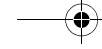
Attaching the handpiece

- Insert the handpiece until it audibly clicks into place.

Removing the handpiece

- ✓ The handpiece is at a standstill.
- Hold the quick coupling and the hose nut firmly and pull the handpiece while turning it slightly. Do **not** pull on or hold the supply hose.

5 Operation

Operating Instructions Midwest Stylus® Plus Air-Driven-High-Speed Handpiece

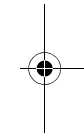
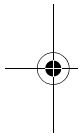
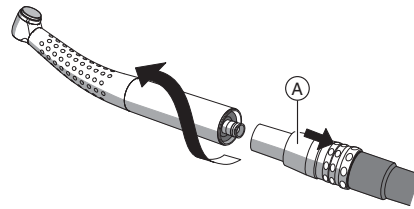
5.2.2 Handpiece with NSK MachLite connection

Attaching the handpiece

- Insert the handpiece until it audibly clicks into place.



Removing the handpiece



- ✓ The handpiece is at a standstill.
- Slide back the sleeve (A) on the quick coupling toward the supply hose and pull the handpiece while turning it slightly. Do **not** pull on or hold the supply hose.

32

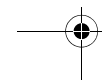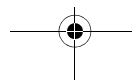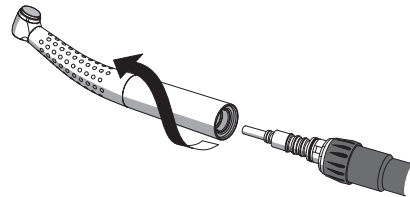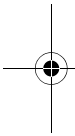6607290 D3673.201.01.01.02



5.2.3 Handpiece with Midwest connection

Attaching the handpiece

- Insert the handpiece until it audibly clicks into place.


Removing the handpiece

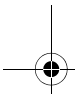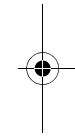- ✓ The handpiece is at a standstill.
- Press the quick coupling release buttons and hold the hose nut firmly and pull the handpiece while turning it slightly. Do **not** pull on or hold the supply hose.


5.3 Inserting and removing burs

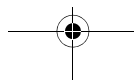IMPORTANT: Check the push button to make sure it moves freely!



5 Operation

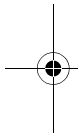34

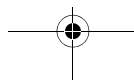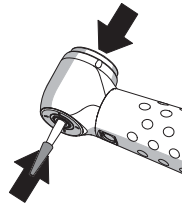Operating Instructions Midwest Stylus® Plus Air-Driven-High-Speed Handpiece

NOTICE! Use only burs and diamond polishers that are sharp and undamaged. Use clean burs and diamond polishers to avoid dirt in the clamping system.

⚠ CAUTION! A loose or partially removed bur instrument can detach itself from the head or break off. This may cause injury! Therefore, use the handpiece only if the bur instrument is inserted at least 10 mm and is clamped securely in place.

Inserting the friction grip bur

✓ The handpiece is at a standstill.

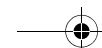
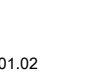
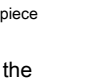
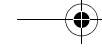
1. Press the button and insert the bur up to the stop.
2. Pull on the bur to check that it is firmly attached.

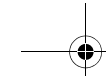
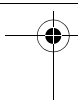
Removing the friction grip bur

✓ The bur must not be moving.



➤ Press the button and remove the bur.

6607290 D3673.201.01.01.02

English

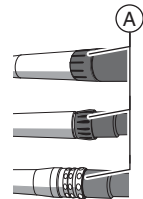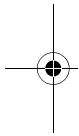5.4 Adjusting the cooling spray

➤ Adjust the flow rate of the cooling water using the control ring (A) (> 50 ml/min).

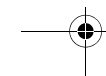
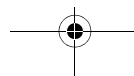
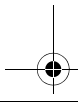
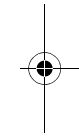
Tip: You can measure the amount of cooling water with a measuring cup and watch.

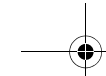
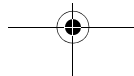
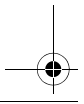
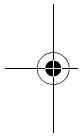
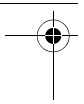
Water flow rate

F (Sirona) quick coupling: The maximum water flow is set when the control ring clicks into place.

KaVo quick coupling: The maximum water flow is set when the two markings face each other.

MachLite quick coupling (NSK): The maximum water flow is set when the control ring clicks audibly into place.

6 Follow-up

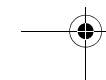
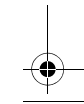
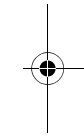
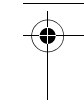
6.1 After each treatment session

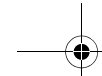
NOTICE! Condition immediately, or at the latest, one hour after treatment.

NOTICE! Only use Midwest® Plus Aerosol Spray.

- ✓ The bur instrument must not be moving.
- ✓ Wear appropriate protective clothing.

1. Purge the water and air channels on the treatment center for 30 seconds.
2. Remove the bur instrument with tweezers.
3. Pre-disinfect directly at the treatment center [→ 38].
4. Remove the handpiece.





5. Transport the handpiece to the hygiene room in a suitable transport container.
6. Conduct automatic conditioning [→ 39]. Manual conditioning [→ 41] is possible in exceptional cases if the national/local regulations are followed.
7. Spray the turbine [→ 43].
8. Sterilize the turbine and the accessories [→ 47].


6.2 At the end of the work day

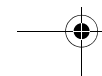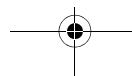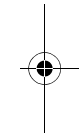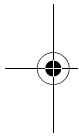- Lubricate the turbine [→ 43].

7 Conditioning

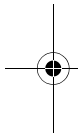38

7 Conditioning

7.1 Conduct pre-disinfection

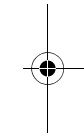
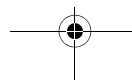
- ✓ Wear appropriate protective clothing.
- ✓ All disinfectants must be approved in your country and have proven bactericidal, fungicidal and virucidal properties. Only use disinfectants with **no** protein-fixing properties.

1. Spray the surface with disinfectant.
2. Wipe the disinfectant off with a cloth.

↘ For further processing, the handpiece should be dry and free of residue.

In the USA and Canada, for example, you can use:

- CAVICIDE®
- CAVIWIPES™

Please observe the manufacturer's instructions for using instrument disinfectants.

7.2 Automated cleaning and disinfecting ...

NOTICE! Do **not** carry out automated cleaning of the quick couplings.

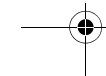
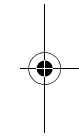
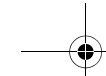
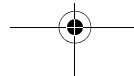
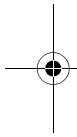
7.2.1 ... with cleaning and disinfection equipment

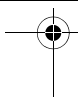
The handpiece can also be cleaned and disinfected in suitable cleaning and disinfection equipment.

The cleaning and disinfection equipment used must be approved by its manufacturer for the cleaning and disinfection of dental instruments and comply with EN ISO 15883-1 (e.g. 95°C (203°F) and 10 min. holding time).



For further details, refer to the operating instructions supplied with the unit.

- ✓ The handpiece is conditioned with a cleaning and disinfection device.







7 Conditioning


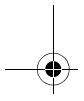
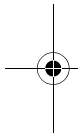
40

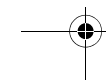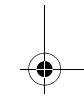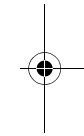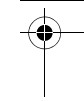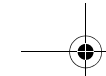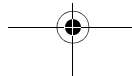Operating Instructions Midwest Stylus® Plus Air-Driven-High-Speed Handpiece

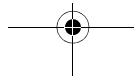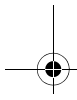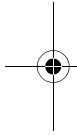1. Check whether the handpiece is clean after conditioning under good lighting (min. 500 lux) and color rendering index (min. 80 Ra).
2. If it is still dirty, repeat the process.
 - ↳ For further processing, the handpiece should be dry and free of residue.
3. Blow the handpiece out with max. 3 bar.
4. Maintain mechanical parts manually [→ 43].
5. Maintain the push button chuck manually [→ 45].
6. Pack the handpiece in packaging material suitable for sterilization and storage. e.g. paper/plastic composite packaging.
7. Sterilize the adapter [→ 47].

7.3 Manual cleaning and disinfection

IMPORTANT: Manual conditioning is possible in exceptional cases if the national/regional regulations are followed. The national/regional regulations are to be checked before.

NOTICE! Condition immediately, or at the latest, one hour after treatment.

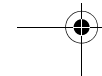
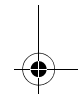
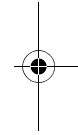
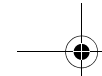
NOTICE! Never clean in an ultrasound bath!

NOTICE! Never immerse in disinfectant solution!

NOTICE! Only use Midwest® Plus Aerosol Spray.


IMPORTANT: Use a soft, clean, and disinfected brush for cleaning.

- ✓ Wear appropriate protective clothing.
- ✓ All disinfectants must be approved in your country and have proven bactericidal, fungicidal and virucidal properties. Only use disinfectants with **no** protein-fixing properties.

7 Conditioning

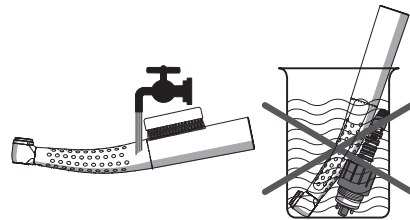
42

Operating Instructions Midwest Stylus® Plus Air-Driven-High-Speed Handpiece

1. Brush the handpiece under running water (< 38 °C, < 100 °F, at least drinking water quality) and good lighting (min. 500 lux) and color rendering index (min. 80 Ra) until no more dirt can be seen, for at least 10 seconds.
2. Flush the drive channels with spray.
3. Clean and disinfect spray channels with suitable agents and adapters according to manufacturer's instructions.
4. Blow spray channels with suitable adapters out with 2.5 - 3 bar until no more moisture displaces, but at least 10 seconds.
5. Conduct thermal disinfection or unwrapped steam sterilization.
6. Maintain mechanical parts manually [→ 43].
7. Maintain the push button chuck manually [→ 45].
8. Pack the handpiece in packaging material suitable for sterilization and storage. e.g. paper/plastic composite packaging.
9. Sterilize the adapter [→ 47].

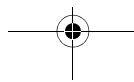6607290 D3673.201.01.01.02




7.4 Manual maintenance



7.4.1 Maintenance of mechanical parts


Intervals

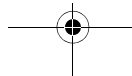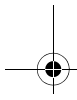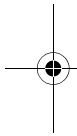- At least every noon and evening
- Prior to each sterilization
- After every thermal disinfection without integrated maintenance
- During a longer preparation (> 10 minutes)



English

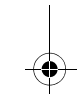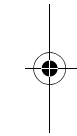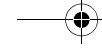7 Conditioning

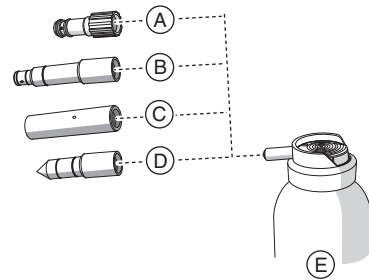
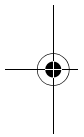
Required accessories

- A Sirona connection spray adapter
- B KaVo connection spray adapter
- C NSK MachLite connection spray adapter
- D Midwest connection spray adapter
- E Midwest® Plus Aerosol Spray

NOTICE! Only use Midwest® Plus Aerosol Spray.


Process

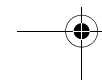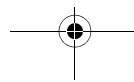- ✓ The spray adapter is disinfected.
- ✓ The O-rings on the spray adapter are intact.

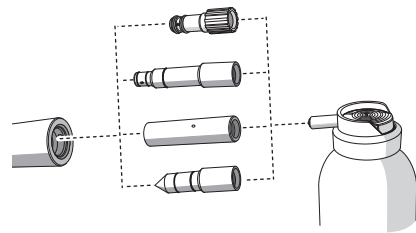Operating Instructions Midwest Stylus® Plus Air-Driven-High-Speed Handpiece


1. Fit the spray adapter onto the nozzle of the spray can.
2. Insert the handpiece as far as it will go and hold it in place.
3. Spray into the handpiece for 1-2 seconds.
IMPORTANT: Hold the spray can upright.
4. Wipe any spray that comes out with a disinfection cloth.
5. Repeat the process until the spray is clear.

7.4.2 Care of the push button chuck

Use Midwest® Plus Aerosol Spray on the push button chuck to remove deposits and ensure proper functioning of the clamping system.

Interval

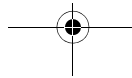- At least once a week


7 Conditioning

English

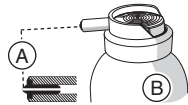7 Conditioning






Operating Instructions Midwest Stylus® Plus Air-Driven-High-Speed Handpiece

Required accessories

A	Black nozzle insert
B	Midwest® Plus Aerosol Spray

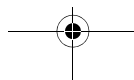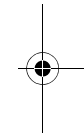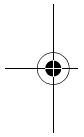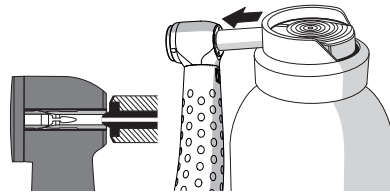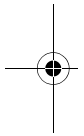NOTICE! Only use Midwest® Plus Aerosol Spray.

Procedure

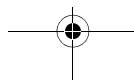
✓ The handpiece is cleaned and disinfected.

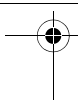
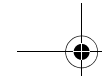
1. Press the handpiece head with the chuck firmly against the spray can nozzle.
2. Spray the chuck for 1 - 2 seconds.
IMPORTANT: Hold the spray can upright.
3. Wipe any spray that comes out with a disinfection cloth.

7.5 Sterilizing



Intervals:

- Prior to initial operation
- Prior to every other use

7 Conditioning




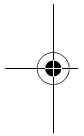
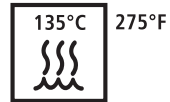
Operating Instructions Midwest Stylus® Plus Air-Driven-High-Speed Handpiece

Process

- ✓ The handpiece is cleaned and disinfected.
- ✓ Handpiece maintenance is complete.
- ✓ If necessary, the handpiece can be sterilized in packaging suitable for sterilization and storage, e.g. paper/plastic composite packaging or container.
- Sterilize the handpiece in the steam sterilizer with saturated water vapor.

Temperature: 135° C (275° F)

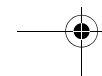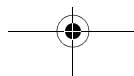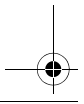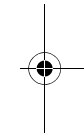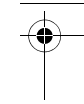Overpressure: 2.13 bar (30.89 psi)


Article	Holding time at 135 °C (275° F)	Drying time
Wrapped instruments	10 minutes	30 minutes
Unpackaged instruments	3.5 minutes	0 - 1 minute

Gravity displacement steam sterilizers are permitted.

NOTICE! Do not exceed 140°C (284°F), even during the drying phase.


After sterilizing

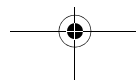1. Remove the handpiece from the steam sterilizer immediately.
⚠ CAUTION! The handpiece will be hot. This may cause burns!
NOTICE! Do **not** immerse the handpiece in cold water to speed up the cooling process. This will damage your handpiece!
2. Store all handpieces so that they are protected from contamination.



7 Conditioning

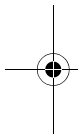
50



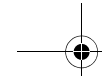
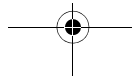
Operating Instructions Midwest Stylus® Plus Air-Driven-High-Speed Handpiece

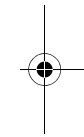3. Sterilize again once the storage period has elapsed.

6607290 D3673.201.01.01.02

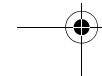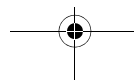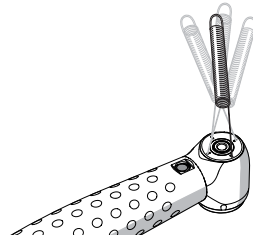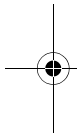
8 Maintenance






Elastomers, e.g. O-rings, must be replaced depending on their degree of wear.

8.1 Maintenance of cooling spray nozzles

If your tap water is very hard, lime deposits may constrict or completely block the cooling spray nozzles.

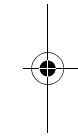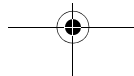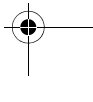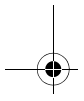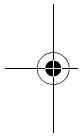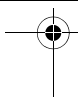1. Carefully clean the nozzle openings by regularly running a disinfected cleaning wire through them.
2. Let the handpiece run briefly with cooling spray.






8.2 Testing the FG clamping system


Interval

Test the FG clamping system at least once a month.

Required accessories

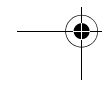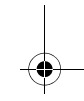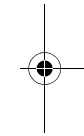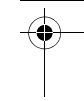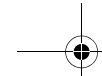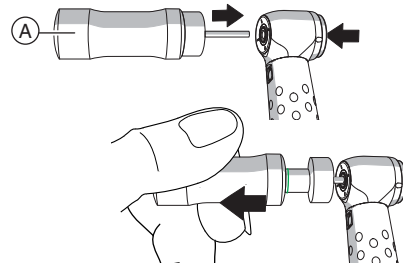Chuck tester

Process

✓ The expiry date of the chuck tester (A) has **not** elapsed.

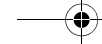
1. Insert the chuck tester into the FG clamping system [→ 33].
2. Tighten the chuck tester until the marking ring appears (withdrawal force: 22 N)

Does the chuck tester slide out of the chuck before the marking ring appears?



⚠ **CAUTION!** The chuck is defective and the secure fit of the bur is not guaranteed. This may cause injury!

1. Do not use the product.
2. Have the clamping system replaced by Midwest.

Tip: Record the time and result of the check for your own information.

8 Maintenance



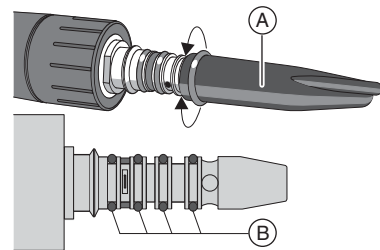
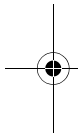
54

8.3 Servicing Sirona quick coupling

8.3.1 Replacing O-rings

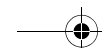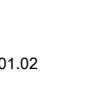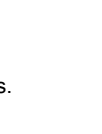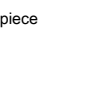
1. Remove the defective O-ring.
2. Slide the tool (A), with the new O-ring attached, to just in front of the corresponding groove (B).
3. Insert the O-ring. Do **not** use any sharp objects.
4. Lightly oil the O-ring with Midwest® Plus Aerosol Spray.
NOTICE! Do **not** use Vaseline or silicone grease on the O-rings.

8.3.2 Replacing the compression ring

In order to guarantee proper functioning of the “anti-retraction valve in the spray water duct”, we recommend replacing the compression ring (C) every six months.

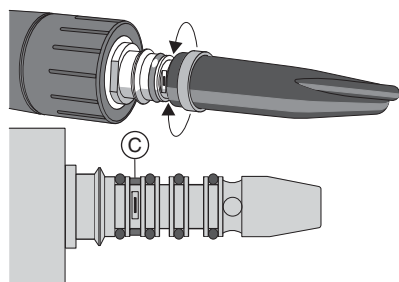
1. Purge the spray water duct at the maximum water flow.
2. Slide the tool, with the new compression ring attached, to just in front of the corresponding groove (C).
3. Insert the compression ring. Do **not** use any sharp objects.
4. Check that the ring fits snugly all the way around and is **not** twisted.

8 Maintenance

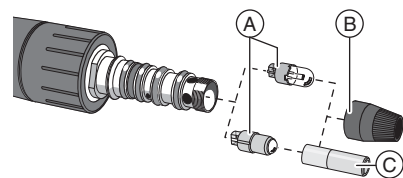Operating Instructions Midwest Stylus® Plus Air-Driven-High-Speed Handpiece


8.3.3 Replacing the halogen lamp/LED

⚠ CAUTION! The lamp can be hot. Risk of burns! Allow the lamp to cool down.

1. Remove the handpiece.
2. Unscrew the cap (B).
3. Remove the defective halogen lamp/LED (A).
4. Insert the new halogen lamp/LED. Ensure the proper position of the contact surfaces.
NOTICE! Pressing on the lens can destroy the LED. Therefore, please use the installation tool (C) provided to insert the LED.
5. Wipe the bulb of the halogen lamp with a clean cloth.
6. Unscrew the cap from the quick coupling.
7. Check the light function of the quick coupling.

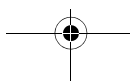Does the LED not light up?

➤ Remove the LED and re-insert it after rotating it 180° around its own axis.

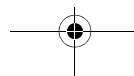
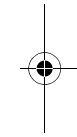
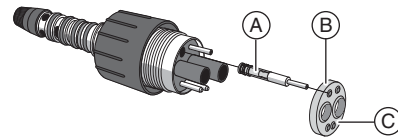
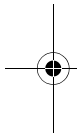
8.3.4 Replacing the spray water cartridge

Required accessories

- Spray water cartridge
- Midwest® Plus Aerosol Spray

If the spray water cartridge leaks or is clogged.

1. Remove the quick coupling from the supply hose.
2. Remove the sealing washer (B).
3. Click the control ring into the maximum water flow. The spray water cartridge (A) can be replaced **only** in this position.
4. Remove the spray water cartridge.
5. Spray oil lightly onto the new spray water cartridge.
6. Insert the new spray water cartridge.



8 Maintenance

Operating Instructions Midwest Stylus® Plus Air-Driven-High-Speed Handpiece

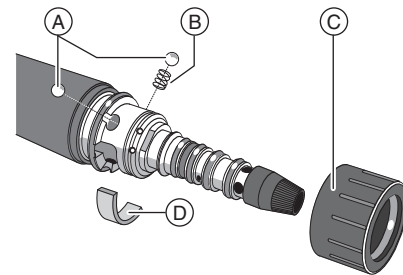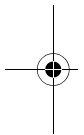
7. Note the position of the bulging side (C) when fitting the sealing washer.

8.3.5 Replacing the felt strip (quick coupling F)

Replace the felt strip if the oil-laden return air that escapes is soiled. We recommend replacing the felt strip at least every three months.

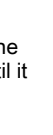1. Click the control ring into the maximum water flow.
2. Place the coupling upright on a firm, flat surface supported by the screw-on cap and firmly push the control ring (C) downward until it disengages.
NOTICE! Note that the balls (A) and springs (B) may drop out.
3. Replace the contaminated felt strip (D).
4. Click the control ring back into position, taking care to keep the balls and springs in place.

9 Spare parts and consumables

Use only original Midwest parts or parts approved by Midwest.

	REF		REF
Midwest® Plus Aerosol Spray (500 ml can)	38 00 80	Halogen lamp (dark brown base)	59 40 291
Chuck tester	33 27 793	Cleaning wire for spray nozzles	24 00 232

Handpiece with Sirona connection

	REF		REF
Spray adapter	59 41 802	O-ring for spray adapter	18 91 840

English

9 Spare parts and consumables

Operating Instructions Midwest Stylus® Plus Air-Driven-High-Speed Handpiece


	REF		REF
Replacement kit for quick coupling: <ul style="list-style-type: none">• 10 O-rings• 2 compressed rings• 1 tool for O-rings and compressed rings	41 75 803	Tool for O-rings and compressed rings	41 74 343
		Screw-on cap for R/F coupling	89 16 645
		Wrench for quick coupling	59 41 794
		LED for R/F coupling	63 14 558
Spring coupling F	89 17 262	Control ring coupling F	41 76 603
2.5 mm Ø ball for coupling F	34 20 742	Felt strip coupling F	41 76 389
Sealing washer for R/F quick coupling	89 16 629	Spray water cartridge for R/F coupling	77 43 444

Handpiece with KaVo connection

	REF		REF
Spray adapter (KaVo)	54 56 954	O-rings (small) for spray adapter	18 90 842
O-ring (large) for spray adapter	18 91 444		

Handpiece with NSK connection

	REF		REF
Spray adapter (NSK ML)	58 67 911	O-ring set (NSK ML)	58 74 958



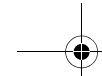
9 Spare parts and consumables

Operating Instructions Midwest Stylus® Plus Air-Driven-High-Speed Handpiece

Handpiece with Midwest connection

	REF			REF
Spray adapter (Midwest)	38 00 85			

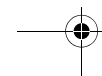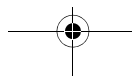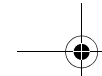10 Storage and transport conditions

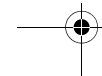
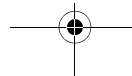
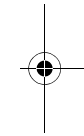
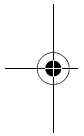
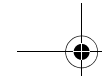
Protect from
moisture

Sensitive
contents


English

10 Storage and transport conditions

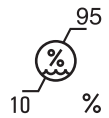Operating Instructions Midwest Stylus® Plus Air-Driven-High-Speed Handpiece

95

Relative humidity

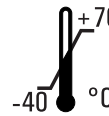
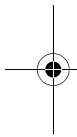
%

106

Air pressure

50 kPa




+70

Temperature

°C

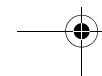
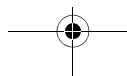
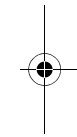
After a severe change in temperature, allow sufficient time for acclimation.

11 Disposal

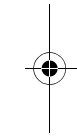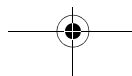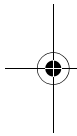- According to current information, the product does not contain any substances that are hazardous to the environment.
- Disinfect the product prior to disposal.
- Observe the applicable disposal regulations for your area.




6607290.book Seite 66 Montag, 5. September 2016 11:53 11

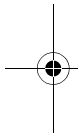
11 Disposal


66

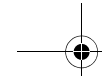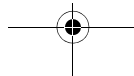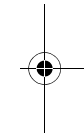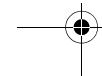Operating Instructions Midwest Stylus® Plus Air-Driven-High-Speed Handpiece

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