






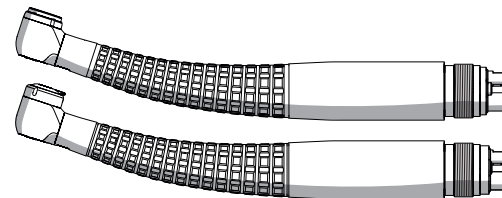
Midwest Tradition® TB/Pro TBF / Midwest Tradition® TL/Pro TLF

Air-Driven-High-Speed Handpiece
with Fixed Connection /
Turbine avec raccordement fixe /
Turbina con conexión fija


English / Français / Español

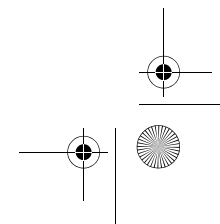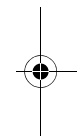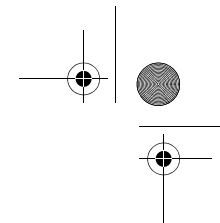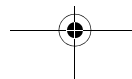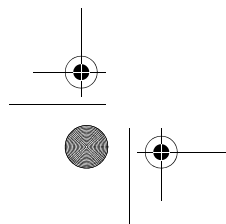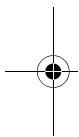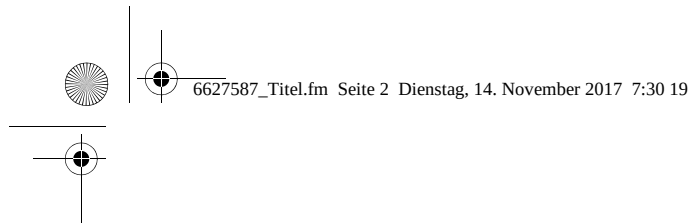Operating Instructions
Notice d'utilisation
Instrucciones de uso


Midwest Tradition® TB/Pro TBF Midwest Tradition® TL/Pro TLF

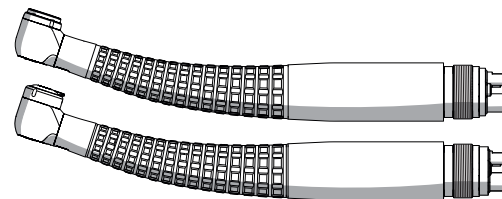Air-Driven-High-Speed Handpiece
with Fixed Connection



English

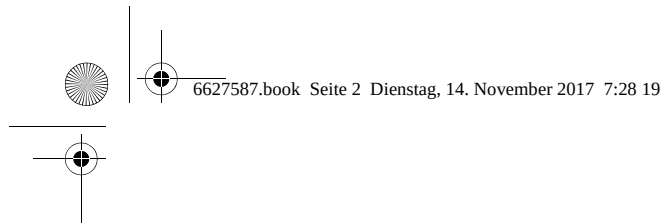
Operating Instructions

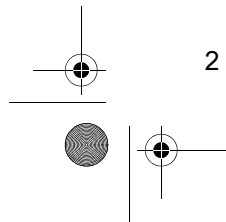
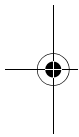
6627587.book Seite 2 Dienstag, 14. November 2017 7:28 19

2

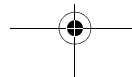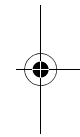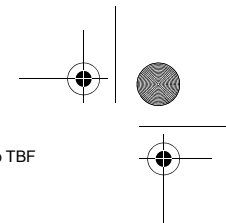Operating Instructions Midwest Tradition® TB/Pro TBF

66 27 587 D3673.201.02.01.02

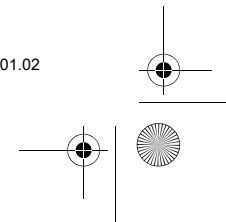
Table of Contents

1	Before you begin ...	6
1.1	Structure of the document	8
1.2	Service life	11
2	Safety information	12
3	Technical description	15
3.1	Task	15
3.2	Structure of Midwest Tradition TB	15
3.3	Structure of Midwest Tradition Pro TBF	16
3.4	Structure of Midwest Tradition TL	17
3.5	Structure of Midwest Tradition Pro TLF	18
3.6	Technical data	19

Table of Contents


Operating Instructions Midwest Tradition® TB/Pro TBF / Midwest Tradition® TL/Pro TLF

	3.7	Available high-speed handpiece connection types	24
	3.8	Product labeling.....	25
4		Preparation	27
	4.1	Initial start-up and longer breaks in use	27
	4.2	Prior to starting the work day.....	27
	4.3	Before each patient	28
5		Operation	29
	5.1	Replacing the high-speed handpiece	30
	5.2	Inserting and removing burs.....	32
6		Follow-up	35
	6.1	After each treatment session.....	35
	6.2	At the end of the work day.....	36

7	Conditioning	37
7.1	Conduct pre-disinfection.....	37
7.2	Automated cleaning.....	38
7.3	Manual cleaning	39
7.4	Manual maintenance	41
7.5	Sterilizing.....	47
8	Maintenance	50
8.1	Testing the FG clamping system.....	50
9	Spare parts and consumables	52
10	Storage and transport conditions	53
11	Disposal	54

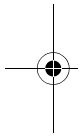1 Before you begin ...

6

Operating Instructions Midwest Tradition® TB/Pro TBF / Midwest Tradition® TL/Pro TLF

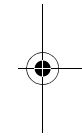
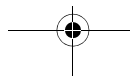
1 Before you begin ...

The Midwest Tradition TB/Pro TBF / Midwest Tradition TL/Pro TLF high-speed handpiece corresponds with the provisions of state-of-the-art technology. The Midwest Tradition TB/Pro TBF / Midwest Tradition TL/Pro TLF high-speed handpiece complies with the ISO 14457 and ISO 9168 standards.

The units, the product is connected to, must comply with the requirements of IEC 60601-1.

1. Read the operating instructions before using the Midwest Tradition TB/Pro TBF / Midwest Tradition TL/Pro TLF high-speed handpiece.
2. Use the Midwest Tradition TB/Pro TBF / Midwest Tradition TL/Pro TLF high-speed handpiece only for the purposes described in the operating instructions.
3. Follow the hygiene regulations, work safety regulations, and accident prevention measures that apply to the Midwest Tradition

66 27 587 D3673.201.02.01.02

Intended use

TB/Pro TBF / Midwest Tradition TL/Pro TLF high-speed handpiece
in your country.

The high-speed handpieces of the Midwest Tradition TB/Pro TBF /
Midwest Tradition TL/Pro TLF High-Speed Handpiece family are
intended for:

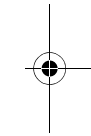
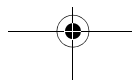
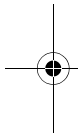
- Preparation of cavities and crowns
- Removal of carious material
- Removal of fillings
- Processing of tooth and restoration surfaces
- Reducing hard tooth structure

None

This product is intended only for use by trained dental personnel in
dental practices and laboratories.

Contraindications

Target group

1 Before you begin ...

Scope of validity of operating instructions

Operating Instructions Midwest Tradition® TB/Pro TBF / Midwest Tradition® TL/Pro TLF

These operating instructions are applicable for the following high-speed handpieces:

- Midwest Tradition® TB
- Midwest Tradition® Pro TBF
- Midwest Tradition® TL
- Midwest Tradition® Pro TLF

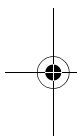
1.1 Structure of the document

1.1.1 Labeling of information

➤ To prevent injuries, please observe the warnings.

Warnings are labeled as follows:


⚠ DANGER! indicates a danger **leading** to death or serious injury if not avoided.

Warnings

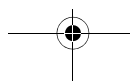8

Instructions for use

⚠ WARNING! indicates a danger that **may lead** to death or serious injury if not avoided.

⚠ CAUTION! indicates a danger that **may lead** to injury if not avoided.

➤ To prevent material damage and additional expenses, please observe all instructions for use.

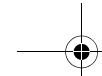
Instructions for use are labeled as follows:

NOTICE! indicates measures for the prevention of material damage.

IMPORTANT: indicates information on the avoidance of additional expenses and other important information.

Tip: indicates information for facilitating work.

English

1 Before you begin ...


Operating Instructions Midwest Tradition® TB/Pro TBF / Midwest Tradition® TL/Pro TLF


1.1.2 Formats and symbols

The formats and symbols used in this document have the following meaning:


✓ Prerequisite 1. First action step 2. Second action step or ➤ Alternative action ↗ Result ➤ Individual action step	Requests you to do something.
Use of formats and symbols [→ 10].	Identifies a reference to another text passage and specifies its page number.
• List	Identifies a list.

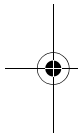10

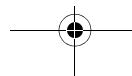66 27 587 D3673.201.02.01.02

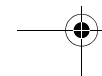1.2 Service life

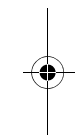
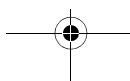
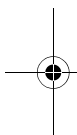
When used as intended:

- Non-moving parts have a typical service life of 5 years
- Moving parts have a typical service life of 3 years

No warranty claim can be inferred here, as wear may occur earlier or later than indicated above depending on use, frequency of sterilization, and frequency of maintenance.

Elastomers, e.g. O-rings, must be replaced depending on their degree of wear.

Check the clamping system of friction grip burs on a monthly basis [→ 50].

2 Safety information

Obligations of the user

Preventing the spread of infections and cross contamination

Malfunction or damage

Potential explosion hazard

12

Operating Instructions Midwest Tradition® TB/Pro TBF / Midwest Tradition® TL/Pro TLF

2 Safety information

- Use only fault-free materials that do **not** deviate from the specified data [→ 19].
- Protect yourself, patients, and others against any foreseeable dangers. To do this, follow the safety information.
- Comply with the Intended use of the equipment.
- You should always keep these operating instructions within reach for further reference.

Prevent the spread of infections and cross contamination between patients, users, and third parties. Sterilize equipment after each patient.

Take the appropriate hygiene measures, e.g. wear protective gloves.

Discontinue use immediately in case of malfunction, unusual or different sounds or damage. Damaged high-speed handpieces may cause injury. Notify the dental depot or the manufacturer.

Do **not** use this product in areas subject to explosion hazards.

66 27 587 D3673.201.02.01.02

Repair

Do **not** attempt to repair the high-speed handpiece yourself.

The replacement of bearings can lead to sudden failures and injuries to patients due to the different levels of wear of the newer and older components. Safe operation is **not** guaranteed when repairs are performed incorrectly. For repairs, contact the dental supplies distributor or the manufacturer.

Spare and accessory parts

Use only original Midwest parts or parts approved by Midwest. Safe operation is **not** guaranteed for parts that have not been approved by Midwest.

First aid measures in the case of lubricant accidents

General information:

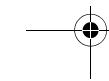
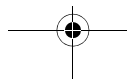
Immediately remove any clothing soiled by the product.

After inhalation:

Supply fresh air; consult doctor in case of complaints.



English

2 Safety information

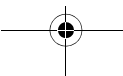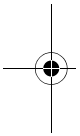Operating Instructions Midwest Tradition® TB/Pro TBF / Midwest Tradition® TL/Pro TLF

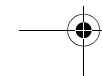
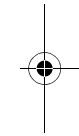
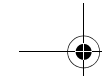
After skin contact: If skin irritation continues, consult a doctor.

After eye contact: Rinse opened eye for several minutes under running water.

After swallowing: If symptoms persist consult doctor.

For details download the Midwest Plus Aerosol Spray Material Safety Data Sheet from the Midwest homepage: www.dentsplysirona.com

If you have any questions, please contact your dental supplies distributor or the manufacturer.


3 Technical description

3.1 Task

The Midwest Tradition TB/Pro TBF / Midwest Tradition TL/Pro TLF high-speed handpiece is used to power dental instruments with an FG shank.

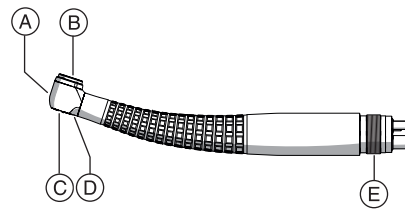3.2 Structure of Midwest Tradition TB


66 27 587 D3673.201.02.01.02

A	High-speed handpiece head
B	Push button
C	Opening of chuck system
D	Cooling spray outlet
E	Fixed connection (in this case: Midwest/ISO 4-hole)

3 Technical description


Operating Instructions Midwest Tradition® TB/Pro TBF / Midwest Tradition® TL/Pro TLF

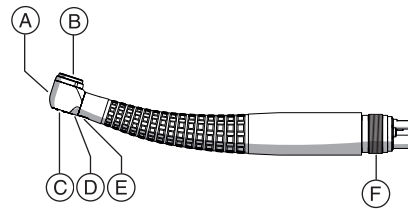3.3 Structure of Midwest Tradition Pro TBF

A	High-speed handpiece head
B	Push button
C	Opening of chuck system
D	Cooling spray outlet
E	Light aperture
F	Fixed connection (in this case: Midwest/ISO 5-hole)

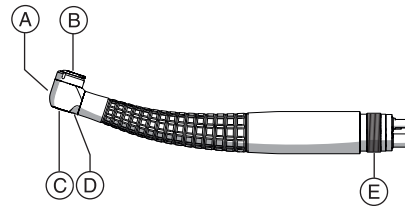
3.4 Structure of Midwest Tradition TL

A	High-speed handpiece head
B	Power Lever
C	Opening of chuck system
D	Cooling spray outlet
E	Fixed connection (in this case: Midwest/ISO 4-hole)

English

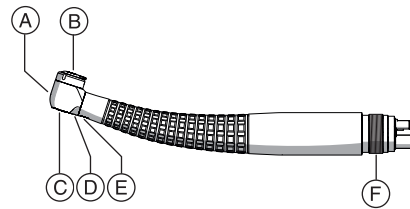3 Technical description

Operating Instructions Midwest Tradition® TB/Pro TBF / Midwest Tradition® TL/Pro TLF


3.5 Structure of Midwest Tradition Pro TLF

A	High-speed handpiece head
B	Power Lever
C	Opening of chuck system
D	Cooling spray outlet
E	Light aperture
F	Fixed connection (in this case: Midwest/ISO 5-hole)

3.6 Technical data

High-speed handpiece Midwest Tradition TB/Pro TBF / Midwest Tradition TL/Pro TLF

	Midwest Tradition TB	Midwest Tradition Pro TBF	Midwest Tradition TL	Midwest Tradition Pro TLF
Light function	-	x	-	x
Back-suction stop in the head housing	x	x	x	x
Anti-retraction valve in the spray water area	-	-	-	-
Push-button cover clamping system	FG	FG	-	-
Power lever clamping system	-	-	FG	FG




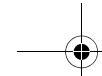
3 Technical description

Operating Instructions Midwest Tradition® TB/Pro TBF / Midwest Tradition® TL/Pro TLF

High-speed handpiece head

	Midwest Tradition TB Midwest Tradition Pro TBF Midwest Tradition TL Midwest Tradition Pro TLF
No-load speed in rpm	410 000 ± 40 000
Maximum torque in Nmm	~ 1.8

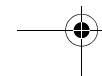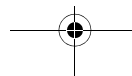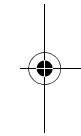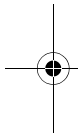Friction grip bur

	Midwest Tradition TB Midwest Tradition Pro TBF Midwest Tradition TL Midwest Tradition Pro TLF
Shank standard	ISO 1797-1, Type 3
Shank diameter in mm	1.59 - 1.60
Maximum total length in mm	21
Maximum working diameter in mm (ISO 2157)	2.1

English

3 Technical description

Operating Instructions Midwest Tradition® TB/Pro TBF / Midwest Tradition® TL/Pro TLF



Treatment center

	Midwest Tradition TB Midwest Tradition Pro TBF Midwest Tradition TL Midwest Tradition Pro TLF
Driving air pressure in bar (psi) (flowing, dry, and clean)	~ 2.3 - 3.0 (34-43)
Driving air consumption in NI/min	52 ± 12
Return air pressure in bar (psi)	< 0.3 (4.35)
Spray air pressure in bar (psi)	~ 2.5 (36)
Spray air flow in NI/min	> 2.5


	Midwest Tradition TB Midwest Tradition Pro TBF Midwest Tradition TL Midwest Tradition Pro TLF
Spray water pressure in bar (psi)	2.0 ± 0.2 (29)
Water content in spray in ml/min	> 50


The indicated pressures can be checked by a service engineer with a star tester that is placed between the supply hose and the high-speed handpiece.

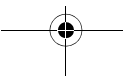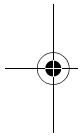English

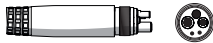3.7 Available high-speed handpiece connection types

	Connection type available for
	Midwest/ISO 4-hole fixed connection
	Midwest/ISO 5-hole fixed connection

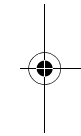Operating Instructions Midwest Tradition® TB/Pro TBF / Midwest Tradition® TL/Pro TLF

3.8 Product labeling

Sterilizable in a steam
sterilizer (autoclave)
at the temperature
specified

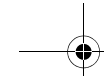
Serial Number

20XX

Manufacturing date


3 Technical description


English

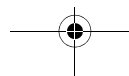66 27 587 D3673.201.02.01.02



25

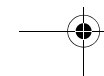3 Technical description

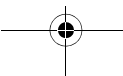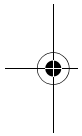Operating Instructions Midwest Tradition® TB/Pro TBF / Midwest Tradition® TL/Pro TLF

Data matrix code (here: an example)

Content of the data matrix code:

+E2768752050/\$\$+5192/16D20170309E

A

B

C

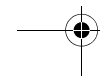
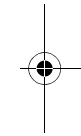
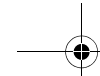
D

A Manufacturer code (here: E276)

B REF number (here: 875205)

C Serial No. (here: 5192)

D Manufacture date (YYYYMMDD)

4 Preparation

4.1 Initial start-up and longer breaks in use

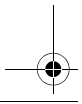
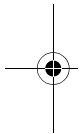
- ✓ The high-speed handpiece is only able to reach full power if indicated operating pressures are set [→ 19]. If necessary have the pressures checked by a service technician with a star tester that is placed between the supply hose and the quick coupling.
- Sterilize the high-speed handpiece and accessory parts prior to startup.
- Clean and maintain the high-speed handpiece after longer breaks in use.

4.2 Prior to starting the work day

- Purge the water and air channels for 30 seconds.




4 Preparation

4.3 Before each patient

1. Purge the water and air channels for 30 seconds.
2. Attach the high-speed handpiece.
3. Insert the bur.
4. Set a sufficient amount of cooling water (> 50 ml/min).
5. Use filtered water only (< 50 μ m).
6. Check the nozzles for blockages and lime deposits, for example, and clean the nozzles if necessary.

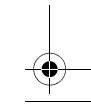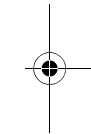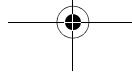⚠ CAUTION! Insufficient cooling leads to overheating of the preparation site and damage to the tooth substance. Ensure that the water content is > 50 ml/min.

5 Operation

NOTICE! Use only burs and diamond polishers that are sharp and undamaged. Use clean burs and diamond polishers to avoid dirt in the clamping system.

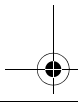
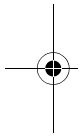
⚠ CAUTION! A loose or partially removed bur can detach itself from the head or break off. This may cause injury! Therefore, use the high-speed handpiece only if the bur is inserted at least 10 mm and is clamped securely in place.

⚠ CAUTION! Insufficient cooling leads to overheating of the preparation site and damage to the tooth substance. Ensure that the water content is > 50 ml/min.

⚠ CAUTION! Do not pull the patient's cheek back with the handpiece while the high-speed handpiece is rotating. This would actuate the pushbutton, thus creating a risk of burning the patient's oral mucosa.

5 Operation

30

Operating Instructions Midwest Tradition® TB/Pro TBF / Midwest Tradition® TL/Pro TLF

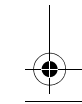
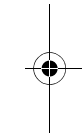
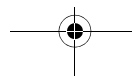
⚠ CAUTION! A loose or partially removed bur can detach itself from the head or break off. This may cause injury! Therefore, use the high-speed handpiece only with a closed power-lever.

⚠ CAUTION! Stop using the high-speed handpiece, if the power lever does not stay closed by itself. If the chuck system is not completely closed, this could actuate the mechanics and lead to overheating of the high-speed handpiece head. There is a risk of burning the patient's oral mucosa.

IMPORTANT: For application details and operating data of the different preparation tools, please refer to the information provided by the manufacturer.

5.1 Replacing the high-speed handpiece

⚠ CAUTION! Do **not** remove the handpiece while the high-speed handpiece is running.


5.1.1 High-speed handpiece with fixed connection



Connecting the high-speed handpiece

1. Attach the high-speed handpiece to the supply hose, observing the various tube diameters.
2. Screw the screw cap (A) hand-tight. Hold the high-speed handpiece securely when doing this.

Removing the high-speed handpiece

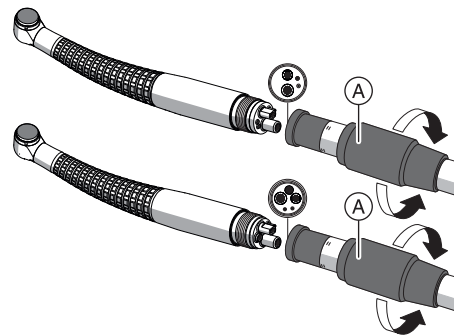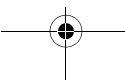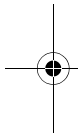1. Unscrew and remove the screw cap.
2. Pull the high-speed handpiece off of the supply hose.


5.2 Inserting and removing burs



Operating Instructions Midwest Tradition® TB/Pro TBF / Midwest Tradition® TL/Pro TLF

IMPORTANT: Check the push button to make sure it moves freely!

NOTICE! Use only burs and diamond polishers that are sharp and undamaged. Use clean burs and diamond polishers to avoid dirt in the clamping system.

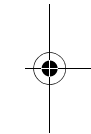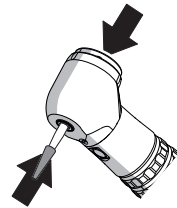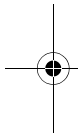⚠ CAUTION! A loose or partially removed bur instrument can detach itself from the head or break off. This may cause injury! Therefore, use the high-speed handpiece only if the bur instrument is inserted at least 10 mm and is clamped securely in place.





5.2.1 Push-button clamping system

Inserting the friction grip bur

- ✓ The high-speed handpiece is at a standstill.
- 1. Press the button and insert the bur up to the stop.
- 2. Pull on the bur to check that it is firmly attached.

Removing the friction grip bur

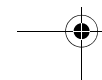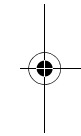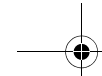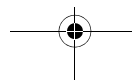- ✓ The bur must not be moving.
- Press the button and remove the bur.

5.2.2 Power lever clamping system

Inserting the friction grip bur

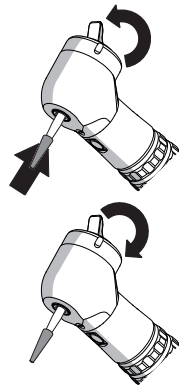
✓ The high-speed handpiece is at a standstill.

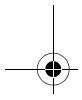
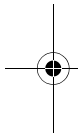
1. Push the power lever up and insert the bur up to the stop.
2. Push the power lever downwards and position it into direction of the sleeve.
3. Pull on the bur to check that it is firmly attached.

Removing the friction grip bur

✓ The bur must not be moving.

1. Push the power lever up and remove the bur.
2. Push the power lever downwards.

6 Follow-up

6.1 After each treatment session

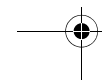
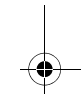
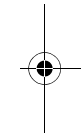
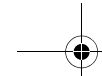
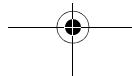
NOTICE! Condition immediately, or at the latest, one hour after treatment.

NOTICE! Only use Midwest Plus Aerosol Spray.

✓ The bur must not be moving.


✓ Wear appropriate protective clothing.

1. Purge the water and air channels on the treatment center for 30 seconds.
2. Remove the bur with tweezers.
3. Predisinfect directly at the treatment center [→ 37].
4. Remove the high-speed handpiece.
5. Transport the high-speed handpiece to the hygiene room in a suitable transport container.


6 Follow-up

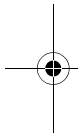36

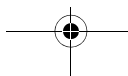
Operating Instructions Midwest Tradition® TB/Pro TBF / Midwest Tradition® TL/Pro TLF

6. Conduct automatic reprocessing. Manual conditioning is possible in exceptional cases if the national/local regulations are followed.
7. Lubricate the high-speed handpiece [→ 41].
8. Sterilize the high-speed handpiece and the accessories [→ 47].

6.2 At the end of the work day

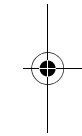- Lubricate the high-speed handpiece [→ 41].

7 Conditioning

7.1 Conduct pre-disinfection

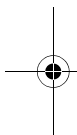
- ✓ Wear appropriate protective clothing.
- ✓ All disinfectants must be approved in your country and have proven bactericidal, fungicidal and virucidal properties. Only use disinfectants with **no** protein-fixing properties.

1. Spray the surface with disinfectant.
2. Wipe the disinfectant off with a cloth.

↪ For further processing, the high-speed handpiece should be dry and free of residue.

In USA and Canada use **only** following agents:

- CAVICIDE®
- CAVIWIPES™



Please observe the manufacturer's instructions for using instrument disinfectants.



7.2 Automated cleaning

The cleaning equipment used must be approved by its manufacturer for the cleaning of dental instruments.

For further details, refer to the operating instructions supplied with the unit.

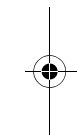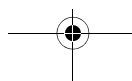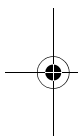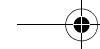- ✓ The high-speed handpiece is conditioned with a cleaning device.
- 1. Check whether the high-speed handpiece is clean after conditioning under good lighting (min. 500 lux) and color rendering index (min. 80 Ra).
- 2. If they are still dirty, repeat the process.
 - ↪ For further processing, the high-speed handpiece should be dry and free of residue.

3. Blow the high-speed handpiece out with max. 3 bar.
4. Maintain mechanical parts manually [→ 41].
5. Maintain the push button chuck manually [→ 43].
6. Pack the high-speed handpiece in packaging material suitable for sterilization and storage. e.g. paper/plastic composite packaging.
7. Conduct sterilization [→ 47].

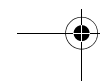
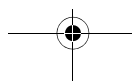
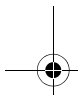
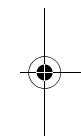
7.3 Manual cleaning

IMPORTANT: Manual conditioning is possible in exceptional cases if the national/regional regulations are followed. The national/regional regulations are to be checked before.


NOTICE! Condition immediately, or at the latest, one hour after treatment.

NOTICE! Never clean in an ultrasound bath!

NOTICE! Never immerse in disinfectant solution!

7 Conditioning

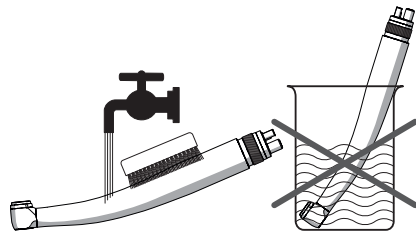Operating Instructions Midwest Tradition® TB/Pro TBF / Midwest Tradition® TL/Pro TLF

NOTICE! Only use Midwest Plus Aerosol Spray.

IMPORTANT: Use a soft, clean, and disinfected brush for cleaning.

✓ Wear appropriate protective clothing.

1. Brush the high-speed handpiece under running water ($< 38^{\circ}\text{C}$, $< 100^{\circ}\text{F}$, at least drinking water quality) and good lighting (min. 500 lux) and color rendering index (min. 80 Ra) until no more dirt can be seen, for at least 10 seconds.
2. Flush the drive channels with spray.
3. Maintain mechanical parts manually [→ 41].
4. Maintain the push button chuck manually [→ 43].
5. Pack the high-speed handpiece in packaging material suitable for sterilization and storage. e.g. paper/plastic composite packaging.
6. Conduct sterilization [→ 47].

7.4 Manual maintenance

7.4.1 Maintenance of mechanical parts

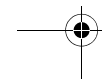
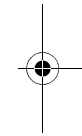
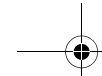
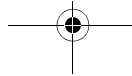
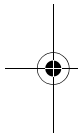
Intervals

- After each patient
- Prior to each sterilization
- During a longer preparation (> 10 minutes)


Required accessories

- A Black nozzle insert
B Midwest Plus Aerosol Spray

NOTICE! Only use Midwest Plus Aerosol Spray.

7 Conditioning

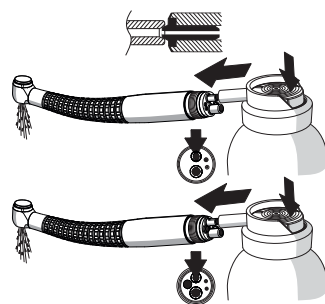Operating Instructions Midwest Tradition® TB/Pro TBF / Midwest Tradition® TL/Pro TLF

Process

Highspeed handpiece with fixed connection

✓ The black nozzle insert is in the spray can tube.

1. Take the highspeed handpiece off of the supply hose.
2. Insert the nozzle of the spray can in the driving air tube. Hold the highspeed handpiece securely with a cloth when doing this.
3. Spray into the high-speed handpiece for 1-2 seconds.
IMPORTANT: Hold the spray can upright.
4. Wipe any spray that comes out with a disinfection cloth.
5. Repeat the process until the spray is clear.

7.4.2 Care of the push button chuck


Use Midwest Plus Aerosol Spray on the push button chuck to remove deposits and ensure proper functioning of the clamping system.

Interval

- At least once a week


Required accessories

A	Black nozzle insert
B	Midwest Plus Aerosol Spray

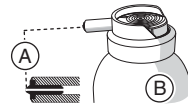NOTICE! Only use Midwest Plus Aerosol Spray.

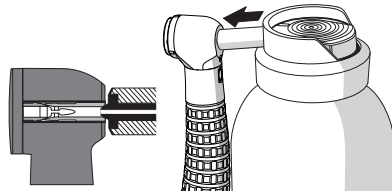7 Conditioning

Operating Instructions Midwest Tradition® TB/Pro TBF / Midwest Tradition® TL/Pro TLF

Process

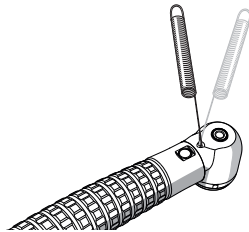
- ✓ The high-speed handpiece is cleaned and disinfected.
- 1. Press the high-speed handpiece head with the chuck firmly against the spray can nozzle.
- 2. Spray the chuck for 1 - 2 seconds.
IMPORTANT: Hold the spray can upright.
- 3. Wipe any spray that comes out with a disinfection cloth.

7.4.3 Care of cooling spray nozzle openings


If your tap water is very hard, lime deposits may constrict or completely block the cooling spray nozzles.

Interval

- At least once a week
1. Carefully clean the nozzle openings by running a cleaning wire through them.
 2. Let the high-speed handpiece run briefly with cooling spray.

7 Conditioning

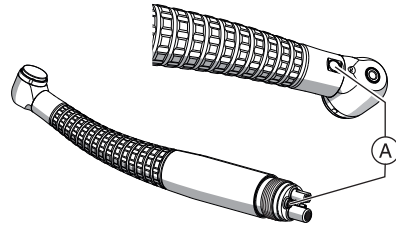Operating Instructions Midwest Tradition® TB/Pro TBF / Midwest Tradition® TL/Pro TLF

7.4.4 Cleaning the light guide surfaces

Check the fiber optic surface and clean it if needed.

1. In order to avoid scratching the surfaces (A), blow off any dirt particles with air using a dry syringe.
2. Wipe the surfaces with a Q-tip or a soft cloth and alcohol.

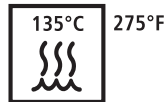
7.5 Sterilizing

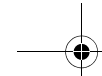
Intervals:

- Prior to initial operation
- Prior to each subsequent use

Process

- ✓ The high-speed handpiece is cleaned.
 - ✓ High-speed handpiece maintenance is complete.
 - ✓ If necessary, the high-speed handpiece can be sterilized in packaging suitable for sterilization and storage, e.g. paper/plastic composite packaging or container.
- Sterilize the high-speed handpiece in the steam sterilizer with saturated water vapor.



7 Conditioning

Operating Instructions Midwest Tradition® TB/Pro TBF / Midwest Tradition® TL/Pro TLF


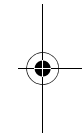
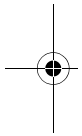
Temperature: 135 °C (275 °F)
Overpressure: 2.13 bar (30.89 psi)




Article	Holding time at 135 °C (275° F)	Drying time
Wrapped instruments	10 minutes	30 minutes
Unpackaged instruments	3 minutes	0 - 1 minute

Gravity displacement steam sterilizers are permitted.

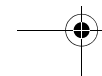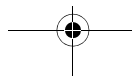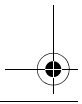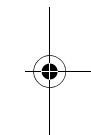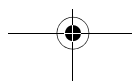
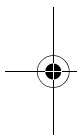
NOTICE! Do not exceed 140 °C (284 °F), even during the drying phase.

After sterilizing

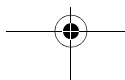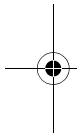1. Remove the high-speed handpiece from the steam sterilizer immediately.
⚠ CAUTION! The high-speed handpiece will be hot. This may cause burns!
NOTICE! Do **not** immerse the high-speed handpiece in cold water to speed up the cooling process. This will damage your high-speed handpiece!
2. Store all high-speed handpieces so that they are protected from contamination.
3. Sterilize again once the storage period has elapsed.

8 Maintenance

Elastomers, e.g. O-rings, must be replaced depending on their degree of wear.

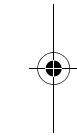
8.1 Testing the FG clamping system

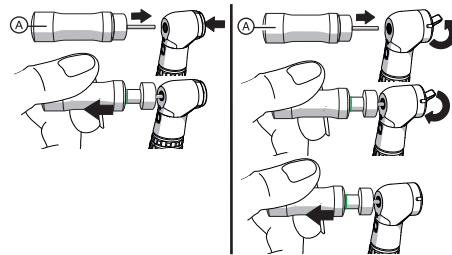
Interval

Test the FG clamping system at least once a month.

Required accessories

Chuck tester

Process

✓ The expiry date of the chuck tester (A) has **not** elapsed.

1. Insert the chuck tester into the FG clamping system.
2. Tighten the chuck tester until the marking ring appears (withdrawal force: 22 N)

Does the chuck tester slide out of the chuck before the marking ring appears?

⚠ **CAUTION!** The chuck is defective and the secure fit of the bur is not guaranteed. This may cause injury!

1. Do not use the product.
2. Have the clamping system replaced by Midwest.

Tip: Record the time and result of the check for your own information.

9 Spare parts and consumables

	REF			REF
Midwest Plus Aerosol Spray	33 00 80		Cleaning wire for spray nozzles	24 00 232
Sealing washer, 4-hole	58 55 460		Chuck tester	33 27 793
Sealing washer, 5-hole	66 30 748			

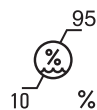
10 Storage and transport conditions

Protect from
moisture


Sensitive
contents

Relative
humidity


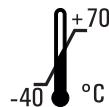
Air pressure


Temperature

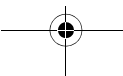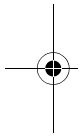After a severe change in
temperature, allow sufficient
time for acclimation.

11 Disposal

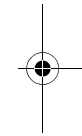- According to current information, the product does not contain any substances that are hazardous to the environment.
- Disinfect the product prior to disposal.
- Observe the applicable disposal regulations for your area.

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Sirona Dental Systems GmbH