Dentsply Sirona

Aquasil® Ultra Smart Wetting® Abformmaterial

Aquasil Ultra XLV **Aquasil Ultra Monophase** **Aquasil Ultra Rigid**
Aquasil Ultra LV **Aquasil Ultra Heavy** **Aquasil Ultra Xtra**

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- Dispense the material by applying slow, steady pressure on plunger. Do not use excessive force.
- To disassemble, withdraw the plunger, tap mixing tip against the coupler to dislodge, and remove through side opening.
- Properly discard used cartridge/mixing tip and prepare syringe for reuse.

3.3 DECA® cartridge loading

NOTE: Heavy cartridge - dropping may result in damage to the cartridge rendering it unusable. The DECA® Cartridge fits Duomix, duomix II, Pentamix, MixStar, Dynamix and Volume Mix dynamic mixers. Carefully remove dispensing cartridge from the package. Also refer to separate dynamic mixer manufacturer's instructions.

- Remove the cap from the cartridge.
- Insert the cartridge into the dynamic mixer.
- Bleed the cartridge before installing a new mixing tip.
- Install a mixing tip. Use only DENTSPLY mixing tips. Other tips may not seal properly.
- Place locking ring over the mixing tip and turn until ring is locked in place.
- Trim material (assistant) into intraloral syringe (Aquasil Ultra Monophase only) or a suitable tray.
- Immediately prior to subsequent usage**, remove the used mixing tip, bleed cartridge, install a new mixing tip.

3.4 Impression technique: Single or Dual Phase one-step ("double-mix") Technique with a Rigid Tray or with Double Arch Technique.

NOTE: Tray and wash material mix timing - It is highly recommended to utilize an assistant to assure simultaneous syringing and tray loading.

2. Assume hemostatis and clean the field with air water spray.

3. Syringe material: Backfill the impression syringe or attach an intraoral tip to the end of the mixing tip, or prepare digit® Targeted Delivery System unit dose dispensed syringe.

Tray material (assistant) into impression tray. Do not overfill tray.

3. Simultaneous with tray loading, syringe (dentist) completely around the preparation/abutment.

NOTE: Tray and wash material insertion timing:

- Super Fast Set wash tray material: Loaded tray must be inserted within 35 seconds from time of first syringing of wash material intraorally.
- Fast Set wash and tray material: Loaded tray must be inserted within 35 seconds from time of first syringing of wash material intraorally.
- Regular Set wash and tray material: Loaded tray must be inserted within 1 minute 10 seconds from time of first syringing of wash material intraorally.
- Aquasil Ultra Xtra extended time wash and tray material: Loaded tray must be inserted within 1 minute 45 seconds from time of first syringing of wash material intraorally.

4. Insert filled tray into the mouth.

5. Minimum Removal Time (from the start of mix)

- Super Fast Set: 2 minutes 30 seconds
- Fast Set: 3 minutes
- Regular Set: 5 minutes
- Aquasil Ultra Xtra: 5 minutes 30 seconds

NOTE: If Regular Set Wash is used with Fast Set Tray material, Minimum Removal Time is 4 minutes from start of mix.

6. Follow disinfection procedures before shipping and/or pouring.

Timing Guide	Super Fast Set: up to 0'35" (35 sec)	Super Fast Set: 2'30" (2 min 30 sec)
	Fast Set: up to 0'35" (35 sec)	Fast Set: 3'00" (3 min)
	Regular Set: up to 1'10" (1 min 10 sec)	Regular Set: 5'00" (5 min 10 sec)
	Aquasil Ultra Xtra: up to 1'45" (1 min 45 sec)	Aquasil Ultra Xtra: 5'30" (5 min 30 sec)
000'0" (0 sec)		
Begin simultaneous syringing and tray loading	Complete syringing and seat loaded tray	Remove set impression

3.5 Impression technique: Dual Phase two-step ("putty-wash") Technique with a primary tray impression

1. Complete primary impression with Aquasil Ultra Xtra Tray, Aquasil Putty or Reprosil Tray material.

2. Place one or more polyethylene plastic sheets over the impression material surface. This sheet creates the uniform spacing required for this technique. It also protects the surface from contamination.

3. Insert filled tray into the mouth. Move tray slightly from side to side and back and forth. This creates the spacing for the wash material.

4. Remove tray from the mouth when material has reached a rubbery consistency and set aside. Do not disturb impression until fully set.

5. Remove polyethylene sheet, rinse impression with water and air dry.

2. It is recommended to utilize an assistant and two cartridge guns, or one cartridge gun and digit® Targeted Delivery System unit dose dispensed syringe to assure simultaneous syringing and tray loading.

3. Follow impression steps above, using wash material for both intraoral syringing and loading into primary tray impression.

3.6 Impression Technique: Edentulous/Overdenture Impression Technique.

Aquasil Ultra Monophase, Aquasil Ultra Heavy and Aquasil Ultra Rigid are acceptable for border molding.

- Border Molding Technique
 - Dispense material on to the borders of the tray.
 - Rinse the tray and perform border molding techniques.
 - When material has reached a firm consistency, remove tray and examine borders. The addition of a small mix can be used if voids or deficient areas are present. Over-extensions can be easily removed with a scalpel blade.
- Final impression can be made with Aquasil Ultra XLV or LV, Aquasil Ultra Monophase, Aquasil Ultra Heavy or Aquasil Ultra Xtra Material.
- Remove model from impression by returning slowly to break seal.

3.7 Impression technique: Edentulous/Overdenture Impression Technique.

1. Load the impression tray with Aquasil Ultra tray material.

2. It is suggested to simultaneously syringe, using Aquasil Ultra wash material, completely around the model teeth.

3. Set model into the loaded tray, retaining impression in position until firmly set.

4. Remove model from impression by pulling slowly to break seal.

4. HYGIENE

4.1 Cross-contamination

- Do not reuse single use products. Dispose in accordance with local regulations.
- Recross process reusable products as described below.

4.1 Cleaning the dispenser

The cartridge dispensing guns and digit® Targeted Delivery System syringe may be cleaned by scrubbing with hot water and soap or detergent. Excess material may be cleaned with alcohol-moistened gauze. Refer to Dynamic Mixer manufacturer's instructions for dispenser cleaning and disinfection instructions.

4.2 Disinfection of the dispenser gun and cartridge and digit® Targeted Delivery System syringe

Disinfect dispenser guns, cartridges and used tips left in place on cartridges for storage and digit® Targeted Delivery System syringe with a hospital-level tuberculocidal disinfectant according to national and local regulations. Use impregnated wipes not sprays. **NOTE:** Vigorous wiping can destroy the label. Wipe cartridge gently. Some phenolic based agents and iodophor based products may cause surface staining.

Agents containing organic solvents, such as alcohol may tend to dissolve the plastic dispenser and plunger. The disinfectant manufacturer's directions should be followed properly for optimum results.

4.2 Sterilization

Following cleaning and disinfection, digit® Targeted Delivery System syringe may be steam autoclaved at 134°C. Minimum 3 minute holding time. **Technique Tip:** Unset Aquasil Ultra Smart Wetting Impression Material can be removed from clothing with dry cleaning solvent.

4.3 Disinfection of the impression material

The impression should be disinfected with a hospital-level tuberculocidal disinfectant according to national and local regulations.

- Iodophors, sodium hypochlorite (0.525%), chlorine dioxide, and dual or synergized quaternary ammoniums are approved disinfectants.
- Water-based gels such as spraying or immerse the impression in any recommended hospital level disinfectant for the contact time recommended by the disinfectant manufacturer for optimum results.
- Water-based disinfectant solutions are preferred.

4.4 Casting

The impression should be removed from the disinfectant, rinsed with water and lightly air dried.

- The impression may be poured in plaster or stone 30 minutes after disinfection or delayed up to fourteen days. For epoxy dies, delay pour 60 minutes.
- Do not attempt a direct toothright.
- If the impression is to be shipped, use suitable packaging to preclude distortion.
- The material is compatible with a range of dental plasters and die stones. Impressions may be silver or copper-plated.

5. LOT NUMBER AND EXPIRATION DATE

Do not use after expiration date. ISO standard usage: "YYYY/MM".
The following numbers should be quoted in all correspondences:

- Reader number
- Lot number
- Expiration date

<div> <div></div> <div>Manufactured by Dentsply Caulk</div> <div>38 West Clarke Avenue Milford, DE 19963 USA</div> <div>Made in USA</div> <div>1-302-422-4511</div> <div>www.dentsplysirona.com</div> </div>	<div> <div></div> <div>EC REP</div> <div>De Trey GmbH</div> <div>De-Trey-Strasse 1 78467 Konstanz</div> <div>Germany</div> <div>49-7531-983-0</div> <div>www.dentsplysirona.com</div> </div>	<div> <div></div> <div>Distributed by DENTSPLY® Canada</div> <div>1610 Woodbridge, Ontario</div> <div>L4L 4A3 Canada</div> <div>Germany</div> <div>49-7531-983-0</div> <div>www.dentsplysirona.com</div> </div>	<div> <div></div> <div>578002 (R 10/19/17)</div> </div>
---	--	--	---

Pentamix, MixStar, Dynamix and Volume Mix are not registered trademarks of DENTSPLY International.

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Clasificación: ISO 4023

Tipo 3: Consistencia Light-bodied: Aquasil Ultra XLV, Aquasil Ultra LV and Aquasil Ultra Xtra (wash)
Tipo 2: Consistencia Medium-bodied: Aquasil Ultra Monophase, Aquasil Ultra Heavy and Aquasil Ultra Xtra (tray)
Tipo 1: Consistencia Heavy-bodied: Aquasil Ultra Rigid

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