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Treatment centers

Materials and general instructions.

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DESIGN, TECHNOLOGY AND QUALITY – AN INTEGRATED UNIT

Sirona is committed to maximizing user benefits. For this reason it is essential to gear our treatment centres to your individual mode of working, ergonomic requirements and sustained well-being. How do we go about this? By delivering outstanding user-friendliness, quality, technical innovations, flexible equipment options as well as excellent design. The outcome is a unit that engenders your full confidence. We would like to thank you for the trust you place in us.

You can look forward to receiving a treatment centre which caters for your unique personality and sense of clinical responsibility. **Enjoy every day. With Sirona.**

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DESIGN

Your commitment to the highest level of quality and aesthetics expresses itself directly in the design of our treatment centres. This is manifested in the perfect interaction between form and function, coupled with outstanding stability. Modern and unobtrusive design features streamline the user's workflow and make a positive impact on the comfort and well-being of the patient. Our products have won four prestigious design awards. The Multi-Motion headrest and the TENEO treatment centre number among the best designed products – in any business sector. This conclusion was reached by the international jury of the iF design awards. In addition, the TENEO user interface has been singled out for the iF communication design award. And the SINIUS treatment centre has received the "red dot" award.

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MATERIALS

The right choice of materials has a determining impact on quality. We combine tried-and-tested materials and new alternatives. This results in a level of quality that you can see and feel. We ensure stringent quality standards, durability, stability and aesthetics – right down to the very last detail. A further decisive advantage is that our treatment centres are “Made in Germany”. Development, production and quality assurance all take place at our factory in Bensheim.

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GENERAL INSTRUCTIONS – CARE AND CLEANING

You have acquired a product which exemplifies outstanding quality and design. In so doing you have demonstrated your discerning standards with regard to materials, design and ease of use. This brochure contains important information about materials, their special properties, as well as care and cleaning recommendations. These recommendations supplement, but do not replace, the information contained in our operating instructions. Further details can be found in the document „Care and Cleaning by the Practice Team“. A continuously updated list of approved detergents, disinfectants and care products can be downloaded from the Sirona website (Service > Care and cleaning).

National hygiene and disinfection requirements/guidelines must be observed (e.g. RKI, ADAS, CDC, etc.).

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GENERAL INSTRUCTIONS

To ensure long-term user satisfaction your Sirona treatment centre requires regular cleaning, care and disinfection. Special recommendations apply.

Please pay attention to the following:

- Avoid any contact with rough surfaces and sharp objects.
- The earlier contamination is detected, the easier it will be to remove.
- Stains and medicinal product residues must be removed immediately.
- You should use only those care and cleaning products that have been recommended by Sirona.
- Please pay attention to detailed information contained in the operating instructions.

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UPHOLSTERY

Properties

The look and feel of the upholstery depends on a smart combination of materials. The surface is absolutely waterproof and thus hygienic. The durability of the upholstery plays a major role in maintaining the softness and ergonomic shape of the seat cushion and backrest. Our upholstery materials are selected on the basis of clinical and functional criteria and then processed and assembled using perfect craft skills. Subject to regular care and cleaning, the upholstery will retain its original sheen and suppleness.

Basic factors:

- Light-coloured upholstery is more prone to dirt accumulation than darker coloured equivalents.
- Patients should never be seated on damp upholstery.
- The upholstery should always be thoroughly dried.

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UPHOLSTERY

Discolouration

Certain dyes contained in clothing (e.g. jeans) can discolour and penetrate coated upholstery fabrics. Leather belts and suede garments are particularly aggressive in this respect. Unless detected and cleaned immediately, such discolourations will become permanent.

Care and cleaning

- After each patient has been treated check to ensure that no discolouration has occurred. If necessary, thorough cleaning should be carried out.
- Care and cleaning procedures must be performed at least once a week. Light-coloured upholstery requires daily care and cleaning.
- The upholstery must be disinfected after each patient. Please allow sufficient contact time for the disinfectant and then dry the upholstery thoroughly.

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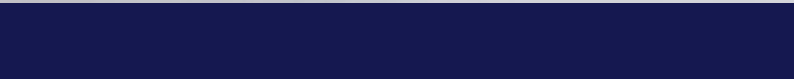
UPHOLSTERY

Accumulations of dirt

Over a period of time greyish-black accumulations of dirt may occur – especially if regular cleaning has not been carried out and/or if unsuitable detergents and utensils have been deployed.

Care and cleaning

- After each patient has been treated check to ensure that no dirt has accumulated. If necessary, thorough cleaning should be carried out.
- Irrespective of this, care and cleaning procedures must be performed at least once a week.
- The upholstery must be disinfected after each patient. Please allow sufficient contact time for the disinfectant and then dry the upholstery thoroughly.

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PAINTWORK

Properties

The surfaces of our high-quality products are either spray-painted or powder-coated. Metallic finishes create a brilliant and lively effect. Despite their inherent hardness, the paintwork surfaces are prone to wear and tear. For this reason all contact with rough and sharp objects should be avoided.

Care and cleaning

- Remove any dirt and disinfectant residues with the aid of a mild detergent and then wipe dry.
- Do not clean and disinfect the paintwork surfaces with the aid of dyed wipes. Remove medicinal product residues immediately using a damp cloth.

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GLAS

Properties

The glass cuspidor bowl creates a light, translucent impression. Fingerprints remain invisible due to the bowl's frosted surface. Although glass is a relatively robust material, it is nonetheless prone to scratches. For this reason avoid any contact with rough surfaces, sand grains and sharp metal objects.

Care and cleaning

- Please observe the operating instructions when cleaning the monitor, the SIROLUX operating light and the glass cuspidor bowl.

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PLASTICS

Properties

Plastics adapt themselves to a wide range of different requirements. Despite their low weight, they are elastic, robust and durable. Plastics are pleasant to the touch, resistant to slipping and do not cause allergies. Above all, they are easy to clean and disinfect.

Care and cleaning

- Remove any dirt and disinfectant residues with the aid of a mild detergent and then wipe dry.
- Do not clean and disinfect the plastic surfaces with the aid of dyed wipes.
- Remove medicinal product residues immediately using a damp cloth.

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STAINLESS STEEL AND ENAMEL

Properties

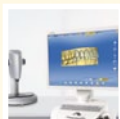
Stainless steel is a high-strength material which is corrosion-resistant and easy to clean. Enamel is a durable vitreous material which is resistant to chemicals and medicinal products. In the case of the cuspidor bowl an enamel coating is applied to a steel surface. As is the case with glass materials avoid any contact with rough surfaces, sand grains and sharp metal objects.

Care and cleaning

- Remove any dirt and disinfectant residues with the aid of a mild detergent and then wipe dry.
- Clean the enamel cuspidor bowl in accordance with the operating instructions.

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