

inLab 3D SERVICE PACK V3.87

# Update description

The Service Pack V3.87 will become available at the beginning of August. It is recommended only for those users who frequently encounter fractures in zirconium oxide restorations, despite using original materials. Following the installation of V3.87, frameworks (design mode "framework" or "reduced") are proposed with a wall thickness of 0.6 mm (instead of 0.5 mm). The frameworks can be reduced with the aid of the usual design tools. Coloured markings will be shown only if the wall thickness is less than 0.5 mm. This Service Pack does not contain any further modifications.

## Installation requirements

The Service Pack V3.87 does not contain the modifications included in the previous service packs. For this reason, Service Pack V3.86 must be installed first, followed by Service Pack V3.87. To undo the installation of V3.87, we recommend that Service Pack V3.86 is installed directly.

To receive regular information about the latest product developments we recommend that you subscribe to the inLab Newsletter under [www.sirona.com/inlab-newsletter](http://www.sirona.com/inlab-newsletter).

August 2011