

Software update for SIROLaser – Advance und Xtend – via USB 2.0 stick

Software download

- Before starting the software update please delete all files on your USB stick 2.0.
- Click on link below to start the download process.

▶ **Software update for SIROLaser – Advance & Xtend Vx.x.x**
(mm.dd.yyyy) (xx MB)

- Please choose option 'Start'.
- Extract the content of the downloaded file directly to your USB stick.

Please notice:

- The software can be used to update the SIROLaser Advance and SIROLaser Xtend.
- This software update includes all features of the former software versions.
- The new software automatically applies your current settings.
- The USB stick and the power cable should remain connected until the software update is finished.
- The software update may need up to 5 minutes.

Software download 'additional languages'


- If you are interested in switching the language setting of the SIROLaser Advance/Xtend to Czech, Greek, Hungarian, Polish, Rumanian, Slovakian or Slovenian please click on link below:

▶ **Software 'additional languages' for SIROLaser – Advance & Xtend' Vx.x.x**
(mm.dd.yyyy) (xx MB)


- Please proceed with download and installation as described for the regular software update.

Software installation

1. Go to 'Set up' in main home screen.

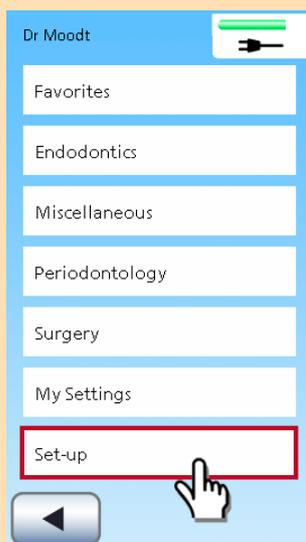

2. Select 'software update'.

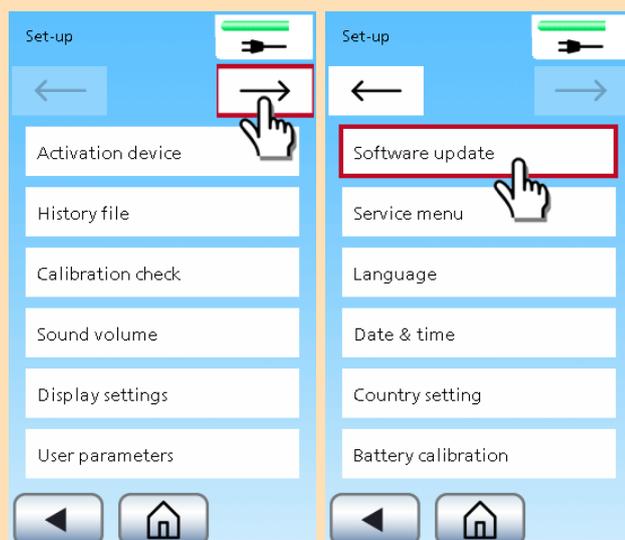


3. Follow the instructions and insert the USB stick.

- The system might need some time to recognize the USB stick.
- **The USB stick and the power supply must stay connected until the software update is finished.**

4. Press 'OK'.



5. The software update now takes place and may need up to 5 minutes.

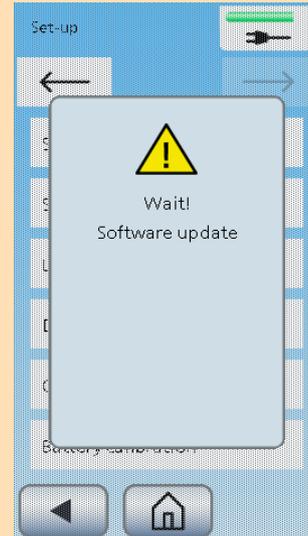

- When the software download is finished, the yellow LEDs first light up permanently, then they are flashing. Also, the fan is active several times.
- Then, the SIROLaser – Advance or Xtend automatically restarts again.
- Please remove the USB stick.
- Now your SIROLaser – Advance or Xtend is ready again.

Possible information screens

- Please close this screen by pressing the arrow key.



- Please press OK, change the USB stick and start the software update process again.



- Connect the power supply and press OK.

