







Processing times Materials for CEREC SpeedFire (in min.)

Manufacturer	Material Name	Dry*	Dry* Multi	Sinter Single Crown (dry)*	Sinter Multi Crown (dry)*	Sinter Single Crown (wet)*	Sinter Multi Crown (wet)*	Sinter bridge (dry)*	Sinter bridge (wet)*	Crystalize/ Cryst(multi)/Cryst/ Glaze/Cryst/GI Multi	Glaze/ Glaze mult*	
	inCoris ZI meso	12	-	35,5	-	47,5	-	-	-	-	-	
	inCoris TZI	12	12	18,5	32	30	48,5	32	48,5	-	9	
	inCoris TZI C	12	12	13,5-17	25	25,5-29	41,5	25	41,5	-	9	
	CEREC Zirconia	12	12	13,5-17	25	25,5-29	41,5	25	41,5	-	9	
	CEREC Zirconia Meso	12	-	25	-	41,5	-	-	-	-	-	
	CEREC BlocsC	-	-	-	-	-	-	-	-	-	-	11,5
	CEREC Blocs C PC	-	-	-	-	-	-	-	-	-	-	11,5
	CEREC Blocs C In	-	-	-	-	-	-	-	-	-	-	15,5
	CELTRA Duo	-	-	-	-	-	-	-	-	-	-	11
	KATANA Zirconia (STML)	12	12	18,5-31	18,5-31	31,5-43	31,5-42,5	43	43	-	8,5	
	VITA Mark II	-	-	-	-	-	-	-	-	-	11,5	
	VITA SUPRINITY	-	-	-	-	-	-	-	-	34,5	12,5	
	VITA RealLife	-	-	-	-	-	-	-	-	-	11,5	
	VITA TriLuxe	-	-	-	-	-	-	-	-	-	11,5	
	VITA TriLuxe forte	-	-	-	-	-	-	-	-	-	11,5	
	VITA YZ HT	12	12	14-17	25	26-29	41,5	25	41,5	-	-	
	Chairside Zirconia	12	12	19,5-24	-	33-35,5	-	24,5	35,5	-	9	
	IPS e.max CAD LT,MT,HT	-	-	-	-	-	-	-	-	24-39	20-34,5	
	IPS e.max CAD MO, Impulse	-	-	-	-	-	-	-	-	30,5-42	20-34,5	
	IPS e.max CAD Abutment LT	-	-	-	-	-	-	-	-	24-39	20-34,5	
	IPS e.max CAD Abutment MO	-	-	-	-	-	-	-	-	30,5-42,5	20-34,5	
	IPS e.max ZirCAD LT	11-15	11-15	19-28,5	28,5	29:5-44	43,5	28,5	44	-	17	
	IPS e.max ZirCAD MT	11-15	11-15	49-2h21,5	49	60-2h37	1h4	2h21	2h37	-	17	
	Initial LiSi Block	-	-	-	-	-	-	-	-	-	12	

* Sinter-/Glazier- und Kristallisierungszeiten sind von der Restaurationsdicke und der Farbe abhängig
Sintering-/glazing- and crystallization times depend on the restoration's thickness and color

** Dentsply Sirona ist nicht für die Prozesszeiten der einzelnen Partnermaterialien verantwortlich, diese wurden von den jeweiligen Materialpartnern validiert und entsprechend von Dentsply Sirona ins System überführt.
Dentsply Sirona is not responsible for the process times of the individual partner materials. These have been validated by the respective material partners and transferred to the system by Dentsply Sirona.