

Primescan Autoclavable Sleeve

Quick Reference Guide

Pre-Use



Perform sterilization process of the autoclavable sleeve according to manufacturer specifications of the autoclave.

Use on Patient



Remove a disposable window from its bag for every patient.



Insert the window into the recess until it snaps and sits firmly.



Remove the white protective sleeve from the scanner head.



Carefully re-attach the steel sleeve until it locks in place.



Position the scanner in the cradle and start using as soon as it's warmed up.

Note: Make sure that the cradle has been disinfected with a wipe disinfectant.



For every batch (50 disposable windows), a calibration needs to be carried out with the autoclavable sleeve, prior to the first scan. After calibration, discard the window and wipe the sleeve with a disinfectant.

Sterilization Post-Use



Wipe the sleeve and the scanner with a wipe disinfectant.



Remove the sleeve from the scanner and the disposable window from the sleeve. Afterwards, discard the disposable window according to the local standard procedure for waste disposal.



Wipe the cradle with a wipe disinfectant, place the white protective sleeve on the scanner head and place the scanner in the cradle.



Repeat the process of pre-use sterilization.



For all references please see the relevant chapters in the current Instructions for Use for CEREC Primescan AC.