

Azento®


CBCT to Azento®

- Export protocol for Morita


This protocol gives a detailed description of the steps to be followed for exporting your images from the i-DIXEL software in DICOM format. The export protocol is usable with the i-DIXEL software version 2.0 or higher. This protocol aims at an optimal visualization of your images.

## Part 1: i-DIXEL software


1. Open 3D image in CT Viewer.

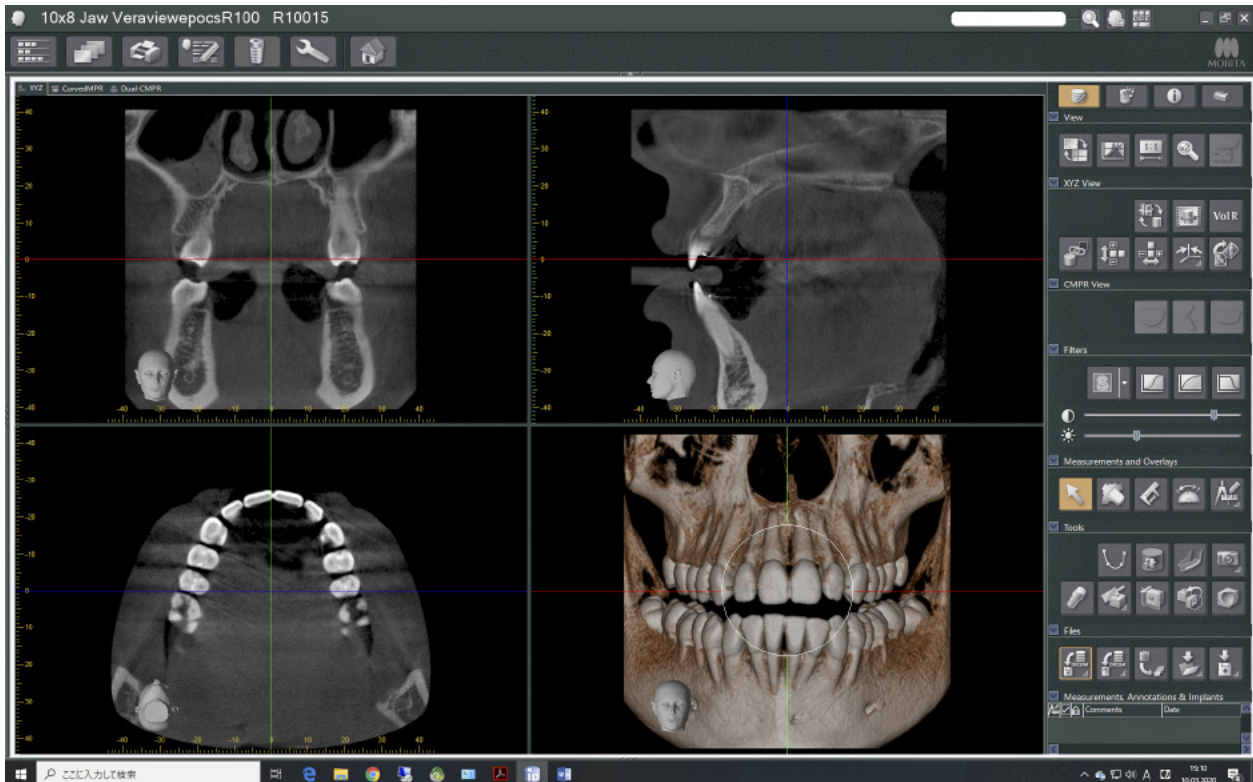


2. Select DICOM export button.

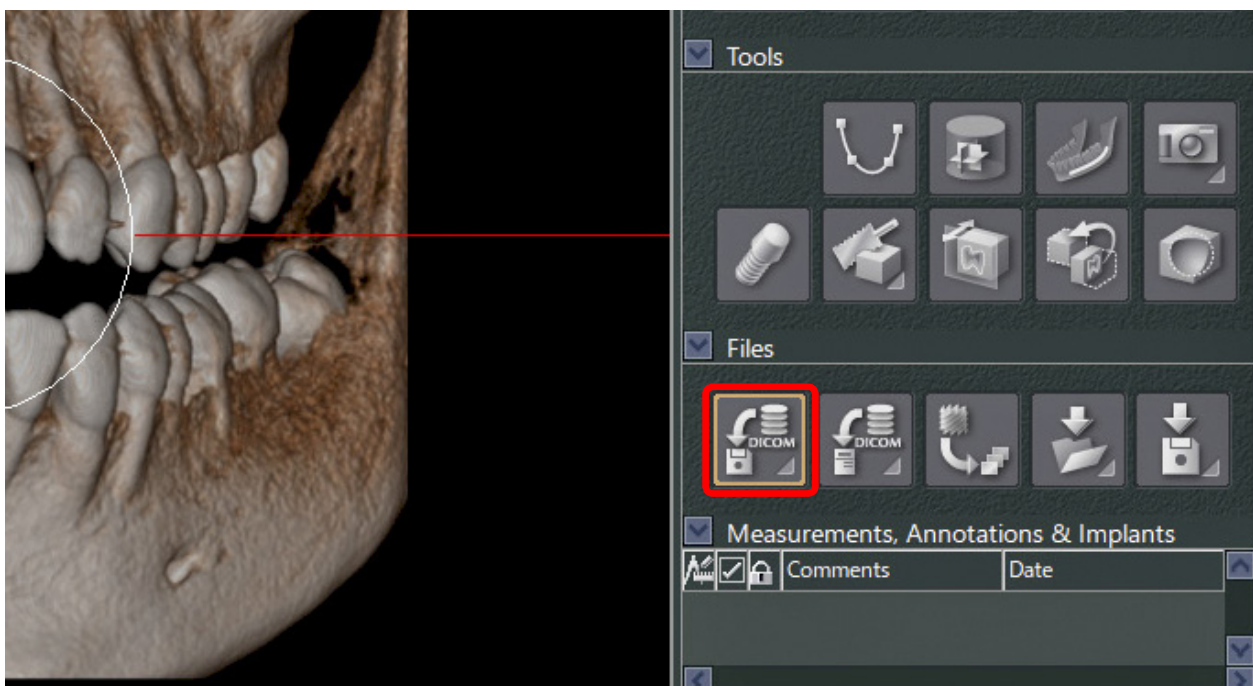


3. Select Axial series.

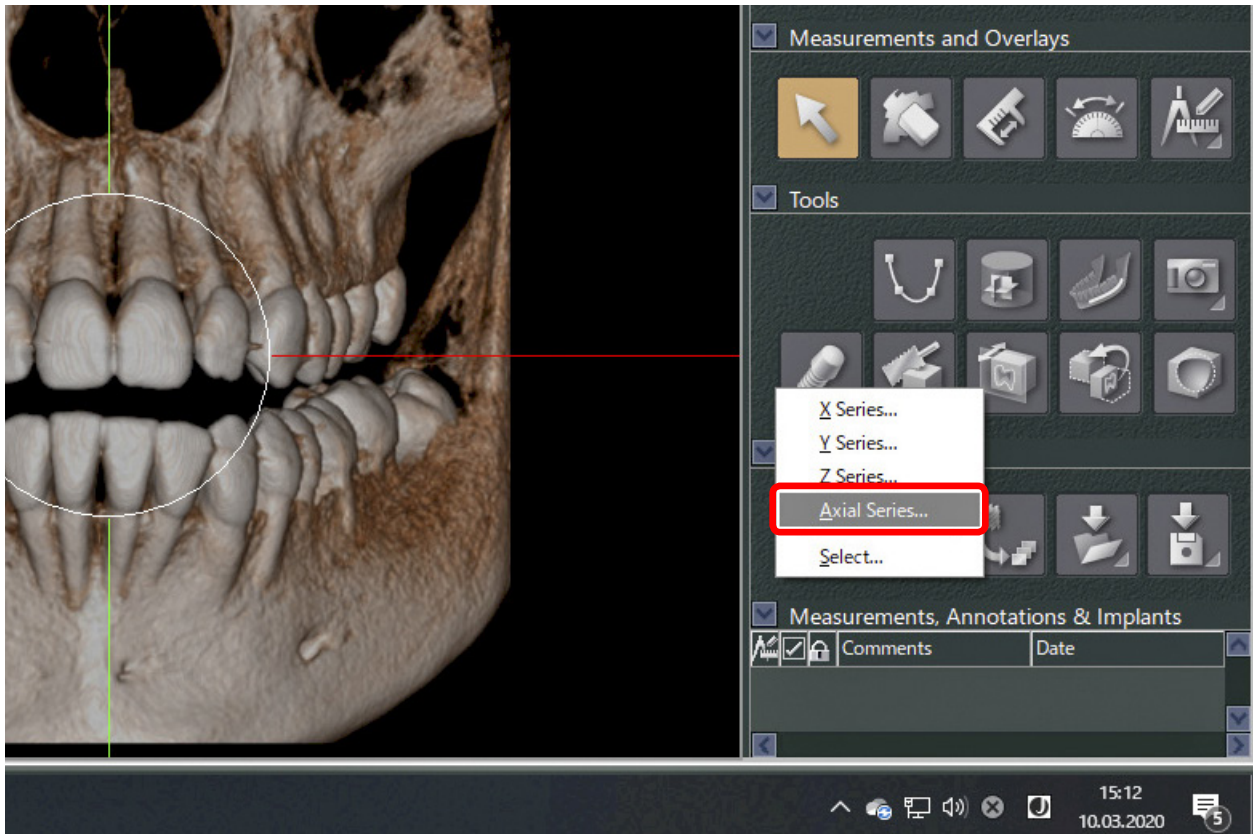


4. Specify data storage location and set interval as small as possible, and Thickness as 1 mm.

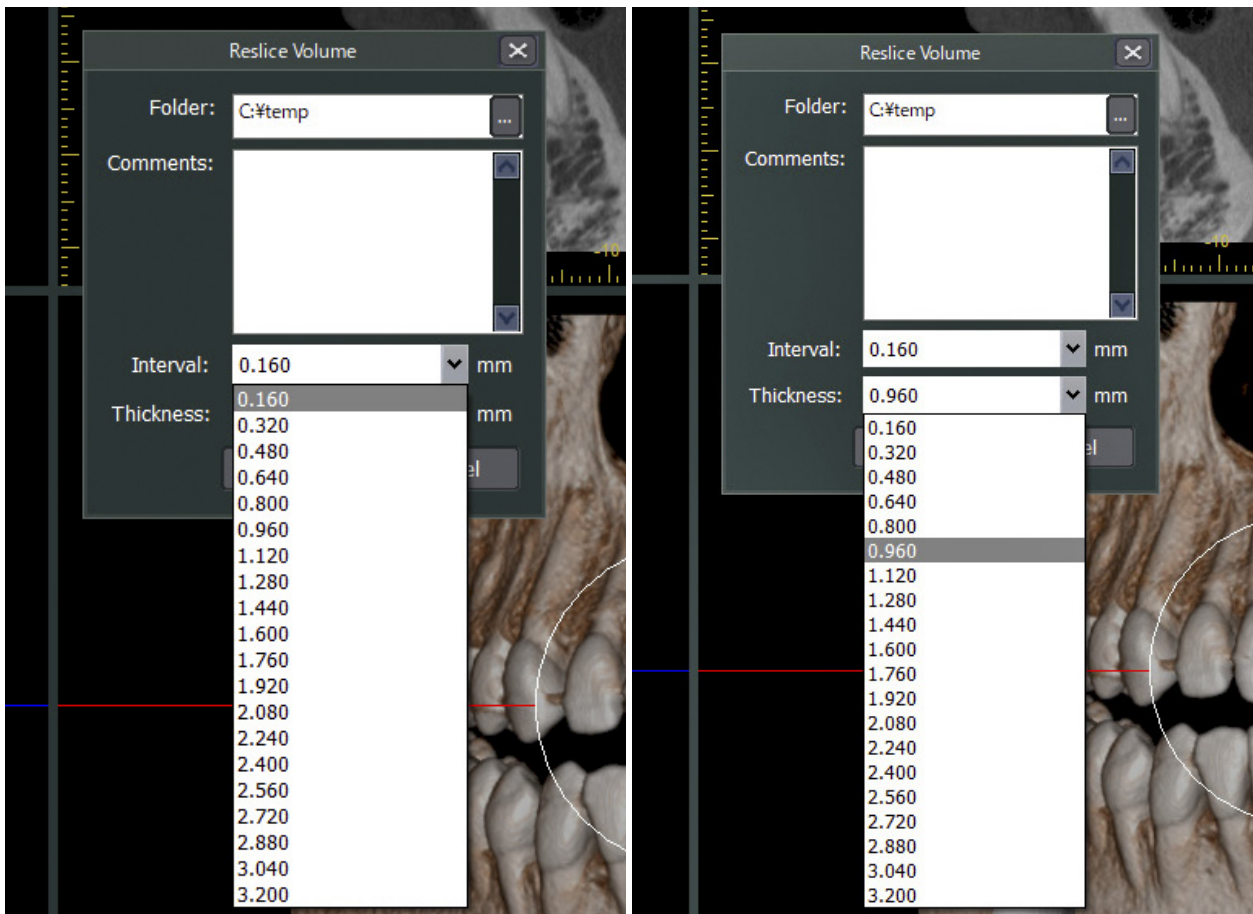


5. Click Ok to export DICOM images.

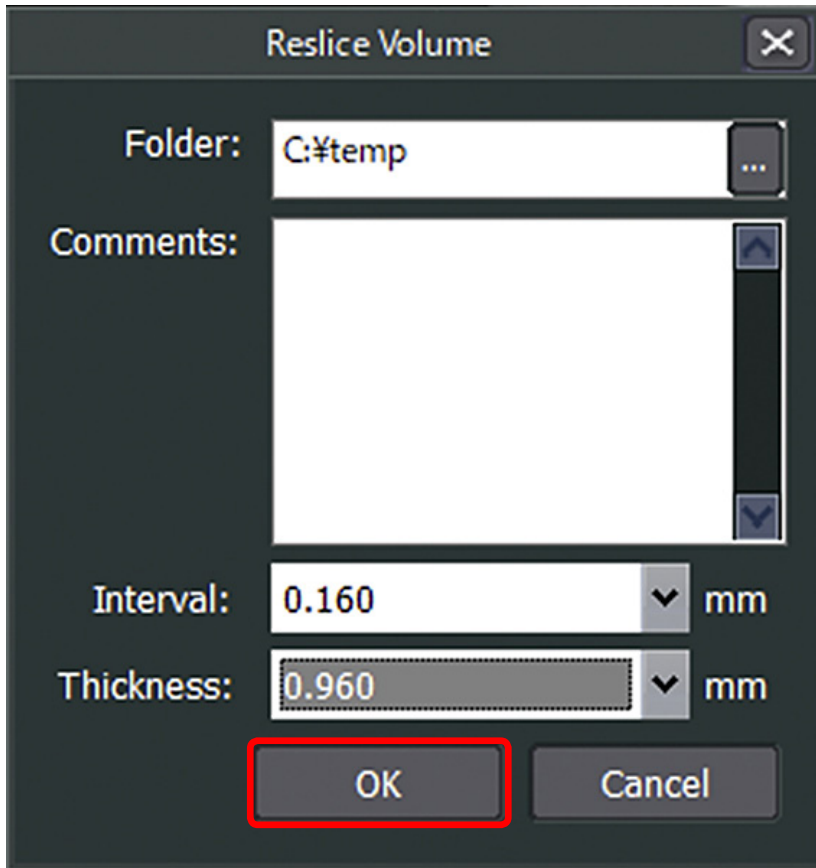

6. Click Ok.

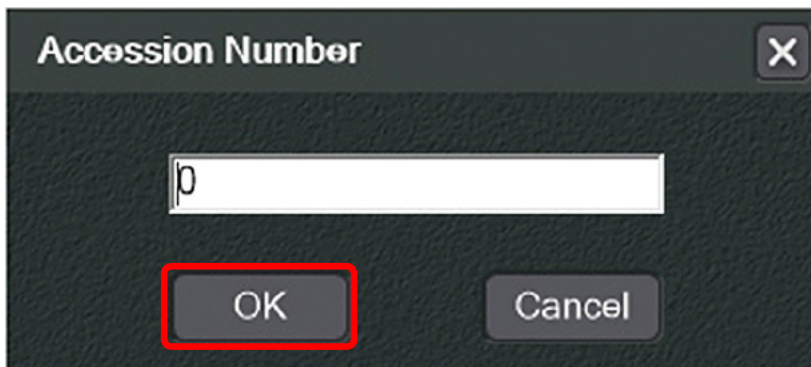


7. Wait for a while.

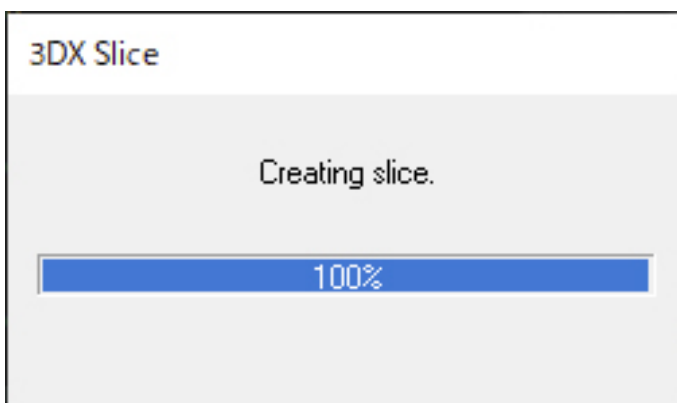



8. Finish.

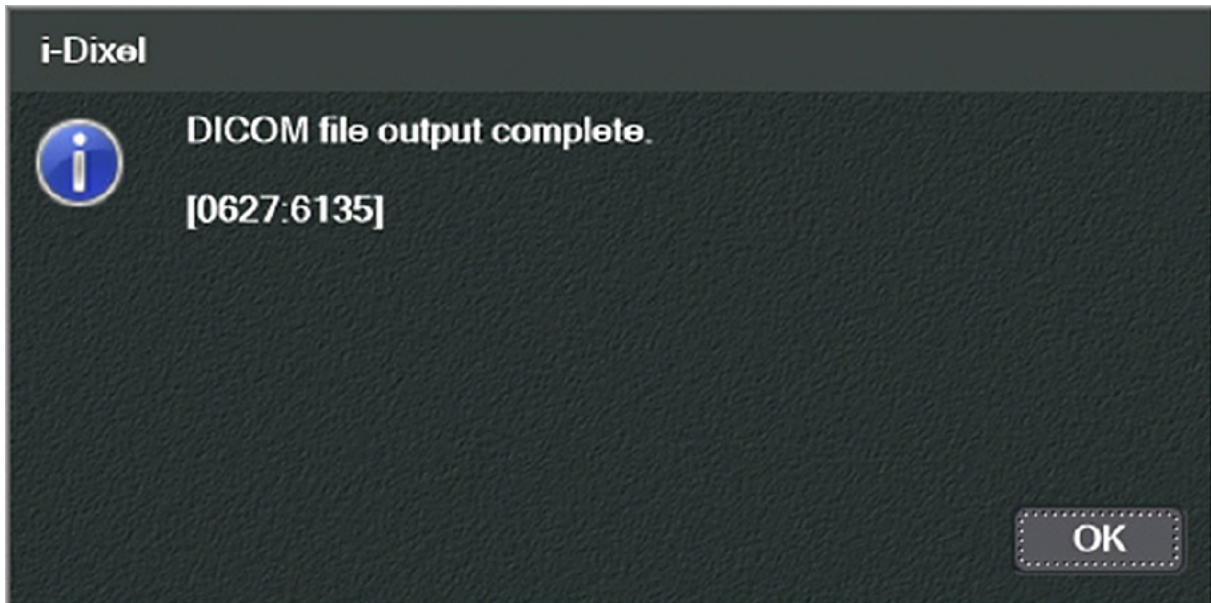


A Windows File Explorer window showing a directory of DICOM files. The address bar displays "C:#temp#R10015\_20200310151611". The table below lists the files with their names, update times, types, and sizes.

名前	更新日時	種類	サイズ
SLZ000.dcm	10.03.2020 15:17	Open DICOM or ZI...	613 KB
SLZ001.dcm	10.03.2020 15:17	Open DICOM or ZI...	613 KB
SLZ002.dcm	10.03.2020 15:17	Open DICOM or ZI...	613 KB
SLZ003.dcm	10.03.2020 15:17	Open DICOM or ZI...	613 KB
SLZ004.dcm	10.03.2020 15:17	Open DICOM or ZI...	613 KB
SLZ005.dcm	10.03.2020 15:17	Open DICOM or ZI...	613 KB
SLZ006.dcm	10.03.2020 15:17	Open DICOM or ZI...	613 KB
SLZ007.dcm	10.03.2020 15:17	Open DICOM or ZI...	613 KB
SLZ008.dcm	10.03.2020 15:17	Open DICOM or ZI...	613 KB
SLZ009.dcm	10.03.2020 15:17	Open DICOM or ZI...	613 KB
SLZ010.dcm	10.03.2020 15:17	Open DICOM or ZI...	613 KB
SLZ011.dcm	10.03.2020 15:17	Open DICOM or ZI...	613 KB
SLZ012.dcm	10.03.2020 15:17	Open DICOM or ZI...	613 KB
SLZ013.dcm	10.03.2020 15:17	Open DICOM or ZI...	613 KB
SLZ014.dcm	10.03.2020 15:17	Open DICOM or ZI...	613 KB
SLZ015.dcm	10.03.2020 15:17	Open DICOM or ZI...	613 KB
SLZ016.dcm	10.03.2020 15:17	Open DICOM or ZI...	613 KB
SLZ017.dcm	10.03.2020 15:17	Open DICOM or ZI...	613 KB
SLZ018.dcm	10.03.2020 15:17	Open DICOM or ZI...	613 KB
SLZ019.dcm	10.03.2020 15:17	Open DICOM or ZI...	613 KB
SLZ020.dcm	10.03.2020 15:17	Open DICOM or ZI...	613 KB
SLZ021.dcm	10.03.2020 15:17	Open DICOM or ZI...	613 KB
SLZ022.dcm	10.03.2020 15:17	Open DICOM or ZI...	613 KB
SLZ023.dcm	10.03.2020 15:17	Open DICOM or ZI...	613 KB
SLZ024.dcm	10.03.2020 15:17	Open DICOM or ZI...	613 KB
SLZ025.dcm	10.03.2020 15:17	Open DICOM or ZI...	613 KB
SLZ026.dcm	10.03.2020 15:17	Open DICOM or ZI...	613 KB
SLZ027.dcm	10.03.2020 15:17	Open DICOM or ZI...	613 KB
SLZ028.dcm	10.03.2020 15:17	Open DICOM or ZI...	613 KB
SLZ029.dcm	10.03.2020 15:17	Open DICOM or ZI...	613 KB
SLZ030.dcm	10.03.2020 15:17	Open DICOM or ZI...	613 KB


# Part 2: Processing your images

## Preparations to order an Azeno solution

1. Highlight the DicomRM folder that is created and send to compressed zipped folder.

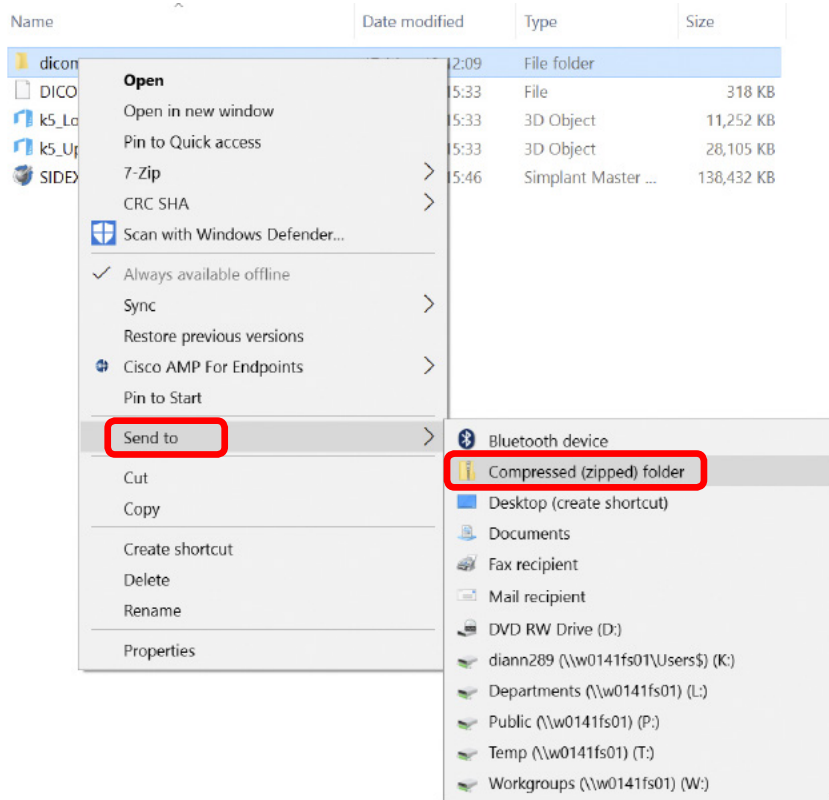

2. When creating the order on orderdigitalsolutions.com, click 'Browse' and select the DICOMRM.zip folder that was created. Then click 'open' to start the upload.

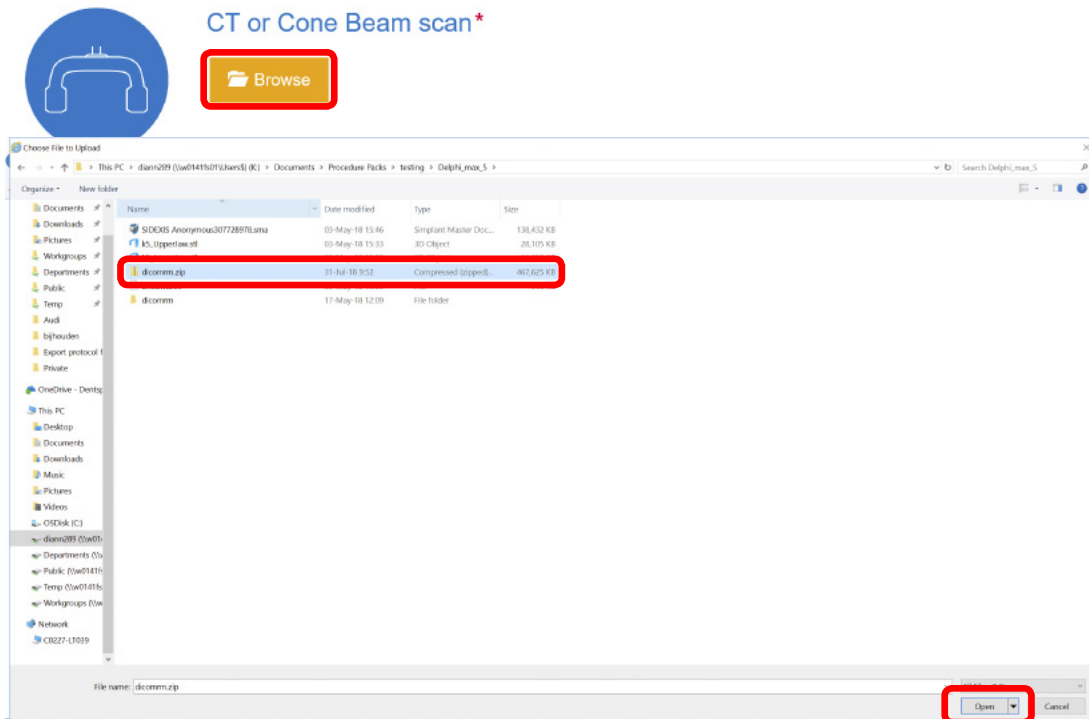

All trademarks and company names are the property of their respective owners. Dentsply Sirona does not waive any rights to its trademarks by not using the symbols ® or ™. 32671963-USX-2003 © 2020 Dentsply Sirona. All rights reserved.  
[www.dentsplysirona.com](http://www.dentsplysirona.com)