

# Getting started

Every component in the Palodent®Plus Sectional Matrix System is optimized to make every step easier. Request a demo to see first hand how the Palodent®V3 components stack up against your current system.



**659670**  
EZ Coat Introductory Kit

100 Matrices - 25 each size: 3.5 mm, 4.5 mm, 5.5 mm, 6.5 mm  
75 Wedges - 25 each size: Small, Medium, Large  
30 WedgeGuards - 10 each size: Small, Medium, Large  
1 Universal Ring; 1 Narrow Ring; 1 Forceps; 1 Pin Tweezers

## REORDER INFORMATION

### Ring Refills

**659900** Universal (1)  
**659760** Universal (2)  
**659770** Narrow (2)

### Matrix Refills EZ Coat

**659610** Size 3.5 mm (50)  
**659620** Size 4.5 mm (50)  
**659630** Size 5.5 mm (50)  
**659640** Size 5.5 mm (90)  
**659650** Size 6.5 mm (50)  
**659660** Size 7.5 mm (50)

### Matrix Refills

**659710** Size 3.5 mm (50)  
**659720** Size 4.5 mm (50)  
**659730** Size 5.5 mm (50)  
**659740** Size 5.5 mm (100)  
**659750** Size 6.5 mm (50)  
**659860** Size 7.5 mm (50)

### Wedge Refills

**659780** Small (100)  
**659790** Medium (100)  
**659800** Large (100)

### WedgeGuard Refills

**659830** Small (50)  
**659840** Medium (50)  
**659850** Large (50)

### Instrument Refills

**659810** Forceps Refill (1)  
**659820** Pin Tweezers Refill (1)



### Universal Ring (light blue)

- Nickel-titanium for strength and memory
- Stackable for multiple restorations
- V-shaped, glass-fiber reinforced plastic tines



### Narrow Ring (dark blue)

- Tines set closer together for optimal separation on smaller teeth
- Up to 4kgs of separating force even on premolars



### EZ-Coat Matrices (5 sizes)

- Ultra-thin, non-stick coating makes matrix removal easier and reduces light reflection, improving depth of field for better visibility
- Greater curvature for anatomical precision and less interference with ring placement
- Sizes color-coded for easy identification



### Matrices (5 sizes)

- Pronounced marginal ridge for accurate anatomy
- Gingival apron is optimally shaped to prevent gaps in gingival-axial corner



### Wedges (3 sizes)

- Compresses on entry and flares upon exit for easy placement and seal
- Hollow underside allows placement of second wedge from opposite side and protection of the papilla



### WedgeGuards (3 sizes)

- Guard detaches after prep, leaving the wedge in place
- Facilitates faster and easier cutting of cavity and lowers risk of interproximal damage to adjacent tooth



### Pin Tweezers

- Pin-in tip positively grips hole in matrix and wedge for control
- Naturally closed position minimizes risk of dropping matrix band or wedge
- Built-in ball burnisher tip



Better  
outcomes  
start here

Palodent®Plus  
Sectional Matrix System

THE DENTAL  
SOLUTIONS  
COMPANY™

Dentsply  
Sirona

THE DENTAL  
SOLUTIONS  
COMPANY™

Dentsply  
Sirona

See over page for  
4 Steps to Better  
Class II's

# Palodent®Plus

“70% of clinicians find contact creation to be the most challenging part of a Class II restoration.”<sup>1</sup>

<sup>1</sup>DentalTown (2012). Restorative Dentistry. Monthly Poll: What is the most challenging part of Class II Restoration?



## Producing high quality Class II Restorations

The Palodent®Plus system is an innovative sectional matrix system with Nickel-Titanium rings, anatomically shaped matrix bands and adaptive self-guiding wedges that provides predictable, tight contacts and restorations that accurately replicate the natural tooth anatomy.


The Palodent®Plus matrix system delivers improved outcomes, even when faced with wide cavities and missing cusps. The contoured design of the matrices and wedges ensure there is minimal finishing of the restoration. Used by tens of thousands of clinicians in dental offices and schools globally, better dentistry starts with Palodent®Plus

“Love it, love it. I can’t believe I practiced for the last 13 years without it. I am embarrassed that my Class II fillings would look like straight walls without the contours!”

M. Rodrigue, DDS



## Better outcomes with a Sectional Matrix System

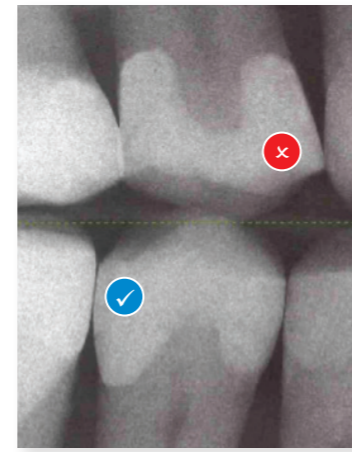


Circumferential Band (Tofflemire)	Sectional Matrix System Restoration
✗ Fails to restore proximal anatomy	✓ Easy to use system
✗ Thin contact at the marginal ridge	✓ Natural contours
✗ Increased likelihood of fracture, occlusal interference, recurrent caries and periodontal disease	✓ Tight, anatomically correct contact point
✗ Large food trap below	✓ Minimal interproximal finishing
✗ Increased chance of light or open contact	
✗ Extensive finishing may be required	

### Sectional System vs Circumferential System


The Tofflemire circumferential matrix band was developed in the 1940s to aid placement of dental amalgam. It was not designed with today's modern restorative materials in mind, which can result in less than optimal outcomes when placing composites.

In contrast, sectional matrix systems were developed specifically for placing composite materials.

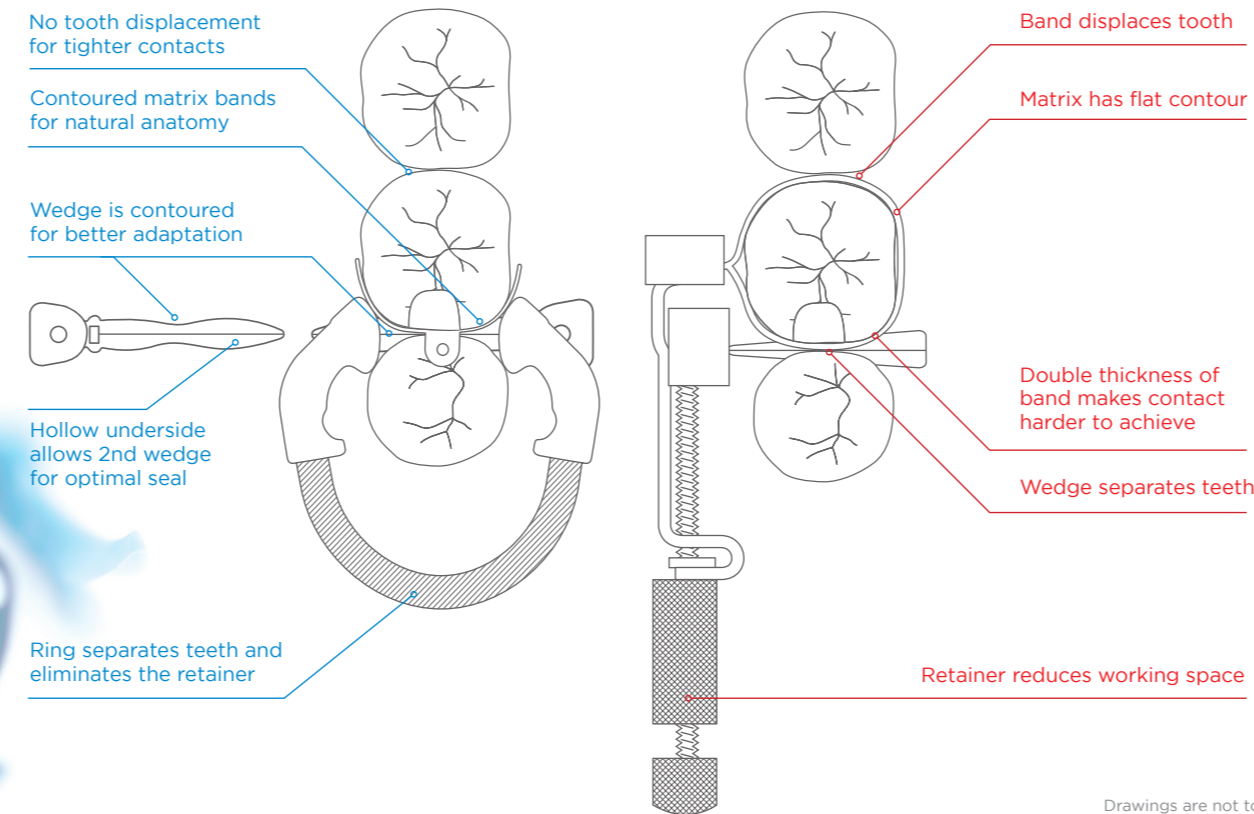


## Take the stress out of Class II restorations


- 1 Strong Ni-Ti Rings for outstanding spring strength and memory
- 2 Glass-fiber reinforced plastic tines are V-shaped to accommodate the wedge
- 3 Pin tweezer hole for easy placement & removal of wedge
- 4 Top tab for easy insertion and side tab for removal of matrix
- 5 Gingival apron to prevent gaps in gingival-axial corner
- 6 Greater curvature: matrix wraps around tooth
- 7 Pronounced marginal ridge for natural-looking anatomy

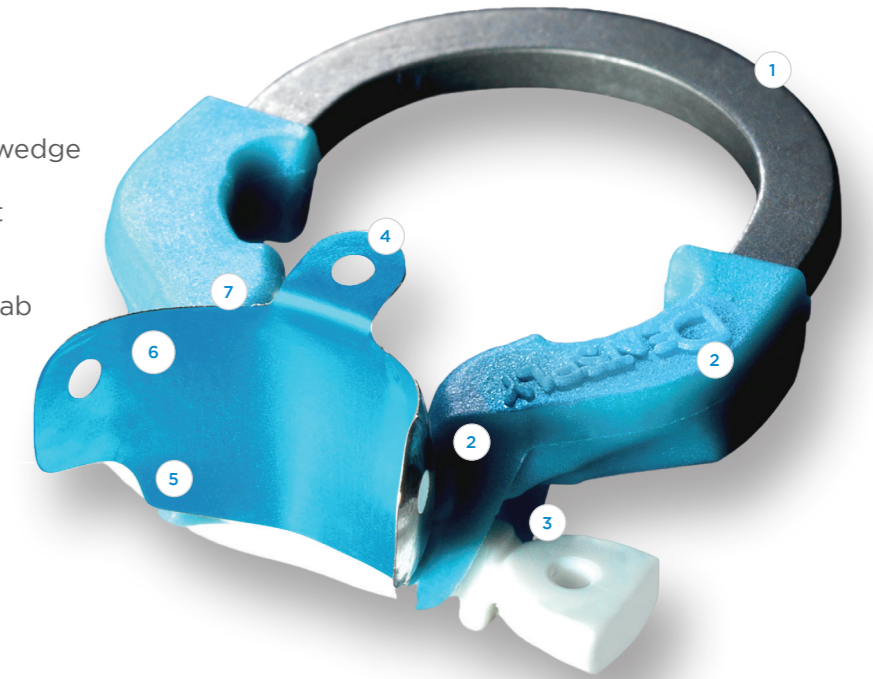


### 4 steps to better Class

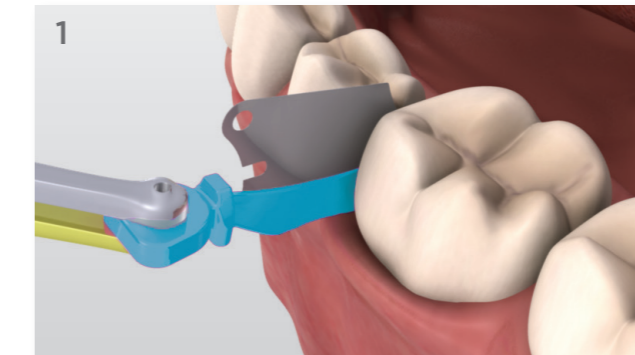


Using the Pin-Tweezers, place a Palodent®Plus Matrix with Pin Tweezers.

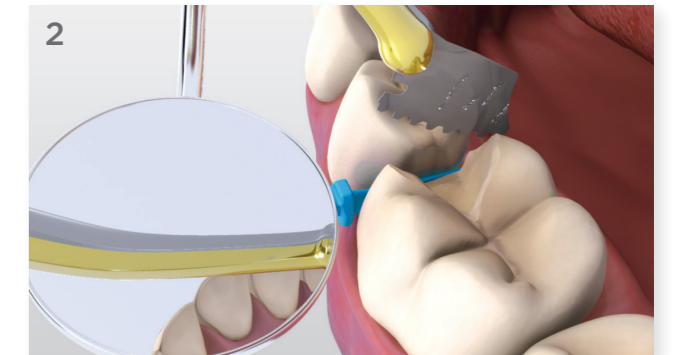

After prep, if using Palodent®WedgeGuard, support Wedge with mirror or dental instrument while using Pin Tweezers to detach the shield.

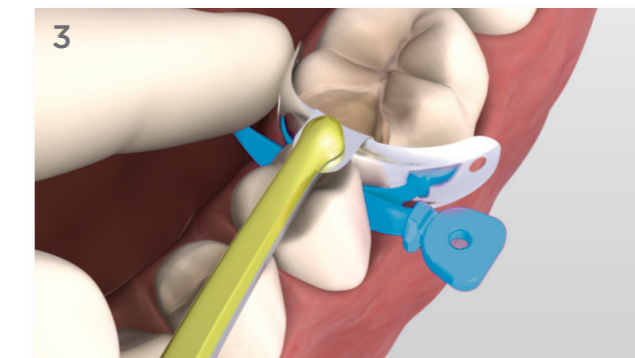

Place Palodent®Plus Matrix with Pin Tweezers.

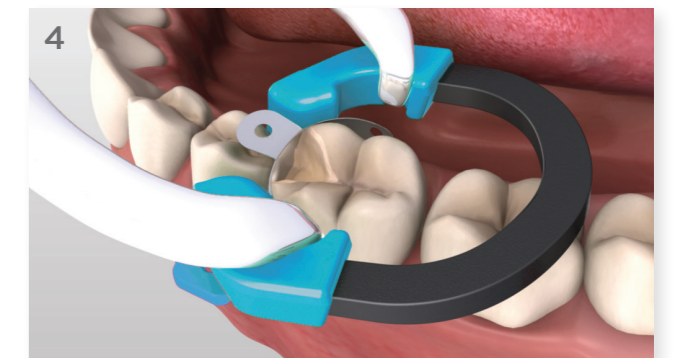

Place the Palodent®Plus Ring on top of the wedge to provide separation.

\*Refer to the Directions for Use for full instructions and reprocessing information.