

A high-action photograph of a male surfer in a black wetsuit riding a yellow surfboard inside the barrel of a massive, curling blue wave. The water is turbulent and white with foam. In the top right corner, there is a red rectangular box containing a white plus sign and the text 'WE KNOW ENDO.'

+
**WE
KNOW
ENDO.**

WaveOne® Gold

Surf the canal
with confidence

THE DENTAL
SOLUTIONS
COMPANY™

 **Dentsply
Sirona**



Engineered with the latest reciprocating technology

The single-file simplicity of WaveOne® is now enhanced in WaveOne Gold. Made with advanced metallurgy, it is the next generation of performance from Dentsply Sirona.

- Increases patient safety,* while reducing shaping time across a wide range of cases**
- Optimized with enhanced design features by leading endodontists
- Part of a complete solution that includes obturation products designed to match the shapes created by WaveOne Gold files
- The WaveOne Gold family now includes a single glide path file with reciprocating motion for simplicity from the start





Optimized tip

Variable taper

GOLD thermal treatment

Evolving cross-section

Reciprocating motion

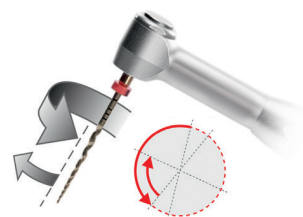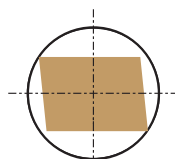

*compared to WaveOne®

**internal laboratory testing results

Simplicity for you, safety for patients

Reinforces patient safety

- Primary WaveOne Gold file is 50% more resistant to cyclic fatigue than WaveOne Primary file
- Reduced screwing effect** compared to standard rotary systems


Covers a wider* range of canal morphologies

- Enhanced file flexibility thanks to the Gold technology
- Extended size range (Small, Primary, Medium, Large)


Shortens the shaping time

- Takes the cutting efficiency to a higher level
- A single file per treatment also translates into time savings for canal shaping and irrigation


Cyclic fatigue resistance**

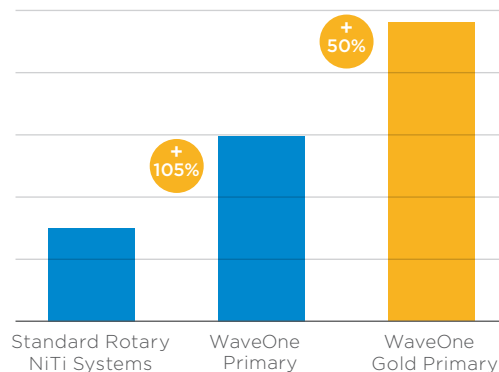

Flexibility**

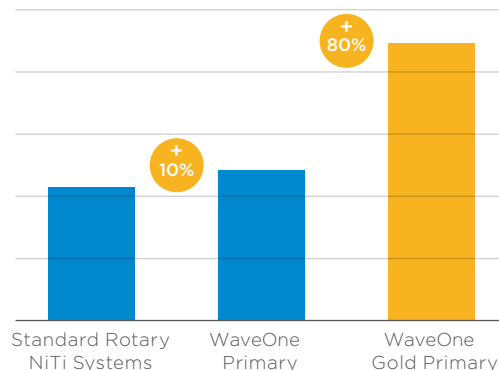

Shaping working time**

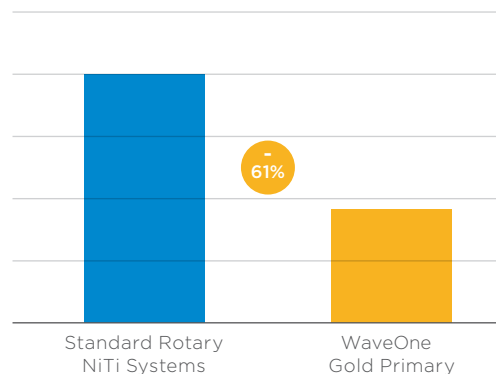


*compared to WaveOne®

**internal laboratory testing results



The next generation of performance

Dentsply Sirona has teamed up with the best endodontic professionals to develop WaveOne Gold.

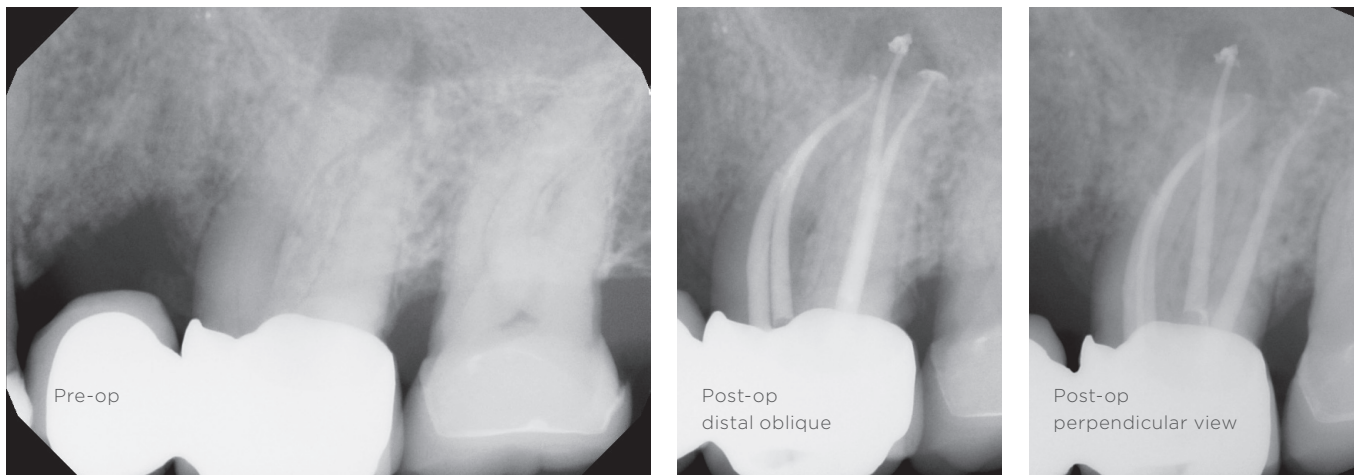

The design team speaks – Dr C. Ruddle, Dr J. Webber, Dr W. Pertot, Dr S. Kuttler

Shaping canals has never been easier

Since its launch in 2011, Dentsply Sirona's WaveOne has set a new standard in root canal shaping. All around the world general dentists and endodontists have voiced their appreciation of the simplicity WaveOne brings to their efforts to provide quality endodontic treatment to their patients.

We have created a new and exciting generation of WaveOne to bring you WaveOne Gold.

WaveOne Gold brings to the endodontic procedure a metallurgically advanced single-file technique for shaping canals. Further, we optimized the tip diameters, tapers, and the cross-section to produce a file that really improves safety, efficiency and flexibility when preparing canals. WaveOne Gold takes simplicity and treatment success to another level.

One complete solution

The WaveOne Gold family of products provides a comprehensive treatment solution to promote confidence and predictable shape.

Motors

X-Smart® Plus and X-Smart IQ® are pre-programmed for use with WaveOne® Gold for simplicity

Glide Path

The WaveOne Gold® Glider optimizes your glide path before shaping with WaveOne Gold Primary**

Shaping

WaveOne Gold shaping files are available in four sizes with three working lengths (21, 25, 31 mm) to treat a wide range of cases

Obturation

Dedicated WaveOne Gold obturation products include paper points, variable tapered gutta-percha points and Thermafil® obturators

Dentsply Sirona Restorative Solutions

Please consult our website for restorative solutions
www.dentsplysirona.com

Create your path with confidence




This new single glide path file has been designed in collaboration with a team of endodontist experts : Dr E. Berrutti, Dr G. Cantatore, Dr A. Castellucci, Dr S. Kuttler, Dr W. Pertot, Dr C. Ruddie, Dr J. Webber, Dr J. West.

Optimize your shaping preparation with a single reciprocating glide path file that is part of the WaveOne Gold solution.


- Respects the natural shape of the canal
- Increases patient safety with enhanced flexibility and resistance to cyclic fatigue
- Sterile and ready for single use, which preserves cutting efficiency, reduces breakage and prevents cross contamination

The WaveOne Gold Glider is only for use in reciprocating motion with compatible Dentsply Sirona motors. Helpful pre-programmed motor settings are available.

Lower bending resistance than PathFile®, for an increased flexibility thanks to the gold heat treatment**

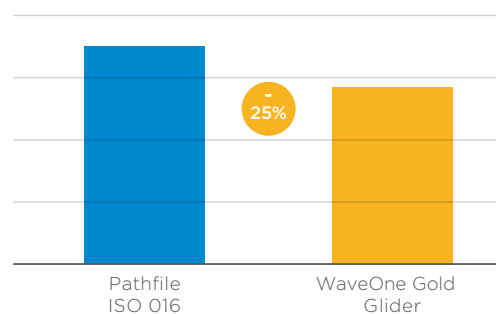

Better resistance to cycle fatigue**

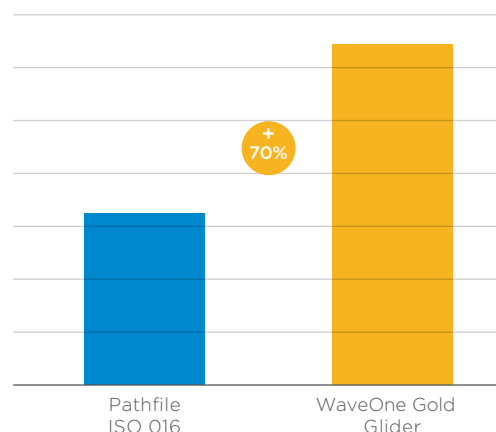

WaveOne Gold Glider



015 2%

WaveOne Gold



020 7% Size Small



025 7% Size Primary



035 6% Size Medium



045 5% Size Large

ISO color coded

The majority** of root canals can be shaped with a single WaveOne Gold Primary file

**internal laboratory testing results

Expanded selection of complimentary products

With the addition of a glide path file, WaveOne Gold gives you the convenience of an entire family of products designed to work together, right from the start.


Original WaveOne Gold family of products



Expanded WaveOne Gold family of products under our new Dentsply Sirona branding




The new gold standard of gutta-percha introduced for WaveOne Gold

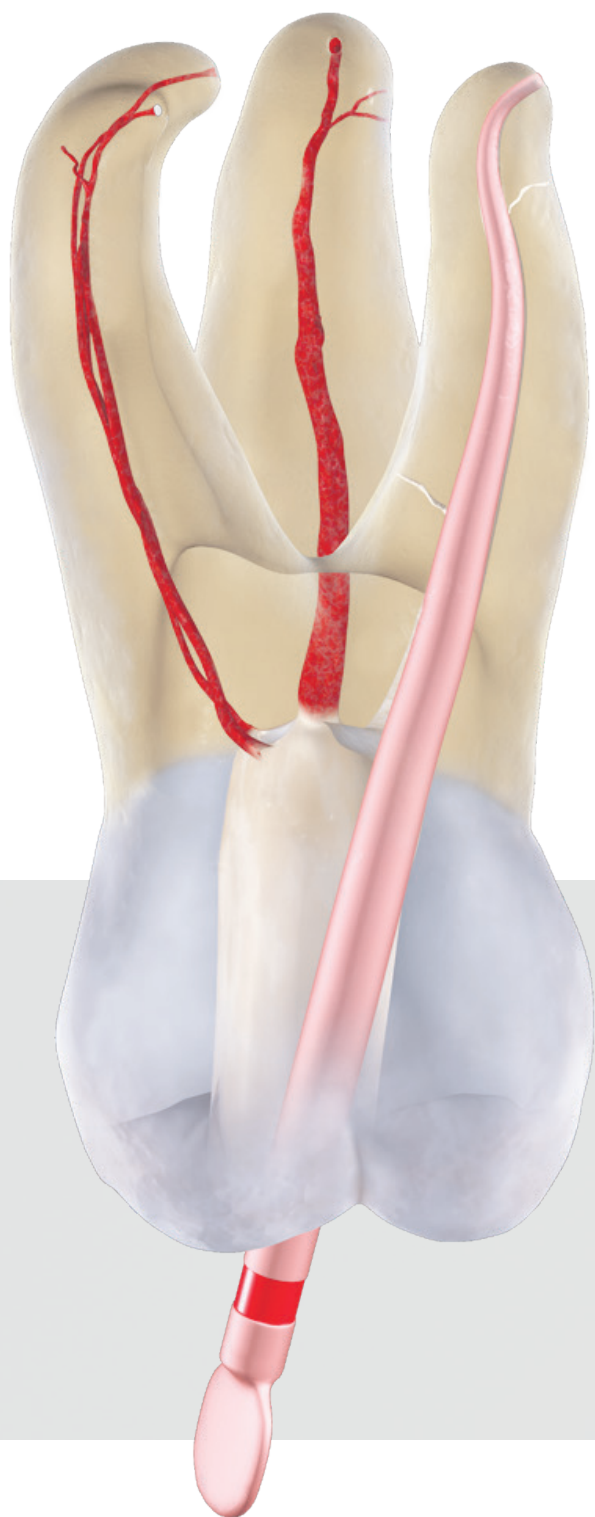

Conform fit

Variable tapered gutta-percha point, specially moulded to fit your canal prepared with WaveOne Gold.

Superior apical gutta-percha point filling volume**

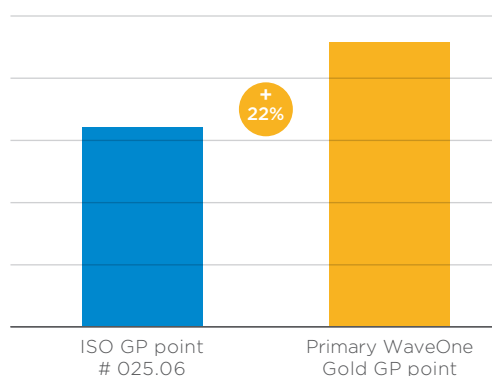

Clearly identifiable: same color code as the file.



***internal laboratory testing results*

Paper points and Thermafil® for WaveOne Gold



Paper points


- Paper points are packaged in individual sterile packs of 5 points
- Ready to use, this packaging matches your average need for a canal treatment

Thermafil for WaveOne Gold

The Thermafil you like optimized for your WaveOne Gold shaping.

- Blister pack: Easy to handle
- Batch number and expiration date on each obturator label to increase your traceability

4 sizes (small, primary, medium, large) fit canals prepared with the related WaveOne Gold files.

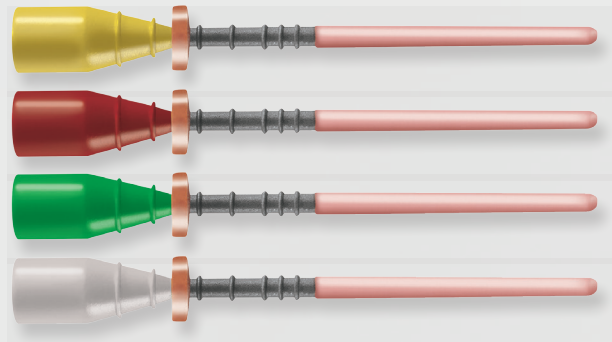



Product references

Reciprocating glide path files



Glider - 015, 02 variable taper

3x	21mm	B ST W1GG 321 015
3x	25mm	B ST W1GG 325 015
3x	31mm	B ST W1GG 331 015

WaveOne® Gold Gutta-Percha Points



Box of 60x Variable Tapered Gutta Percha Points

Small	A 175X 000 00S 00
Primary	A 175X 000 00P 00
Medium	A 175X 000 00M 00
Large	A 175X 000 00L 00

Reciprocation shaping files



Small - 020, 07 variable taper

3x	21mm	A 0753 221 G0S 03
3x	25mm	A 0753 225 G0S 03
3x	31mm	A 0753 231 G0S 03
6x	21mm	A 0756 221 G0S 03
6x	25mm	A 0756 225 G0S 03
6x	31mm	A 0756 231 G0S 03

Primary - 025, 07 variable taper

3x	21mm	A 0753 221 G0P 03
3x	25mm	A 0753 225 G0P 03
3x	31mm	A 0753 231 G0P 03
6x	21mm	A 0756 221 G0P 03
6x	25mm	A 0756 225 G0P 03
6x	31mm	A 0756 231 G0P 03

Medium - 035, 06 variable taper

3x	21mm	A 0753 221 G0M 03
3x	25mm	A 0753 225 G0M 03
3x	31mm	A 0753 231 G0M 03
6x	21mm	A 0756 221 G0M 03
6x	25mm	A 0756 225 G0M 03
6x	31mm	A 0756 231 G0M 03

Large - 045, 05 variable taper

3x	21mm	A 0753 221 G0L 03
3x	25mm	A 0753 225 G0L 03
3x	31mm	A 0753 231 G0L 03
6x	21mm	A 0756 221 G0L 03
6x	25mm	A 0756 225 G0L 03
6x	31mm	A 0756 231 G0L 03

Assortment Reciprocating File 1x Small/Primary/Medium/Large

21mm	A 0754 221 G9A 03
25mm	A 0754 225 G9A 03
31mm	A 0754 231 G9A 03

Thermafil® for WaveOne® Gold



Blister of 6x Obturators

Small	A 1750 B00 00S 00
Primary	A 1750 B00 00P 00
Medium	A 1750 B00 00M 00
Large	A 1750 B00 00L 00

Box of 30x Obturators (5 Blisters)

Small	A 1750 P00 00S 00
Primary	A 1750 P00 00P 00
Medium	A 1750 P00 00M 00
Large	A 1750 P00 00L 00

Kits and motors

X-SMART IQ™ Starter Kit A10541

1 Motor with Contra-Angle
1 Pack of 15 Primary WaveOne Gold Files
1 Pack of 16 Assorted WaveOne Gold Files
1 Blister of 3 Glide Path Files
1 iPad Protective Case

X-SMART® Plus Starter Kit A10358

1 Motor with Contra-Angle
1 Pack of 15 Primary WaveOne Gold Files
1 Pack of 16 Assorted WaveOne Gold Files
1 Blister of 3 Glide Path Files

Motors

X-SMART IQ™ Motor	A1054
X-SMART® Plus Motor	A1032



WaveOne® Gold Absorbent Points



Box of 180x Sterile Paper Points

Small	A 175W 000 00S 03
Primary	A 175W 000 00P 03
Medium	A 175W 000 00M 03
Large	A 175W 000 00L 03

Oven ThermaPrep® 2



230V (EU)	A 0176 230 002 00
230V (UK)	A 0176 230 002 33
115V	A 0176 115 002 00



Building 3 The Heights, Weybridge, Surrey KT13 0NY, United Kingdom
Telephone: +44 (0)1932 853 422
www.dentsplysirona.com

**Dentsply
Sirona**