


OSSIX® Plus Resorbable Collagen Membrane Tips & Guidelines for Usage

OSSIX® Plus is a naturally cross-linked resorbable collagen membrane.

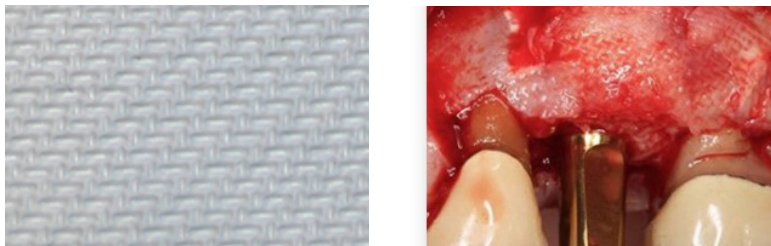

OSSIX® Plus unique pattern

Guidelines to consider when using OSSIX® Plus

1. Start with simple cases and learn the membrane's behavior and qualities.
2. Hydrate the membrane with sterile saline until completely wet (about 30 seconds).
3. If placed adjacent to teeth, leave a 1 mm space to the teeth.
4. Trim membrane with curved scissors and do not leave cuts into the membrane.
5. After laying membrane in place, if more stabilization is required, suture over it with periosteal sutures. It is not recommended to suture through or tack membrane directly.
6. When you reopen the site you will potentially see membrane remnants, do not remove as these will ossify over time.
7. If prematurely exposed, re-suture if flaps are mobile, have the patient rinse with chlorhexidine and avoid hard or hot food. The membrane will last up to 5 weeks when exposed.

Please refer to OSSIX Plus Instructions for Use for the full information on indications, contraindications, warning and precautions.