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**exocad** Secure area EN

Prosthetic components libraries for usage with titanium bases and direct connections.

[View and download libraries](#)

**Dentsply Sirona Inc.**  
Libraries for Atlantis suprastructures FLO-S (US-cleared, restorations designed with this library can only be milled by Dentsply Sirona) and TitaniumBase. \*1)

**Package: DentsplySirona ModelCreator libraries exocad**  
Updated: 22-10-2020, 2 libraries included [+ Add to download list](#)

**Package: DentsplySirona DentalCAD US cleared libraries exocad**  
Updated: 22-10-2020, 1 libraries included [+ Add to download list](#)

**Package: DentsplySirona DentalCAD libraries exocad**  
Updated: 09-07-2020, 1 libraries included [+ Add to download list](#)

**DESS® Dental Smart Solutions by Terrats Medical S.L.**  
Extended Implant libraries for usage with titanium bases for Intraoral and Desktop scanbodies.

## Dentsply Sirona Implants library - installation instruction for Exocad

## Dentsply Sirona Implants libraries - how to use in Exocad

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Download from 3Shape Dental System Control Panel - Download center

Library name	Manufacturer	Implant library	Version	Size	Notes
<input type="checkbox"/> Dentsply Sirona Implants Atlantis suprastructures FLO-S	Dentsply Sirona Implants	Implant library	1.0.0		
<input type="checkbox"/> Dentsply Sirona Implants TitaniumBase	Dentsply Sirona Implants	Implant library	1.0.0		
<input type="checkbox"/> Dentsply Sirona Implants Atlantis suprastructures 1.0.0	Dentsply Sirona Implants	Implant library	1.0.0	6.087 KB	This library is used for digital design guidance for Atlantis suprastructures and for building and printing models. Includes Ankylos®, Astra Tech Implant System®, Invisi® and FLO-S.
<input type="checkbox"/> Dentsply Sirona Implants TitaniumBase	Dentsply Sirona Implants	Implant library	1.0.0	9.327 KB	This library is used when designing restorations for TitaniumBase and for building and printing models. Includes Ankylos®, Astra Tech Implant System® EV and Invisi® S. Support for 3Shape software.

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