

CEREC Equipment Restart Guidelines

Below are guidelines for restarting your CEREC equipment after an inactivity period. These guidelines are based on our CEREC Preparation for Inactivity Period that was released in March. Please refer to your CEREC Equipment Operating Instructions for further information.

Dentsply Sirona CAD/CAM Support, Rev1.April 2020



CEREC MC / MC X / MC XL / Primemill

- Plug power cord back into wall outlet if previously unplugged. Turn unit on
- Add water to proper level in water tank and add 75ml of Dentatec
- Reinsert water tank and run water pump until a steady stream is observed from the spray ports
- Check and top off water in tank as necessary



CEREC Speedfire

- Plug power cord back into wall outlet if previously unplugged
- Turn unit on and allow to bootup



CEREC AC / Omnicam / Primescan

- Plug power cord back into wall outlet if previously unplugged
- Bootup unit
- Confirm communication to camera, milling unit & CEREC Speedfire by:
 - Open CEREC software
 - Select **DS** icon at the top left to open menu bar
 - Select **Configuration**
 - Select **Devices**
 - Confirm proper communication by green check marks



CEREC Speed-Fire



Primemill



Primescan

Your CEREC system is now ready to use.