

Endodontic Activation System Technique Sequence

- 1 Fully prepare canal.
- 2 Fill pulp chamber with NaOCl, EDTA or other rinse solution.
- 3 Select the SmartLite Pro EndoActivator™ Tip that manually fits loosely within 2 mm of working length (tips should not be taken closer than 2 mm from the working length).
- 4 Place the SmartLite Pro EndoActivator™ Barrier Sleeve over the SmartLite Pro EndoActivator™ Attachment.
- 5 Do not apply the barrier sleeve too tightly over the attachment head.
- 6 The barrier should not be penetrated. It is designed to give a complete enclosure of the handpiece and attachment.
- 7 Attach the selected SmartLite Pro EndoActivator™ Tip to barrier protected attachment. Briefly check proper tip operation outside the patient's mouth before intraoral use.




- 8 Place the attached SmartLite Pro EndoActivator™ Tip into the prepared root canal.
- 9 Press the On/Off button once to activate the handpiece.
- 10 Use a pumping action to move the SmartLite Pro EndoActivator™ in short, 2-3 mm vertical strokes.
- 11 Hydrodynamically agitate the intracanal solution for 30-60 seconds.
- 12 Press button 2 times to turn off device. Do not remove the tip from the root canal while still oscillating.
- 13 Irrigate, then use intracanal suction to eliminate loose debris.
- 14 Repeat the above steps for each intracanal irrigant used.



Scan here to easily
access resources
and videos for
your system!

Endodontic Activation System User Interface

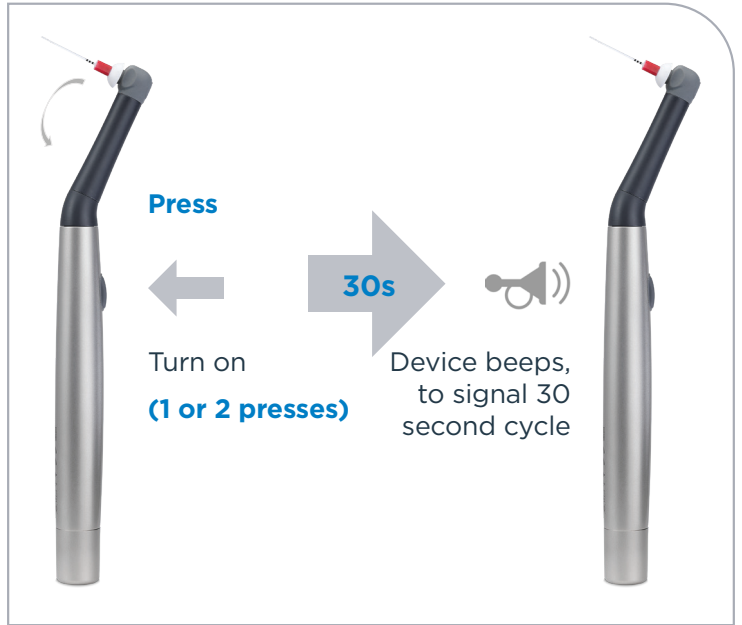

Battery pack



Handpiece



Attachment



Caution: Repeated improper removal could cause damage to the attachment.



Reorder Information



644434
 SmartLite Pro
 EndoActivator™ Barrier
 Sleeves (QTY 100)



644435
 SmartLite Pro
 EndoActivator™ Tips
 (QTY 25) Small: 22mm
 (15/02)



644436
 SmartLite Pro
 EndoActivator™ Tips
 (QTY 25) Medium:
 22mm (25/04)



644437
 SmartLite Pro
 EndoActivator™ Tips
 (QTY 25) Medium
 Long: 28mm (25/04)