Essix® Laminated Sports Mouthguard Material

ENG

INSTRUCTIONS FOR USE

INTENDED USE:

Essix® Laminated Sports Mouthguard Material is thermoformed to fabricate custom sports mouthguards for children and young adults participating in high contact sports.

INDICATIONS FOR USE:

Essix® Laminated Sports Mouthguard Materials are indicated for the fabrication of sports mouthguards.

PERFORMANCE CHARACTERISTICS AND CLINICAL BENEFITS:

Performance characteristics of orthodontic plastics include a good combination of strength, durability and clarity, resistant to cracking or breaking, easily removable from models and easy to trim.

Clinical benefits of orthodontic treatment may include overall improvement of tooth health and correction of malocclusion.

CONTRAINDICATIONS:

This product is contraindicated for patients with a history of allergic reactions to plastics.

WARNINGS:

Excessive heating time during thermoforming will cause plastic to over sag and potential for injury to operator.

Improper thermoforming of the plastic appliance may cause the appliance to crack/break, resulting in sharp edges, loose pieces and possible aspiration of pieces.

PRECAUTIONS:

Patients with a history of allergic reactions to plastics should not use this product.

Do not reuse plastic sheet. Once thermoformed it cannot be reused.

Thermoformed device is for Single Patient Use. Risks of reuse include infection and contamination (i.e. bacterial, viral).

Store plastic sheets in a cool, dry place.

ADVERSE REACTIONS:

Allergic reactions.

STEP BY STEP INSTRUCTIONS FOR USE WITH ESSIX VACUUM MACHINES:

1. Sports mouthguards should be trimmed high to the vestibular margins, leaving the frenums. Full arch models should have the palate trimmed away.
2. **IMPORTANT:** Pre-heat the machine before using! Pre-heat the vacuum machine for at least 3 minutes or until the heating element is a solid orange color. (Heating times may vary with current fluctuations or length of time the heating element has been left on.)
3. Spray the model with Trim-Rite® Release Agent to prevent the material from sticking to the model.
4. Place the plastic into the frame of the vacuum machine and the model on the base plate.
5. Swing the heating element over the plastic. Observe the material as it heats.
6. Material will sag approximately ½" – this is the optimal thermoforming time.
7. Move the heating element away from the plastic.
8. Turn on the vacuum and lower the plastic over the model.
9. Immediately spray the plastic encased model with Essix Freeze Spray® Coolant. Not only does it cool the plastic, prohibiting further thinning, but it "shrinks" the material around the cast, realizing a tighter fit.
10. Let vacuum continue to run until plastic is cool.
11. Trim and finish appliance as desired.

*Heating time may vary depending upon machine. If plastic does not adapt to model, add heating time until adaptation occurs. If plastic "folds" when thermoformed, reduce heating time until material does not fold.

For step-by-step instructions refer to owner's manual of thermo-forming machine used.

Thickness	Vacuum Heat Time**	Essix® SelectVac®	Drufomat
4mm (.160")	75 seconds	2:30	2:00
5mm (.200")	110 seconds	3:30	2:25

**after preheating element

SAFE DISPOSAL:

Unused plastic sheets can be recycled.

Used dental appliances may be considered a biohazard. Follow your country-specific laws, directives, standards and guidelines for the disposal of used devices.

Manufacturer: **Dentsply Sirona Orthodontics Inc.**
7290 26th Court East Sarasota, FL 34243 USA

 www.dentsplysirona.com/orthodontics-e-labeling

DO NOT
REUSE

MKT-DFU0049-ML Rev. 01 Issued 08-2020

Drufomat is not a trademark of
Dentsply Sirona.

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©2020 Dentsply Sirona