

## Enhance® Finishing System

# Coarse when you need it, fine when you want it.

The Enhance Finishing System accomplishes the entire process of excess removal, contouring, and finishing simply by varying the pressure. Push harder to remove excess composite, ease it off for final results. In just one step the Enhance Finishing System delivers a smooth, contoured surface with a natural looking finish. It's all about pressure. **Finish in one step.**



NEW

Enhance® mini

Improved access and versatility

# Fewer Steps. Less Time. Impressive Results.

More than 70% of dentists say that simplicity is one of the most important factors for a finishing and polishing system.<sup>1</sup> That's why we designed the Enhance Finishing System. Achieve the entire finishing process in one step. It doesn't get simpler than that!


Apply more pressure for excess removal and initial contouring; back it off for a naturally smooth surface. With no need to change instruments, you can stay focused on the procedure to achieve a better finish, faster.



Excess Removal and Contouring




Finishing

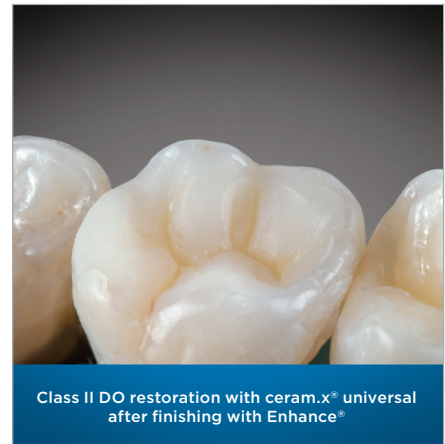

Class II DO restoration with ceram.x® universal after finishing with Enhance®

## Maximum gloss per system

After achieving a natural looking finish with Enhance, create a high-end luster with the Enhance® PoGo® diamond polishing system.

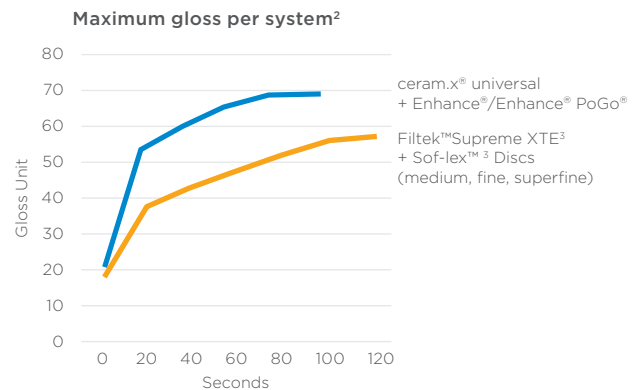

## Ordering Information

### Enhance® Finishing System Refills

40 Finishing Discs	<b>624045</b>
40 Finishing Cups	<b>624055</b>
40 Finishing Points	<b>624065</b>

### Enhance®/Enhance® PoGo® Complete Kit **662020**

Enhance® Finisher	10 each: Discs, Cups, Points
Enhance® PoGo® Polisher	10 each: Discs, Cups, Points

### Enhance® mini Intro Kit **624385**

Enhance® mini	20 each: Discs, Cups, Points
---------------	------------------------------

### Enhance® mini Finishing System Refills

40 Finishing Discs	<b>624345</b>
40 Finishing Cups	<b>624355</b>
40 Finishing Points	<b>624365</b>



For more information visit [www.dentsplysirona.com](http://www.dentsplysirona.com)

Find us on  [www.facebook.com/dentsplysirona.restorative](https://www.facebook.com/dentsplysirona.restorative)



1. Exevia: Dental Shuttle Q4, 2016, Germany  
 2. da Costa J, Ferracane JL (OHSU, Portland, OR)  
 3. Not a registered trademark of Dentsply Sirona, Inc.  
**K79200260-00**

© Dentsply Sirona Inc. 06/2017