---

Neu ab:  
New as of:  
Nouveau depuis:  
Nuevo a partir de:

---

03.2019

 Dentsply  
Sirona

# Hugo, Theo

Gebrauchsanweisung  
Operating Instructions  
Notice d'utilisation  
Manual del operador

Deutsch / English / Français / Español

# Hugo, Theo

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Deutsch

# Hugo, Theo

Operating Instructions

English

# Hugo, Theo

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Français

# Hugo, Theo

Manual del operador

Español

---

Neu ab:

03.2019

---

# Hugo, Theo

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Deutsch

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English

## Table of contents

<b>1</b>	General data .....	<b>3</b>
1.1	Dear Customer,.....	3
1.2	Contact data.....	3
1.3	Structure of the document.....	4
1.3.1	Identification of the danger levels .....	4
1.3.2	Formats and symbols used.....	4
<b>2</b>	Safety information .....	<b>5</b>
2.1	Modifications to the product .....	5
2.2	Accessories.....	5
2.3	Maximum load capacity .....	5
<b>3</b>	Product overview .....	<b>6</b>
3.1	Hugo working stool .....	6
3.2	Theo working stool.....	7
<b>4</b>	Operation .....	<b>8</b>
4.1	Height adjustment with manual release .....	8
4.2	Height adjustment with foot release.....	9
4.3	Backrest .....	9
4.4	Seat tilt .....	10
<b>5</b>	Care and cleaning.....	<b>11</b>
5.1	Care, cleaning agents, and disinfectants .....	11
5.2	Care and cleaning.....	12
<b>6</b>	Technical data .....	<b>13</b>
6.1	Hugo working stool .....	13
6.2	Theo working stool.....	14

# 1 General data

## 1.1 Dear Customer,

Thank you for purchasing your Hugo working stool from Dentsply Sirona.

The technical documentation supplied is also part of the product. You should always keep this documentation within reach.

These Operating Instructions show the various models of the working stool.

Care and cleaning should be carried out based on the corresponding instructions.

It is important to observe all safety information to prevent personal injury and material damage.

Your Dentsply Sirona team

## 1.2 Contact data

### Customer service center

For technical questions, use the contact form on the internet at [www.sirona.com](http://www.sirona.com). Follow the menu items "*CONTACT*" / "*Customer Service Center*" on the navigation bar and click on the "*CONTACT FORM FOR TECHNICAL QUESTIONS*" button.

Sirona Dental Systems GmbH  
Fabrikstrasse 31  
64625 Bensheim  
Germany

Phone: +49 (0) 6251/16-0  
Fax: +49 (0) 6251/16-2591  
E-mail: [contact@sirona.com](mailto:contact@sirona.com)  
[www.sirona.com](http://www.sirona.com)

## 1.3 Structure of the document

### 1.3.1 Identification of the danger levels

To prevent personal injury and material damage, please observe the warning and safety information provided in these operating instructions. Such information is highlighted as follows:

 <b>DANGER</b>
An imminent danger that could result in serious bodily injury or death.

 <b>WARNING</b>
A possibly dangerous situation that could result in serious bodily injury or death.

 <b>CAUTION</b>
A possibly dangerous situation that could result in slight bodily injury.

<b>NOTE</b>
A possibly harmful situation which could lead to damage of the product or an object in its environment.

<b>IMPORTANT</b>
Application instructions and other important information.

**Tip:** Information for simplifying work.

### 1.3.2 Formats and symbols used

The formats and symbols used in this document have the following meaning:

<ul style="list-style-type: none"> <li>✓ Prerequisite</li> <li>1. First action step</li> <li>2. Second action step</li> <li>or</li> <li>➤ Alternative action</li> <li>↔ Result</li> <li>➤ Individual action step</li> </ul>	Requests you to do something.
See "Formats and symbols used [→ 4]"	Identifies a reference to another text passage and specifies its page number.
• List	Designates a list.
"Command / menu item"	Indicates commands / menu items or quotations.

## 2 Safety information

### 2.1 Modifications to the product

Modifications to this product which may affect the safety of the operator, patients or third parties are prohibited by law!

### 2.2 Accessories

In order to ensure product safety, this device may be operated only with original Dentsply Sirona accessories or third-party accessories expressly approved by Dentsply Sirona. The user is responsible for any damage resulting from the use of non-approved accessories.

### 2.3 Maximum load capacity

Please observe the maximum load capacity of the working stool of **max. 135 kg**.

## 3 Product overview

### 3.1 Hugo working stool

#### "Manual" model

Height adjustment with manual release

#### "Manual Plus" model

Height adjustment with manual release and foot ring

### "Manual Freehand" model

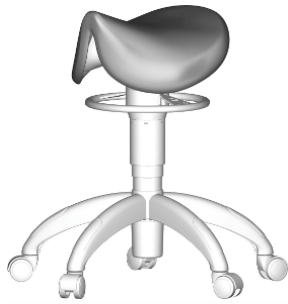
Height adjustment with foot release

## 3.2 Theo working stool

### "Manual" model

Height adjustment with manual release

## 4 Operation

### 4.1 Height adjustment with manual release

#### Hugo working stool

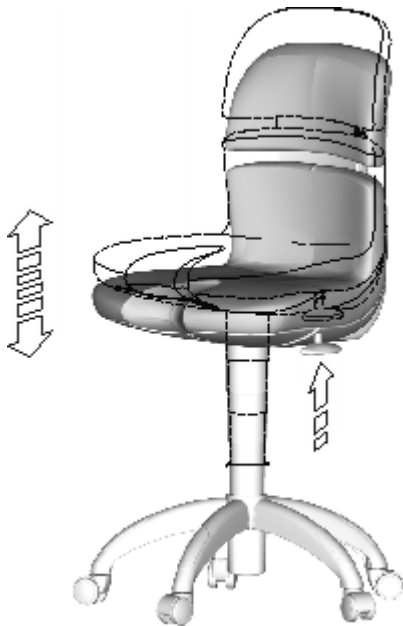
The operating lever for manual release is located **to the left underneath the seat**.

- **Raising**

Take the weight of your body off of the seat as much as possible and pull the operating lever upward.

- **Lowering**

Leave the weight of your body on the seat and pull the operating lever upward.

#### Theo working stool

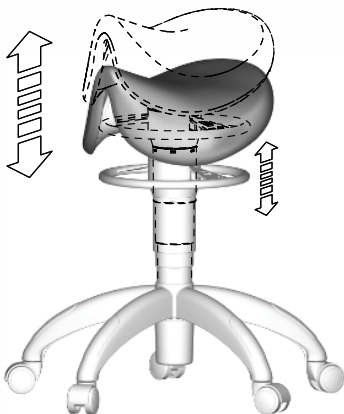
The operating ring for manual release is located **underneath the seat**.

- **Raising**


Take the weight of your body off of the seat as much as possible and press or pull the operating ring.

- **Lowering**

Leave the weight of your body on the seat and press or pull the operating ring.

## 4.2 Height adjustment with foot release

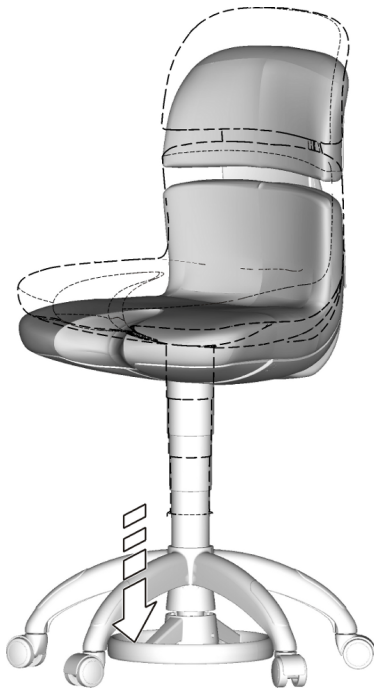- **Raising**  
Take the weight of your body off of the seat as much as possible and press the release ring with your foot.
- **Lowering**  
Leave the weight of your body on the seat and press the release ring with your foot.

When you remove your foot, the seat remains at its current height.

## 4.3 Backrest

To adjust the backrest, simply lean back.

<b>IMPORTANT</b>
------------------

No operating lever
--------------------

<b>IMPORTANT</b>
------------------

<b>Backrest not adjustable!</b>
---------------------------------

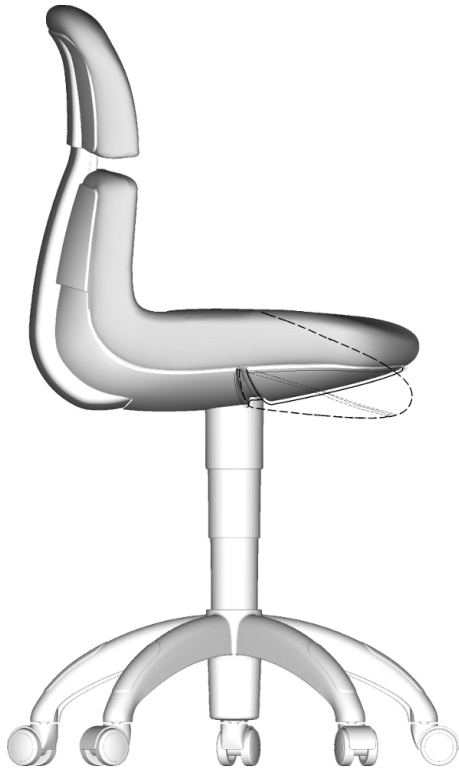
It is not possible to adjust the height or depth of the backrest.
---

## 4.4 Seat tilt

### Hugo working stool

The front edge of the working stool's seat can be lowered.

The operator can lower the sides of the seat individually by pressing against them with his thighs.

<b>IMPORTANT</b>
------------------

No operating lever
--------------------

### Theo working stool

The front edge of the working stool's seat has an adjustable balance system.

The balance system enables the seat to adjust flexibly to your sitting posture by tilting and slanting. When relieved, the seat swings back to its original position.

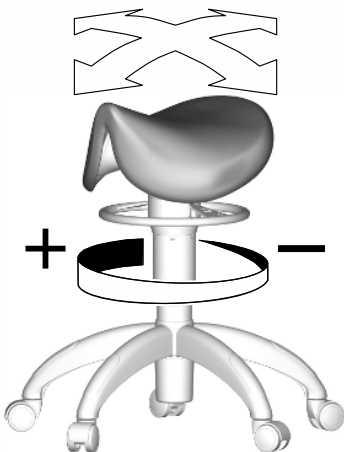
The balance system's range of motion can be set by turning the operating ring.

- **Increasing the range of motion**

Turn the operating ring in a clockwise direction to increase the range of motion of the seat.

- **Reducing the range of motion**

Turn the operating ring in a counterclockwise direction to reduce or block the range of motion of the seat.

## 5 Care and cleaning

### 5.1 Care, cleaning agents, and disinfectants

#### NOTE

##### Approved care, cleaning, and disinfecting agents

Use only care, cleaning, and disinfecting agents approved by Dentsply Sirona!

A continuously updated list of approved agents can be downloaded from the Internet on the online portal for technical documents. You can reach this portal at the address:

[www.dentsplysirona.com/manuals](http://www.dentsplysirona.com/manuals)

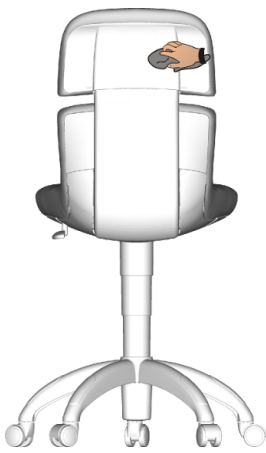
.Click on the menu item "*General documents*" and then open the "*Care, cleaning and disinfection agents*" document.

If you do not have access to the internet, please contact your dental depot to order the list (REF 59 70 905).

## 5.2 Care and cleaning

- **Disinfecting**  
Surfaces can be sprayed or wiped with surface disinfectants.
- **Cleaning**  
Remove dirt, grime and disinfectant residue regularly using mild, commercially available cleaning agents.
- **Upholstery**  
Use only media approved by Sirona for upholstery; see "Care, cleaning, and disinfecting agents"!
- **Protection against medicaments**  
Due to their high concentrations and the substances they contain, many medicaments can dissolve, etch, bleach or discolor surfaces. The only way to prevent damage is to wipe off medicaments immediately with a damp cloth and a cleaning agent!
- **Castors**  
Clean the castors of the working stool at regular intervals to prevent damage to the floor.


## 6 Technical data

### 6.1 Hugo working stool
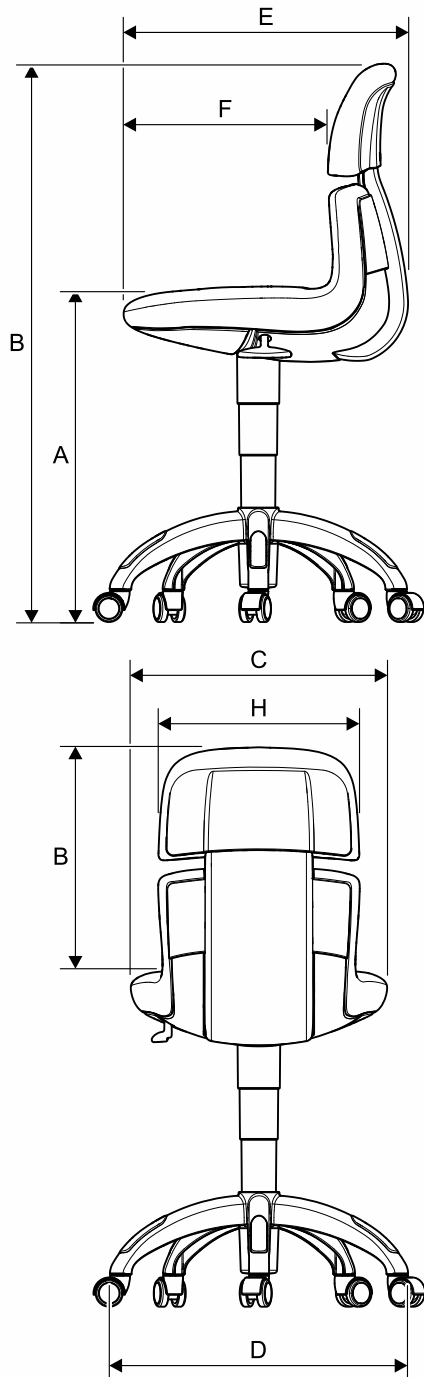
#### Dimensions



		Dimensions in mm
<b>A</b>	Seat height (acc. to DIN, loaded)	
	Pneumatic spring, short	405-540
	Pneumatic spring, long	505-645
<b>B</b>	Total height (in lowest position, unloaded)	
	Pneumatic spring, short	860
	Pneumatic spring, long	960
<b>C</b>	Seat width	470
<b>D</b>	Star base	577
<b>E</b>	Total depth	525
<b>F</b>	Seat depth (acc. to DIN, in lordosis support point)	415
<b>G</b>	Backrest height	420
<b>H</b>	Backrest width	370

#### Total weight

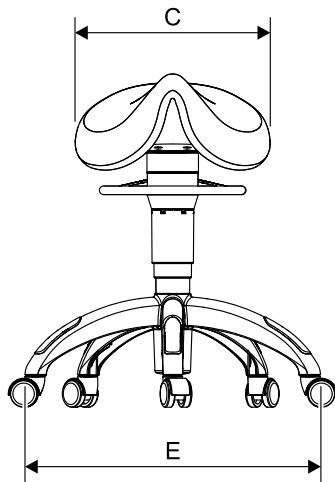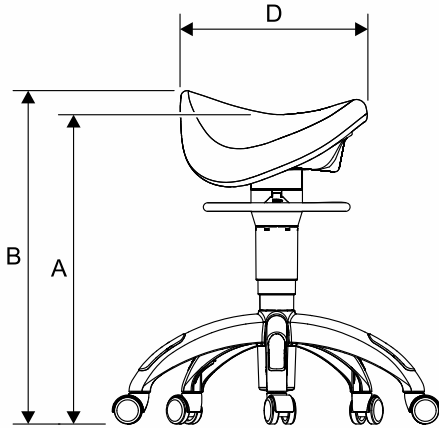11.5–12.0 kg – suitable for persons weighing up to 135 kg

## 6.2 Theo working stool

### Dimensions

		Dimensions in mm
<b>A</b>	Seat height (unloaded)	
	Pneumatic spring, short	556 - 685
	Pneumatic spring, long	630 - 765
<b>B</b>	Total height (in lowest position, unloaded)	
	Pneumatic spring, short	596
	Pneumatic spring, long	670
<b>C</b>	Seat width	340
<b>D</b>	Seat depth	300
<b>E</b>	Star base	525

### Total weight

7.0–7.7kg – suitable for persons weighing up to 135 kg

---

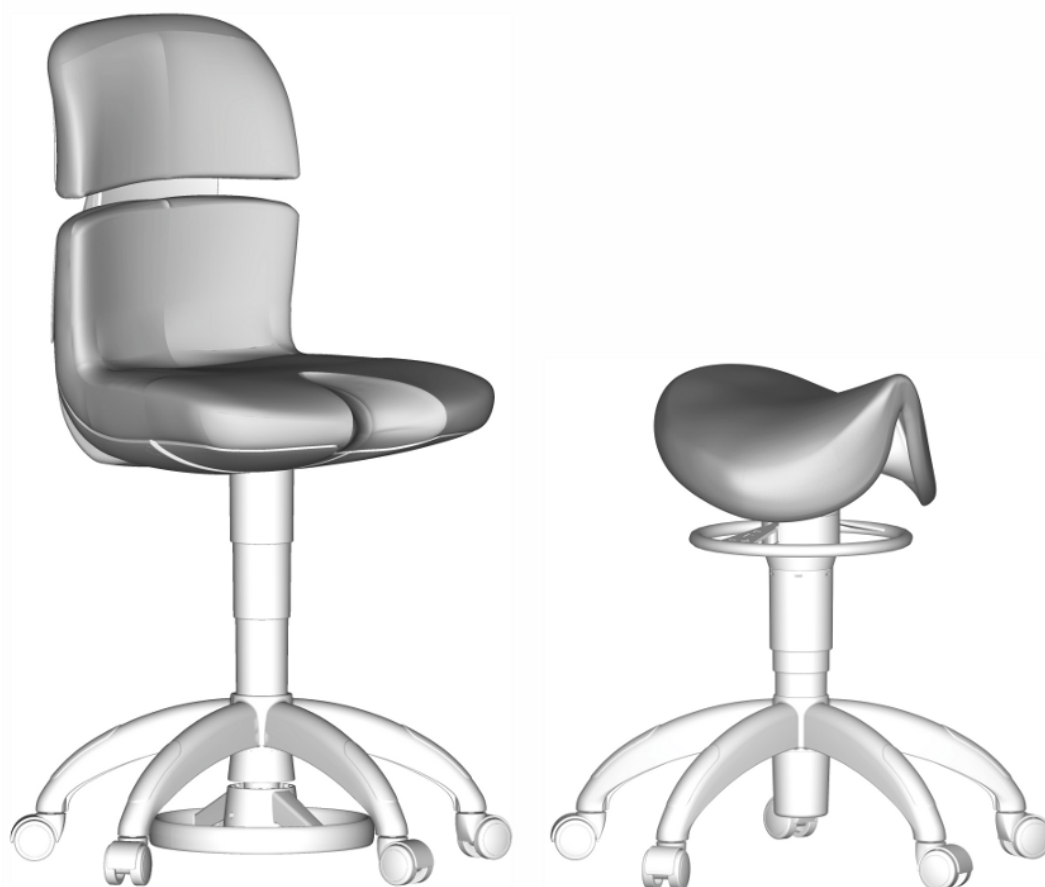
We reserve the right to make any alterations which may be required due to technical improvements.

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© Sirona Dental Systems GmbH