

- 12 Press button 2 times to turn off device. Do not remove the tip from
- 13 Irrigate, then use intracanal suction to eliminate loose debris.
- 14 Repeat the above steps for each intracanal irrigant used.



Scan here to easily access resources and videos for vour system!



Endodontic Activation System User Interface







Reorder Information



644434 SmartLite Pro EndoActivator™ Barrier Sleeves (QTY 100)



644435 SmartLite Pro EndoActivator™ Tips (QTY 25) Small: 22mm (15/02)



644436 SmartLite Pro EndoActivator™ Tips (QTY 25) Medium: 22mm (25/04)



644437 SmartLite Pro EndoActivator™ Tips (QTY 25) Medium Long: 28mm (25/04)

dentsplysirona.com/ifu

