

Maintenance of Turbines and Contra Angle Handpieces

Manual Preparation

Perform on the Treatment Centre

Purge the handpiece at the treatment centre for 30 seconds.



Remove the burr with suitable tweezers.



Clean the handpiece with disinfection wipes which **DO NOT** contain Chlorine.



Cleaning in the Dirty Area

Dry the handpiece with a cloth and remove the handpiece from the hose. Bring it to the dirty area.



DO NOT soak the handpiece in solution or ultrasonic bath.




Brush the handpiece under tap water for at least 10 seconds. Brush from the body to the head to prevent dirty water from entering.




Lubrication

Use the oil spray adapter for the appropriate handpiece.

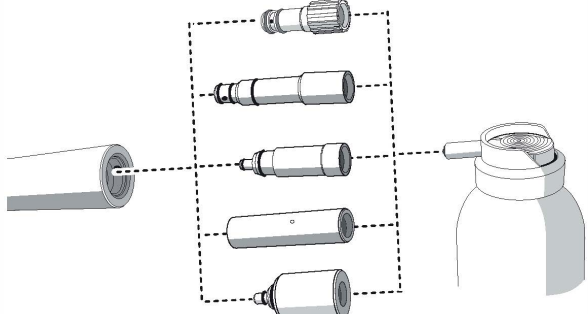



Quick Coupling Turbines
Contra Angles Handpiece

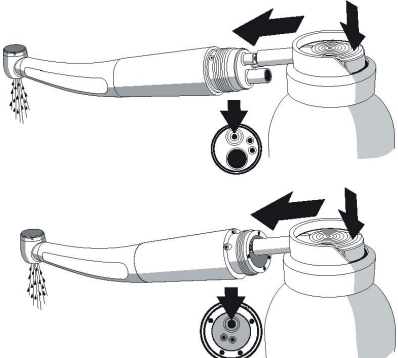

Fixed Connection Turbines



Hold the handpiece head downward in the cloth. Spray for 2 seconds. Repeat until liquid emerging is clean.

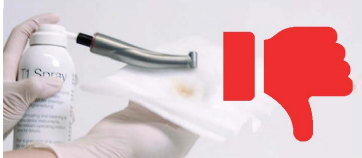

T1 Spray
REF 59 01 665

Sterilisation and Before Use

Package the handpiece. Sterilise the handpiece in EN13060 class B or S steam steriliser.



134°C 274°F



Store until next treatment.



Purge the handpiece at the treatment centre for 10 seconds before use.



Weekly Maintenance

Spray the chucking system from the burr side for 2 seconds.



Clean the nozzles of dirt and deposits with nozzle cleaner.



Cleaning Wire
REF 24 00 232

Clean the light with alcohol. Dry it with a cloth.

