# inLab Speedcure™

Processing Unit  
Verarbeitungseinheit  
Appareil de traitement  
Unità di elaborazione  
Unidad de procesamiento

Instruction Manual  
Bedienungsanleitung  
Manuel d'instructions  
Manuale di istruzioni  
Manual de instrucciones

| Model Number      | Voltage  | Frequency  |
|-------------------|----------|------------|
| Modellnummer      | Spannung | Frequenz   |
| Réf. modèle       | Tension  | Fréquence  |
| Numero di modello | Tensione | Frequenza  |
| Número de modelo  | Voltaje  | Frecuencia |
| 9494800           | 120V     | 50 - 60 Hz |
| 9494801           | 220-240V | 50 - 60 Hz |

# TABLE OF CONTENTS

|   |    |
|---|----|
| Safety .....                                | 1  |
| Technical Data .....                        | 2  |
| Installation Instructions .....             | 3  |
| General Operation .....                     | 4  |
| User Maintenance .....                      | 10 |
| Bulb and Filter Replacement.....            | 10 |
| Fuse Replacement (220-240V Unit Only) ..... | 13 |
| Troubleshooting .....                       | 14 |
| Product Service .....                       | 16 |
| Disposal of Unit.....                       | 16 |
| Warranty .....                              | 17 |

# SAFETY

## FOR PROFESSIONAL USE ONLY.

Please read these operating instructions carefully before installing or operating this equipment.

- Use indoors only.
- Never operate the unit in close proximity to combustible materials or place materials on top of the unit.
- The unit must be electrically grounded to a three-wire electrical outlet or receptacle. The electrical service provided must be a dedicated line of the proper size according to local electrical codes.
- Unit must be placed in a position that allows the power cord to be easily disconnected from the wall.
- Do not attempt to service the unit until you have read and understand this instruction manual.
- Turn off the power switch and disconnect the power cord before attempting to service the unit.
- Do not operate the unit controls with tongs or other tools.
- Do not use solvents or liquid cleaners on the control panel.
- Do not touch the bulbs when the unit door is open.
- Before replacing bulbs, allow the unit to cool to room temperature to avoid burns.
- Do not cover the top of the unit or obstruct the rear fans in anyway.
- Do not heat water or food in the unit chamber.
- Do not submerge the unit into any liquid.
- Do not store flammable products or solvents inside the unit chamber.
- Follow the manufacturer's instructions for use for maximum appliance load per process and cure cycle.
- If the unit is not operated in the manner as specified in this manual, the protection provided by the unit may be impaired.
- Pollution Degree: 2

**CAUTION: RISK OF DANGER! BURN HAZARD IS PRESENT WHEN COVER IS OPENED.  
USE CAUTION AFTER OPENING COVER, BULBS AND METAL SURFACES MAY BE HOT!  
USE HEAT-RESISTANT GLOVES WHEN REMOVING BULBS!**

## SYMBOL TABLE

|  |   |
|--|---|
|  | Alternating current   |
|  | On (Power Switch)   |
|  | Off (Power Switch)  |
|  | Caution, Hot Surface  |
|  | Protective Conductor  |
|  | Caution, Risk of Danger   |
|  | High Intensity Light  |
|  | Dispose of in accordance with the Waste Electrical and Electronic Equipment Directive 2012/19/EU of the European Parliament and the Council of the European Union |

# TECHNICAL DATA

## ELECTRICAL RATINGS:

| Model Number | Voltage  | Amps | Frequency | Watts |
|--------------|----------|------|-----------|-------|
| 9494800      | 120V     | 12A  | 50 - 60Hz | 1200W |
| 9494801      | 220-240V | 10A  | 50 - 60Hz | 1200W |

## DIMENSIONS:

Length: 510mm (20.1")

Width: 366mm (14.4")

Height: 385mm (15.2")

## WEIGHT:

25kg (55 lbs)

## OPERATING TEMPERATURE:

5°C - 40°C (41°F - 104°F)

## FUSE (2):

Mains: F 250V 10A (220-240V only)

## SERVICE PARTS:

Bulb (1 each) 905032

Filter (1 each) 905034

10A Fuse (220-240V only) (1 each) 9320069

# INSTALLATION INSTRUCTIONS

Shipping damage should be reported to the carrier as soon as detected.

## The unit package contains the following:

- 1 - Instruction Manual (this document)
- 1 - inLab Speedcure Processing Unit
- 2 - Spare Bulbs
- 1 - Turntable
- 1 - Gloves (pair)
- 2 - 10A Fuse (220-240V only)

**UNPACKING NOTE:** Save the box and packing materials for future use when transporting the unit.

- Open the box.
- Remove the accessory kit.
- Carefully remove the unit from the box by lifting the packaging supports - lift metal base only. Support unit from the bottom to prevent slipping. **DO NOT LIFT UNIT USING PLASTIC SIDE COVERS.**
- Remove packaging supports.
- Remove plastic bag.
- Remove remaining packaging material on the outside of unit.
- Remove turntable from the top of unit.

## INSTALLATION

- Carefully place the unit on a solid flat surface. Position the unit at least 10cm (4 inches) from the wall or other structures to allow for good airflow.
- Connect unit to a grounded power source.
- Open door and install turntable onto motor shaft, match flat sides of shaft and turntable.

### **WARNING:**

Never operate the unit in close proximity to combustible materials or place materials on top of the unit.

Do not use extension cords with this equipment.


Plug the power cord into a circuit rated a minimum of 15A (120V) or 10A (220-240V).

Do not position the unit in a way that blocks access to the wall outlet or the powercord inlet socket.

Do not position the appliance(s) in a way where there is direct contact with a bulb.

# GENERAL OPERATION

## FUNCTION KEY DESCRIPTIONS

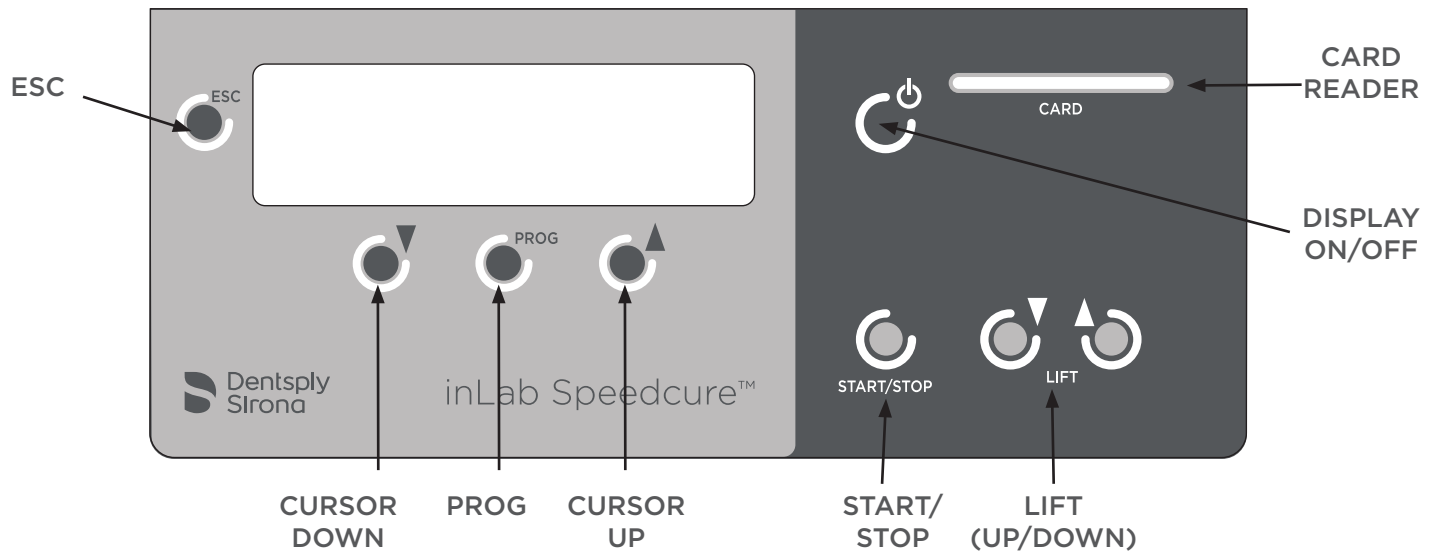**ESC:** Aborts changes, exits menus and clears errors on the screen.

**CURSOR DOWN:** Moves cursor on menu - down.

**PROG:** To view process parameters.

**CURSOR UP:** Moves cursor on menu - up.

**START/STOP:** START or STOP the selected process. Press once to confirm the process name, lamps used and cycle time. Press again to start the process.

**LIFT (UP/DOWN):** Raise and lower the work platform.

**DISPLAY ON/OFF:** Turns display ON & OFF. Unit is in standby when "OFF."

**DISPLAY ON/OFF DOES NOT DISCONNECT ELECTRICAL POWER. BEFORE SERVICING, FIRST SWITCH THE POWER TO THE OFF POSITION AT THE REAR OF THE UNIT. THEN UNPLUG THE UNIT FROM THE ELECTRICAL SOURCE.**


**CARD READER:** Used to load new material programs.

# GENERAL OPERATION (CONTINUED)

## POWER UP SWITCH

The power switch is located on the rear of the unit, as shown.

Turn the power switch to the ON position, as shown.

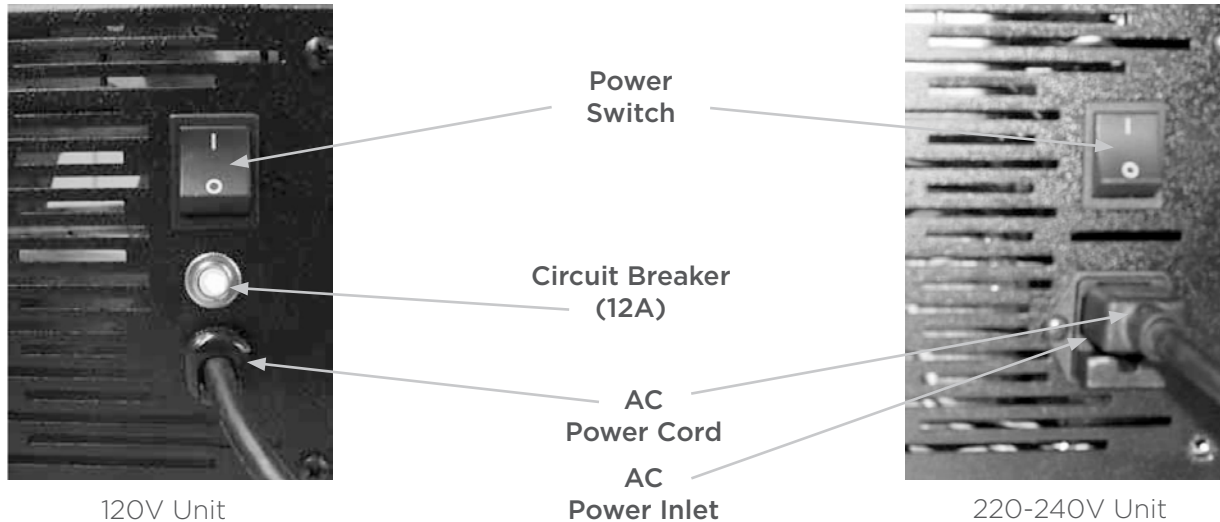The display will turn on and a language screen will be displayed for a short period of time. This screen will advance to the process menu automatically.

## LANGUAGE SELECTION

There are five languages available: English, German, French, Italian and Spanish.

The language selection screen can be accessed at Power Up or by pushing the ESC key once when viewing the process menu. If no selection is made within 5 seconds the unit will default to the process menu.

## LANGUAGE SELECTION SCREEN


To select a language, use the Cursor Up & Cursor Down keys adjacent to the display within 5 seconds. Once the preferred language is highlighted, the 5 second timer will reset; wait 5 seconds to allow the unit to change to the process menu with the new language choice.

## GENERAL OPERATION (CONTINUED)


When the unit is turned on, by means of the power switch, a diagnostic test is performed to check the operation of key systems in the unit. The screen shown below will be displayed. If a problem exists, an error will be displayed. Reference errors in the Troubleshooting section of this manual.

### INITIAL DIAGNOSTICS CHECK SCREEN (DOOR MUST BE CLOSED!)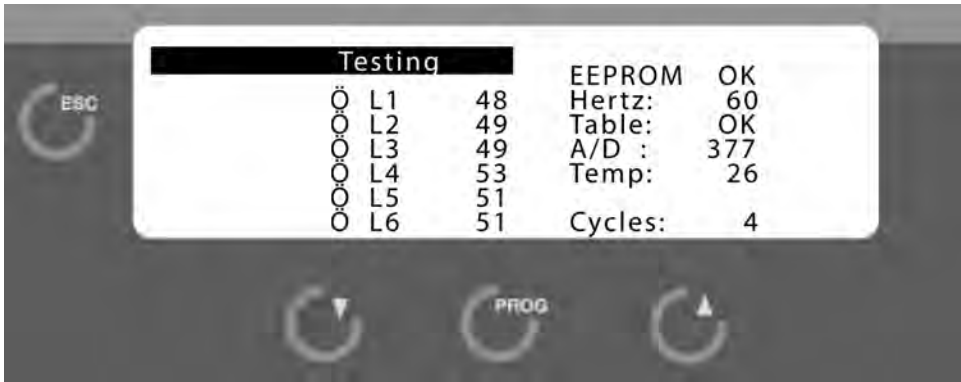


After a successful diagnostics test the process menu will be displayed.

### SELECT PROCESS

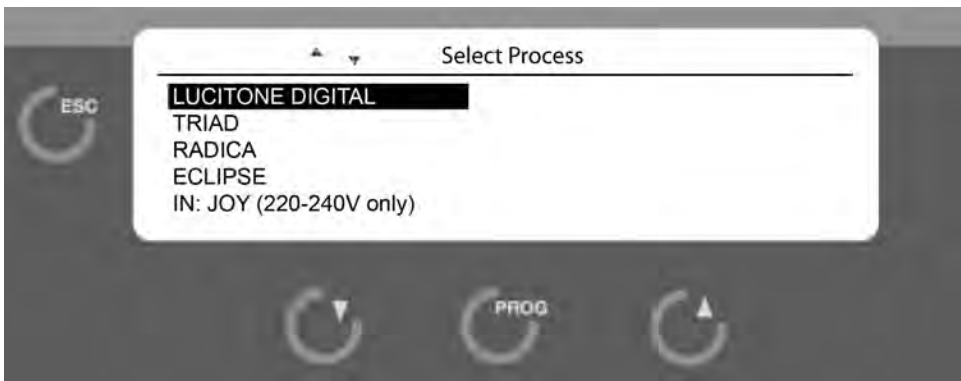Use Cursor Up/Down keys to select process, press PROG.

### PROCESS MENU SCREEN

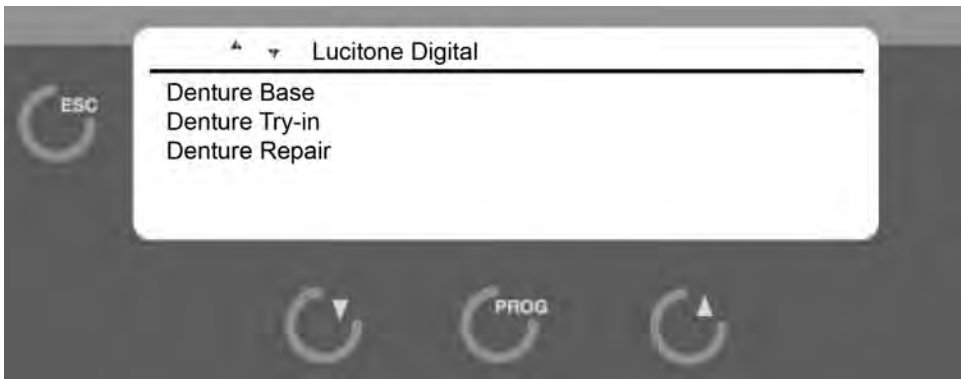
Use the Cursor Up/Down keys to select a process, then press PROG.

# GENERAL OPERATION (CONTINUED)

Once a process is selected, push the PROG key or the START/STOP key to confirm the Process Parameter Screen.

## PROCESS PARAMETER SCREEN

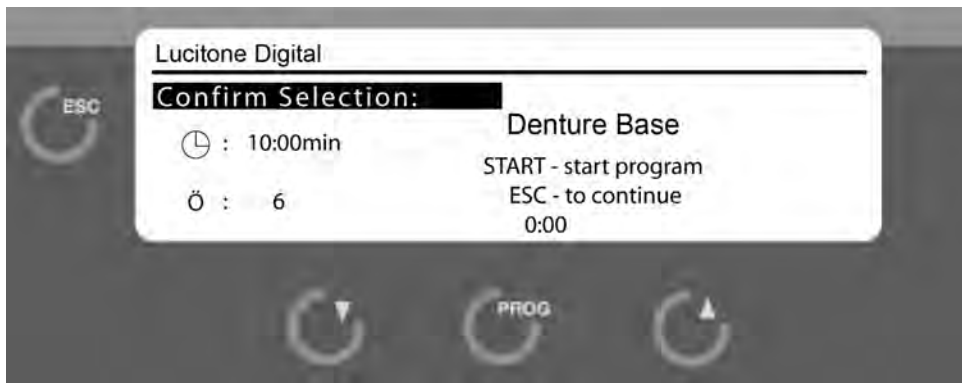This screen shows the name of the process selected, the number of lamps used, and its cycle time (excluding the 3 minutes of cooling time).

To return to the Process Menu press the ESC key or wait 15 seconds, and the screen will automatically return to the main menu. A timer is displayed at the bottom of the screen indicating how much time is left to start the process.

To run this process the START/STOP key must be pushed again.

Once started, the process running screen is shown.


## PROCESS RUNNING SCREEN

The process will run for the length of the processing time displayed. The unit will pause at the end of process to check table rotation; a long beep indicates the start of the cooling phase.

# GENERAL OPERATION (CONTINUED)

## COOLING SCREEN

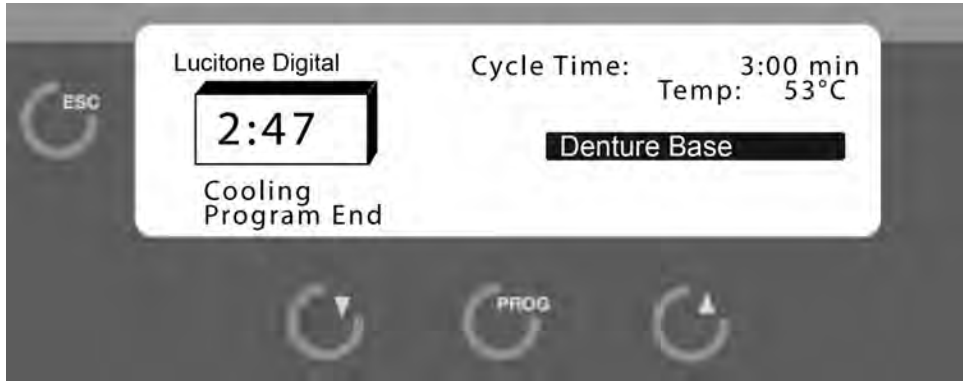Once cooling is completed, the display will return to the process menu. The last process used will be highlighted on the process menu. Do not remove the appliance until the cool cycle has ended.

**CAUTION: Models may still be hot - Use caution when removing materials from chamber.**

## ABORT PROCESS


When the START/STOP key has been pressed during a process, the following screen is shown to confirm the aborting of the process.

## ABORT PROCESS SCREEN

ESC cancels the abort command.

## ABORT PROCESS COOLING SCREEN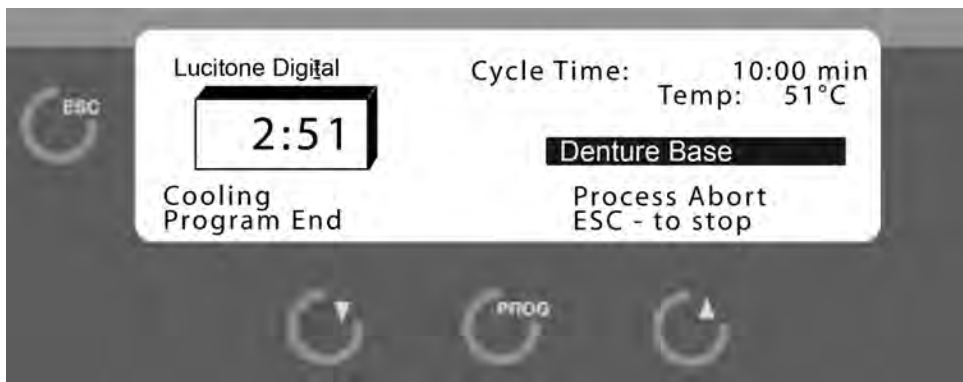

If START/STOP was pushed or the door was opened during the process, a long beep will sound and the cooling screen above will be shown.


Remove appliance only after the cooling cycle has ended.


**CAUTION: Models may still be hot - Use caution when removing materials from chamber.**

# GENERAL OPERATION (CONTINUED)

## LOADING CARD PROGRAMS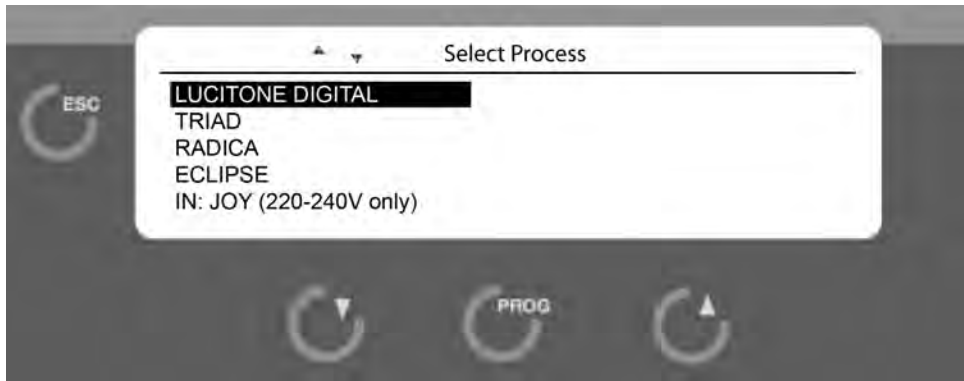

From this screen, insert card with label facing up.

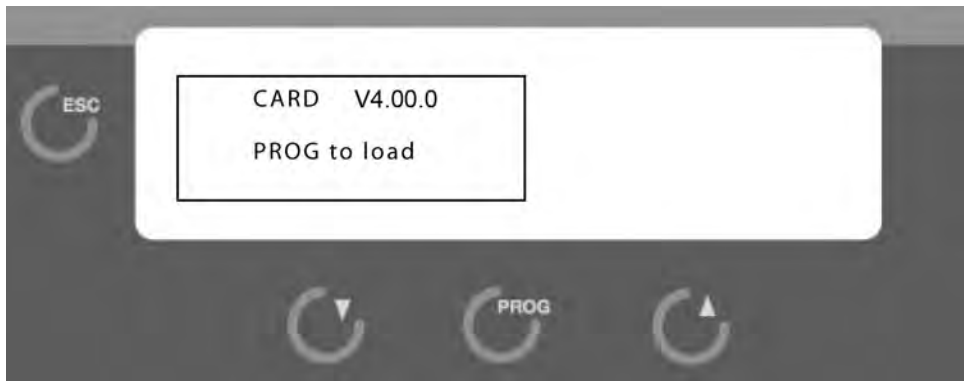Press the PROG key to load data.

Program loading.

Program loaded, remove card. Display will return to Select Process.

# USER MAINTENANCE

## WARNING:

This equipment contains dangerous voltages. Maintenance and repair work should only be performed by an authorized service technician of Dentsply Sirona.

To keep the equipment in good working order follow the guidelines below.

- Examine the equipment regularly for mechanical damage.
- Make sure the lift is not obstructed in any way.
- Do not overload the table (1 lb. max), this can cause the motor to turn slower and cause wear on the bearings.
- Do not attempt to manually force the lift when unit is powered up. This may result in motor damage.

## CLEANING

- Clean the unit at least once a week.
- Clean exterior surfaces with a soft, damp cloth using a mild detergent and water.
- Clean the fan and exhaust vents, using low pressure compressed air, once a month or as needed to prevent overheating.
- Do not use solvents or liquid cleaners on the control panel; they could enter the panel and cause damage.
- DO NOT IMMERSE IN WATER!

# BULB AND FILTER REPLACEMENT

## CAUTION:

TURN OFF THE POWER SWITCH AND UNPLUG UNIT FROM THE WALL OUTLET BEFORE PERFORMING ANY MAINTENANCE ON THE UNIT.


## CAUTION:

RISK OF DANGER! BURN HAZARD IS PRESENT WHEN COVER IS OPENED. USE CAUTION AFTER OPENING COVER, BULBS AND METAL SURFACES MAY BE HOT! ALLOW UNIT TO COOL AT LEAST 15 MINUTES BEFORE SERVICING. USE HEAT RESISTANT GLOVES WHEN REMOVING BULBS!

**NOTE: ALLOW THE UNIT TO COOL COMPLETELY BEFORE REPLACING BULBS TO ELIMINATE BURN HAZARDS.**

When a lamp is burned out, its location number will flash on the display. In the picture below, L2 is burned out. Follow the instructions on the next page for bulb replacement. ONLY USE REPLACEMENT BULBS SUPPLIED BY DENTSPLY SIRONA.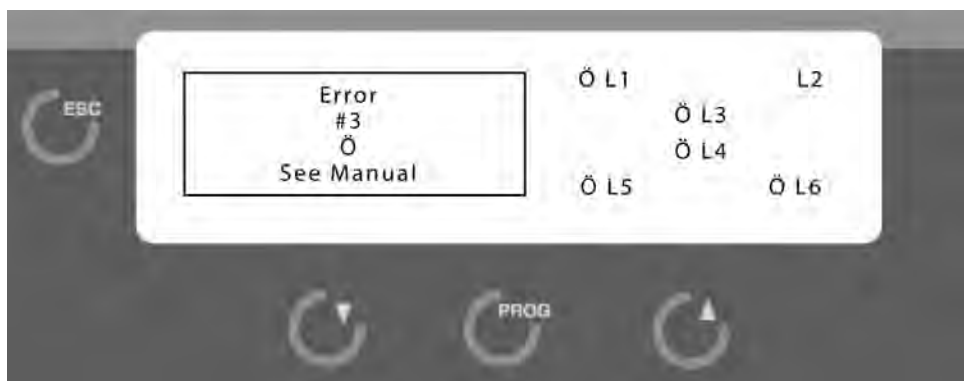

# BULB AND FILTER REPLACEMENT (CONTINUED)

1. Turn the power switch to the OFF position.

2. Unplug the AC power cord from the wall outlet.

3. Loosen the two screws on rear of top panel.

4. Slide top panel backwards until full access to bulbs is achieved.

5. Put supplied gloves on hands.

6. Loosen center bulb bracket retaining screws, if needed, for bulb clearance.

Front of unit

Loosen 2 screws.

**Failure to wear gloves could cause premature lamp failure due to skin oils.**

7. Place center bulb bracket in slots on left side of internal cover. Be careful not to remove the filters in the bracket. Do not touch the filters with bare hands, handle with gloved hands only. If filters are being replaced, the filters have a frosted side and a shiny side. The frosted side should be placed toward the lamp. **ONLY USE REPLACEMENT FILTERS SUPPLIED BY DENTSPLY SIRONA.**

Frosted Side UP!

# BULB AND FILTER REPLACEMENT (CONTINUED)

8. Push the bulb release lever far enough to loosen the bulb from the socket.

9. Remove bulb.


- 10 **IMPORTANT!** Return the bulb release lever to forward position.

11. Insert new bulb. Retaining spring should slide over bulb.

- 12 Reinstall the center bulb bracket, tighten retaining screws.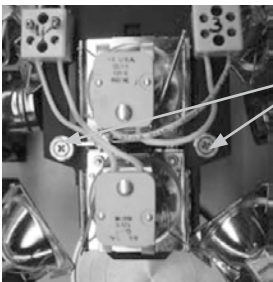

Tighten  
2 screws

13. Close cover, secure with 2 screws.

14. Plug AC Power Cord into wall outlet. Turn power switch to ON position.

# FUSE REPLACEMENT (220-240V UNIT ONLY)

Mains - located in the rear of the unit inside the inlet socket.

1. Turn the power switch OFF.

2. Unplug the power cord from AC power inlet.

3. Remove fuse drawer from inlet socket (contains 2 fuses) using small screwdriver to push on the locking clip.

4. Remove fuse drawer.

5. Replace fuse(s) with supplied F 250V 10A fuse(s).

6. Push fuse drawer into inlet socket until the clip locks.

7. Plug Unit power cord into AC power inlet.


8. Turn Power Switch to ON position.

# TROUBLESHOOTING

NO TABLE ROTATION OR TABLE IS NOT IN THE DOWN POSITION AT THE START OF THE PROCESS.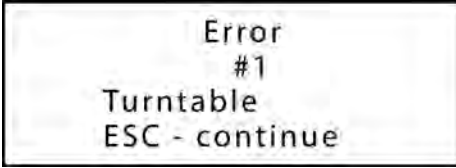


## ERROR 1

Indicates table rotation was not detected, or table is not in the down position at start of menu.

- Make sure the table handle is pushed completely forward and the lift is in the DOWN position, then push ESC. If the lift is still elevated, turn OFF the unit, open the door and gently push the table down to the stop position. Turn the unit ON.

## OVER TEMPERATURE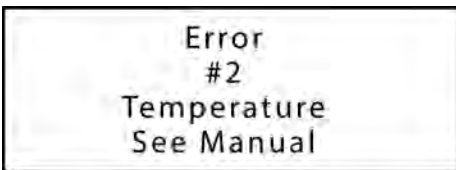


## ERROR 2

The temperature of the unit has exceeded the maximum temperature allowed.

- Turn unit OFF then ON.
- Check blower operation.
- Check fan operation.

## BULB FAILURE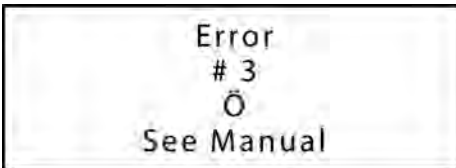

## ERROR 3


One or more bulbs failed.

- The bulb layout seen on the display is the same as arranged in the unit, as viewed from the top.
- The failed bulb will blink on the display.
- Follow the bulb replacement procedure in the Maintenance Section.

**NOTE: A power surge may cause any of the above errors to appear. Follow the instructions provided to clear the error and resume normal operations. If any error persists, call Dentsply Sirona service department for assistance.**

# TROUBLESHOOTING (CONTINUED)


## LIFT OPTICS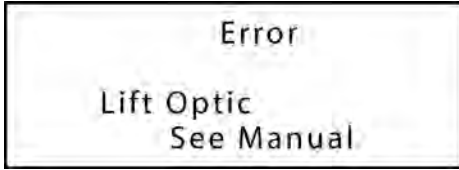

The horizontal light beam was not detected to stop lift.

- Confirm the lift is not obstructed and is in the down position.


## POWER FAILURE

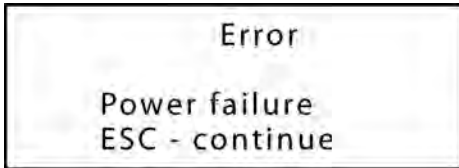AC line power was lost during a process.

- Push ESC to return to the main menu.


## CLOSE DOOR

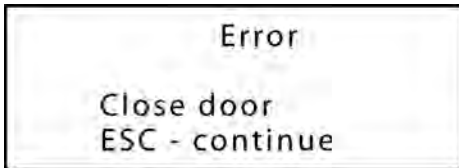START/STOP was pushed with the door open.

- Once the door is closed the display will return to the process menu screen.
- If the door is closed, the door switch may be malfunctioning,

## NO FREQUENCY

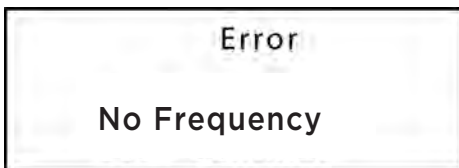AC line power fault occurred.

- Turn OFF the unit using the power switch and restart.

**NOTE: A power surge may cause any of the above errors to appear. Follow the instructions provided to clear the error and resume normal operations. If any error persists, call Dentsply Sirona service department for assistance.**

# PRODUCT SERVICE

**WARNING:**

This equipment is designed with safety features to protect the operator and must not be modified in any form. Maintenance and repair work should only be performed by an authorized service technician of Dentsply Sirona. Failure to observe these precautions may result in burns or electrical shock.

**USA**

- Telephone assistance at 800.835.6639, Option #1 (Equipment Repair and Parts)
- Email: YorkPA-Lab-EquipmentRepair@dentsplysirona.com

**BEFORE RETURNING THE UNIT:**

- Contact a Dentsply Sirona Service Technician to arrange repair at 800.835.6639, Option #1 (Equipment Repair and Parts). Obtain an RMA number, which will be used to track and verify your package. Please note, equipment received without this number may not be identifiable.
- Equipment damaged in shipment as a result of improper packing may not be paid by the carrier.
- Dentsply Sirona will not be responsible for damage resulting from improper packing.

**SHIP PREPAID TO:**

Dentsply Sirona  
ATTN: Equipment Repair  
RMA Number \_\_\_\_\_  
1301 Smile Way  
York, PA 17404 USA

**CANADA**

Contact Technical Support for service information:

- Telephone assistance at 800-263-1437, Option #3
- Email: Service.canada@dentsplysirona.com

**OUTSIDE OF NORTH AMERICA**

Contact your local Dentsply Sirona Representative.

# DISPOSAL OF UNIT

The unit is an electronic unit according to the “Act Governing the Sale, Return and Environmentally Sound Disposal of Electrical and Electronic Devices” (ElektroG).

Unit must not be disposed of in normal domestic waste. For environmental reasons, dispose of the product according to local environmental guideline or regulations.

The unit is not intended for private use. It is manufactured and delivered for commercial use and is to be disposed by the end user according to the specifications of the Electrical and Electronic Equipment Act - ElektroG.

# WARRANTY

**WARRANTY:** Dentsply Sirona warrants this product to be free from defects in material and workmanship for a period of two years from the date of sale. Dentsply Sirona's liability under this warranty is limited solely to repairing or, at Dentsply Sirona's option, replacing such products which are returned to Dentsply Sirona within the applicable warranty period (with shipping charges prepaid), and which are determined by Dentsply Sirona to be defective. This warranty shall not apply to any product which has been subject to misuse; negligence; or accident; or misapplied; or modified; or repaired by unauthorized persons; or improperly installed.

**INSPECTION:** Buyer shall inspect the product upon receipt. The buyer shall notify Dentsply Sirona in writing of any claims of defects in material and workmanship within thirty days after the buyer discovers or should have discovered the facts upon which such a claim is based. Failure of the buyer to give written notice of such a claim within this time period shall be deemed to be a waiver of such claim.

**DISCLAIMER:** The provisions of the above warranty are Dentsply Sirona's sole obligation and exclude all other remedies or warranties, expressed or implied, including those related to MERCHANTABILITY and FITNESS FOR A PARTICULAR PURPOSE.

**LIMITATION OF LIABILITY:** Under no circumstances shall Dentsply Sirona be liable to the buyer for any incidental, consequential or special damages, losses or expenses.

**LIMITATION OF ACTIONS:** The buyer must initiate any action with respect to claims under the warranty described in the first paragraph within one year after the cause of action has accrued.

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DeguDent GmbH