


Endo IQ App Release Version 3.2

Step-by-Step Instructions - Firmware Update

Step 1: Connect a device

If the FW (Firmware) is outdated, the App will provide the Alert

Step 2: Press “UPDATE FIRMWARE”

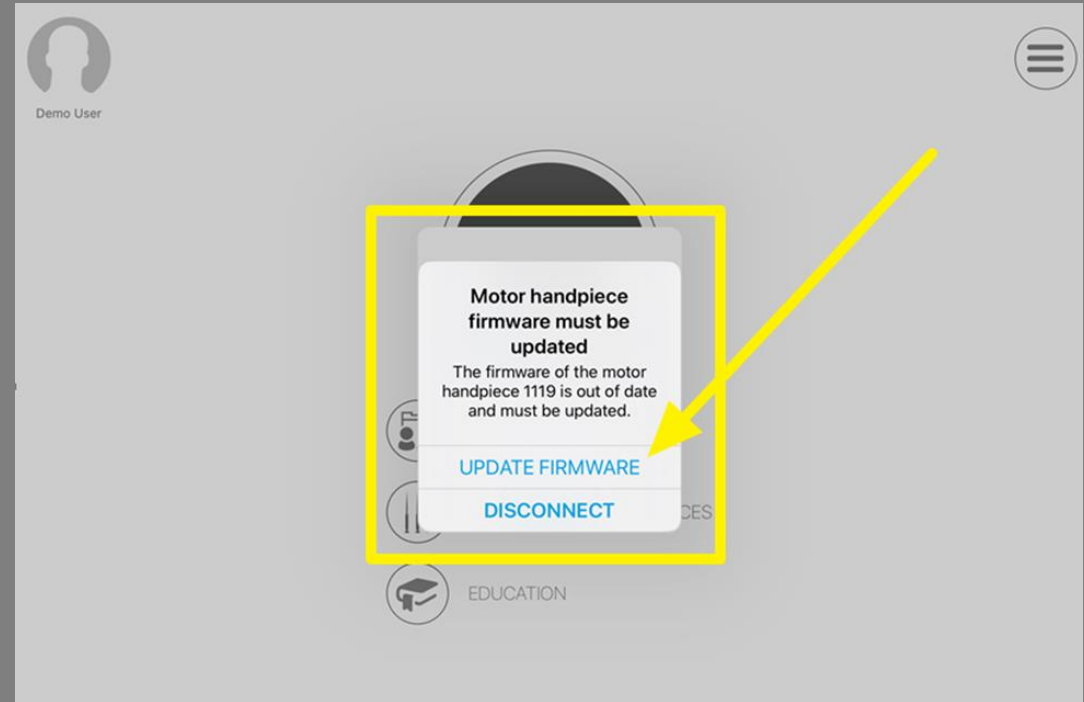

Step-by-Step Instructions - Firmware Update

Step 3: Connect a device to the charger

For Hand Piece: It should have 100% battery level OR be connected to the charger prior to starting FW update



In case the precondition is not satisfied, the App will guide you:

“Please connect the handpiece to the charger”

If this precondition is met, this step is not shown.


Step-by-Step Instructions - Firmware Update

Step 4: Press “Start” to initiate FW update



Step-by-Step Instructions - Firmware Update


Step 5: While FW update is ongoing you can check the steps passed following the list status update; if you need to interrupt the process for some reason, press “Cancel” :

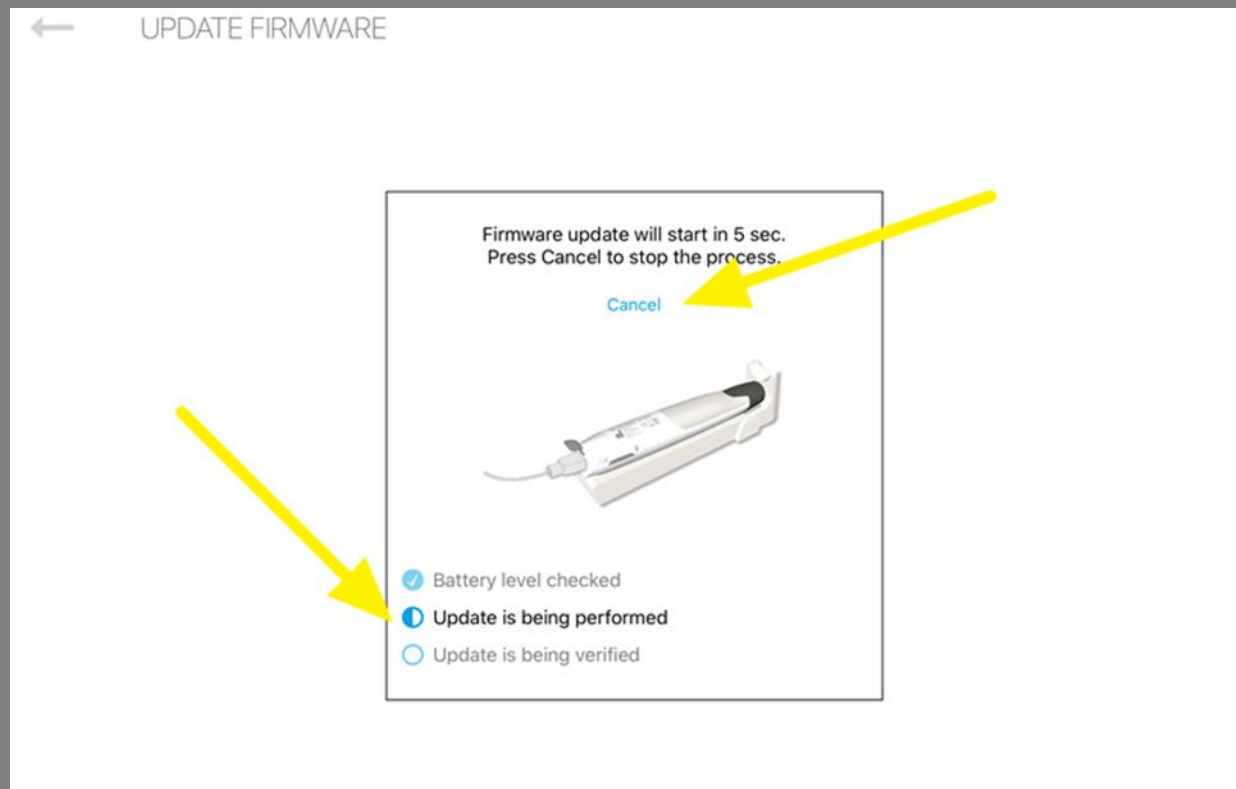

Step-by-Step Instructions - Firmware Update

Step 6: If FW update finished successfully, you will see that all steps are passed in the list and press “Done” to close the FW update screen:

Note: In case FW update failed, the last step will be marked as “failed”

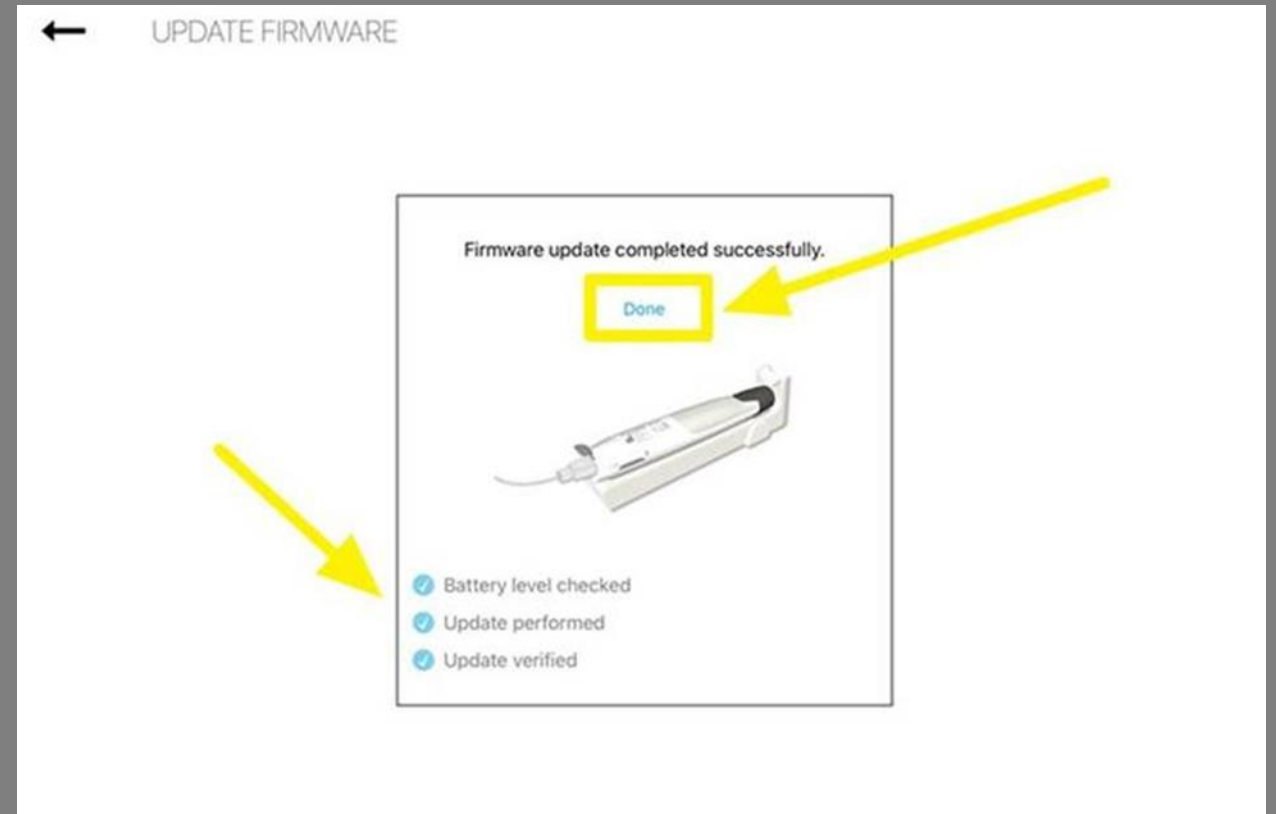


Step-by-Step instructions - Auto-Connect



6.7.5 Auto-connect settings

You can pair your devices so that, for the next time you want to connect them, they automatically get hooked-up to the Endo IQ® App when turning ON Bluetooth® on your devices. This eliminates the need of confirming the serial number every time you want to connect your devices.

N°	Action
A	Turn ON Bluetooth® on your respective devices and confirm the serial number once visible on the screen.

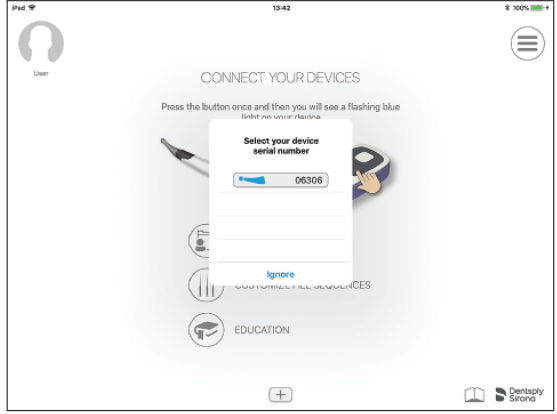

i The initial step, confirming the serial number, already paired your devices. When you turn them OFF and turn them ON again, after you turned ON Bluetooth®, your devices will be automatically connected. This is indicated by a yellow notification as shown on the next page.

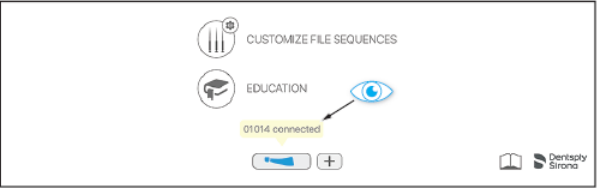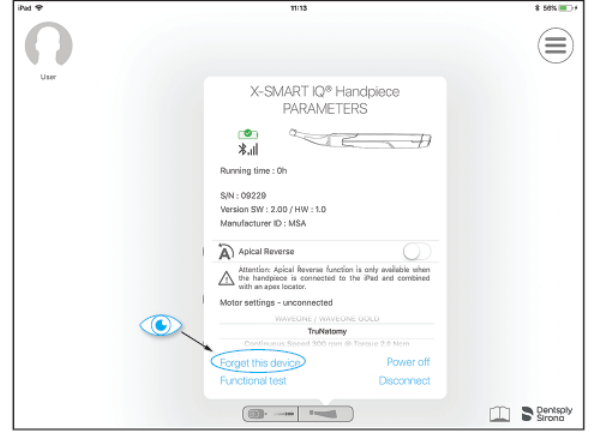N°	Action
	
B	<p>If you do not want your devices to connect automatically the next time, access the device parameters by clicking on the respective device icons on the status bar and click on "forget this device" (see below).</p> <p><i>Now, you have un-paired your device and it will not be automatically connected the next time you turn it ON.</i></p> 

Endo IQ App Version 3.2

For questions and concerns contact your local sales representative or technical support.



Thank You

