In-Ovation[®] R

Orthodontic Self-Ligating Brackets

DIRECTIONS FOR USE
INSTRUCCIONES DE USO
INSTRUÇÕES DE UTILIZAÇÃO
MODE D'EMPLOI
ISTRUZIONI PER L'USO
GEBRAUCHSANWEISUNG
GEBRUIKSAANWIJZING

Manufacturer:

DENTSPLY GAC International

One CA Plaza, Suite 100

Islandia, NY 11749 USA

Tel: 800-645-5530

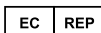
Fax: 631-357-8793

www.dentsplysirona.com/orthodontics

Single
Patient Only

R_x
ONLY

Emergo Europe
Prinsessegracht 20
2514 AP, The Hague
The Netherlands

CE
0120

In-Ovation® R

Orthodontic Self-Ligating Brackets

ENG

INTENDED USE:

Intended to be used in orthodontic treatment of natural teeth.

INDICATIONS FOR USE:

Indicated for patients requiring orthodontic treatment by movement of natural teeth.

CONTRAINDICATIONS:

In-Ovation® R brackets should not be bonded to restorative or compromised enamel surfaces.

In-Ovation® R brackets are contraindicated for patients with an allergy to Nickel.

⚠️ WARNINGS:

Single use only.

Device contains nickel and may cause allergic reactions.

PRECAUTIONS:

For professional use only. U.S. Federal law restricts this device to sale by or on the order of a Dentist.

Bonding of brackets to compromised teeth (i.e., with large restorations, to peg laterals, or pre-existing pathological conditions) can increase the risk of tooth damage.

Do not use product for anything other than orthodontic purposes.

Instruct patients not to chew or bite hard substances such as hard candy, ice or carrots.

If opening the clip via the "Opin R" feature, care should be taken to avoid overextension of the clip which may cause deformation.

Bonding to porcelain crowns or facings may cause chipping or breakage of the crown or facing during treatment or debonding.

For positioning, hold the bracket in the mesiodistal direction. If the bracket is held in the dental axis direction, the clip may close.

When bonding, take precautions with excess bonding material, or it may adhere to the clip, adversely affecting the opening mechanism.

For a severe case of crowding, initially use thin round wire.

Do not reuse the product. Risks of reuse include infection and cross-contamination to patient and healthcare provider.

If deformation or deterioration is recognized, do not use. Contact manufacturer for assistance.

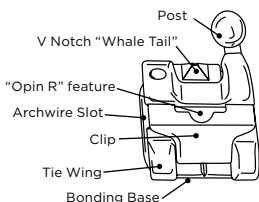
If an allergic reaction occurs, discontinue use immediately and direct the patient to consult a physician.

ADVERSE REACTIONS:

Allergic reaction.

STEP-BY-STEP INSTRUCTIONS:

1. Bond the bracket to each tooth using an orthodontic adhesive. Refer to orthodontic adhesive's Directions for Use.
2. At this time, the clip is in an "open" position. Avoid excess bonding material coming in contact with the clip as it could become solidified and interfere with proper clip functionality.
3. For positioning, align the mesiodistal wings of the bracket with the long axis of the tooth, and the archwire slot with the occlusal surface. Use the Vertical Scribe Line as a visual reference.
4. Insert the archwire into the slot, and push the clip lightly with a fingertip. When fixing the wire to a largely rotated tooth, initially use thin round wire (equivalent to Sentalloy® Blue) and seat the wire with a Ligature Tucking Instrument.
5. When changing the wire, open the clip by pushing down occlusally on the notch ("Whale Tail") of the clip in the center of the gingival tie wings.
6. *Alternatively in cases where the gingival notch is inaccessible due to gingival inflammation, you can also utilize the "Opin-R" notch on the facial surface of the clip applying gentle pressure occlusally with a mini scaler or similar instrument.
*The "Opin R" feature should not be used as the primary method for opening In-Ovation R brackets.
7. To close the clip; simply apply gingival pressure to the clip with a gloved finger.

SUGGESTED DE-BONDING PROCEDURE

For de-bonding, first fully scrape off the bonding material near the base; place a ligature cutter or bracket de-bonding instrument in the occlusal/gingival position and squeeze.

WARRANTY AND LIMITED REMEDY

Manufacturer warrants that this product is free from defects in materials and manufacture.

Manufacturer's sole obligation and customer's sole remedy in the event of a claimed defect shall be limited to replacement merchandise or refund of the purchase price.

All implied warranties, including implied warranties of merchantability or fitness for any particular purpose or use are disclaimed.

LIMITATION OF LIABILITY

Except where prohibited by law, Manufacturer will not be liable for any loss or damages arising from this product, whether direct, indirect, special, incidental, or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability. This limitation does not apply to third party personal injury claims.

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