In-Ovation[®] mini

Orthodontic Self-Ligating Brackets

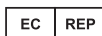
DIRECTIONS FOR USE
INSTRUCCIONES DE USO
INSTRUÇÕES DE UTILIZAÇÃO
MODE D'EMPLOI
ISTRUZIONI PER L'USO
GEBRAUCHSANWEISUNG
GEBRUIKSAANWIJZING

Manufacturer:
DENTSPLY GAC International
One CA Plaza, Suite 100
Islandia, NY 11749 USA
Tel: 800-645-5530
Fax: 631-357-8793
www.dentsplysirona.com/orthodontics

**SINGLE
USE ONLY**

Emergo Europe
Prinsessegracht 20
2514 AP, The Hague
The Netherlands

CE
0120

R_x
ONLY

Made in Japan
#DFU-0103 Rev. 04
Issued 05/17

In-Ovation® mini Orthodontic Self-Ligating Brackets

ENG

INTENDED USE:

Intended to be used in orthodontic treatment of natural teeth.

INDICATIONS FOR USE:

Indicated for patients requiring orthodontic treatment by movement of natural teeth.

CONTRAINDICATIONS:

In-Ovation mini brackets should not be bonded to restorative or compromised enamel surfaces.

In-Ovation mini brackets are contraindicated for patients with an allergy to Nickel.

⚠️ WARNINGS:

Single use only.

Device contains nickel and may cause allergic reactions.

PRECAUTIONS:

For professional use only. U.S. Federal law restricts this device to sale by or on the order of a Dentist.

Bonding of brackets to compromised teeth (i.e., with large restorations, to peg laterals, or pre-existing pathological conditions) can increase the risk of tooth damage.

Do not use product for anything other than orthodontic purposes.

Instruct patients not to chew or bite hard substances such as hard candy, ice or carrots.

If opening the clip via the "Opin R" feature, care should be taken to avoid overextension of the clip which may cause deformation.

Bonding to porcelain crowns or facings may cause chipping or breakage of the crown or facing during treatment or debonding.

For positioning, hold the bracket in the mesiodistal direction. If the bracket is held in the dental axis direction, the clip may close.

When bonding, take precautions with excess bonding material, or it may adhere to the clip, adversely affecting the opening mechanism.

For a severe case of crowding, initially use thin round wire.

Do not reuse the product. Risks of reuse include infection and cross-contamination to patient and healthcare provider.

If deformation or deterioration is recognized, do not use. Contact manufacturer for assistance.

If an allergic reaction occurs, discontinue use immediately and direct the patient to consult a physician.

ADVERSE REACTIONS:

Allergic reaction.

STEP-BY-STEP INSTRUCTIONS:

1. Bond the bracket to each tooth using an orthodontic adhesive. Refer to orthodontic adhesive's Directions for Use.

At this time, the clip is in an "open" position. Avoid excess bonding material coming in contact with the clip as it could become solidified and interfere with proper clip functionality.

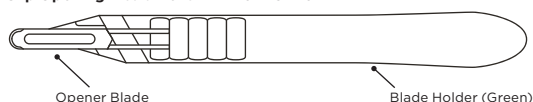
2. For positioning, align the mesiodistal wings of the bracket with the long axis of the tooth, and the archwire slot with the occlusal surface. Use the Vertical Scribe Line as a visual reference.

3. Insert the archwire into the slot, and push the clip lightly with a fingertip. Alternatively, position blade of opener at the occlusal base and swivel instrument gingivally to close.

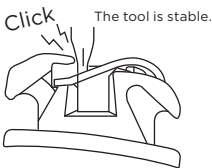
When fixing the wire to a largely rotated tooth, initially use thin round wire (equivalent to Sentalloy® Blue) and seat the wire with a Ligature Tucking Instrument.

4. When changing the wire, open the clip by pushing down occlusally on the clip projection ("ledge") of the clip in the center of the gingival tie wings.

Clip Opening Instrument REF# 07-191-10

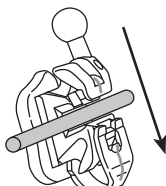
5. Place the round blade tip of opening tool onto the top surface of the clip parallel to arch wire slot, until it gets over a small projection on the clip and clicks into the center of the clip.

Position to begin opening the clip

6. To close the clip; simply apply gingival pressure to the clip with a gloved finger.

SUGGESTED DE-BONDING PROCEDURE

For de-bonding, first fully scrape off the bonding material near the base; place a ligature cutter or bracket de-bonding instrument in the occlusal/gingival position and squeeze.

CLIP CLOSING

- The In-Ovation Mini clip closes with pivoting-slide motion.
- Therefore, pushing the clip straight up may cause clip deformation.
- When closing the clip, lift the clip up slightly to labial direction and then, push the clip with fingertip.
- Alternatively, position blade of opener at the occlusal base and swivel instrument gingivally to close

WARRANTY AND LIMITED REMEDY

Manufacturer warrants that this product is free from defects in materials and manufacture.

Manufacturer's sole obligation and customer's sole remedy in the event of a claimed defect shall be limited to replacement merchandise or refund of the purchase price.

All implied warranties, including implied warranties of merchantability or fitness for any particular purpose or use are disclaimed.

LIMITATION OF LIABILITY

Except where prohibited by law, Manufacturer will not be liable for any loss or damages arising from this product, whether direct, indirect, special, incidental, or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability. This limitation does not apply to third party personal injury claims.

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