# Ortho-Loc

## Bracket Bonding System

DIRECTIONS FOR USE  
INSTRUCCIONES DE USO  
INSTRUÇÕES DE UTILIZAÇÃO  
MODE D'EMPLOI  
ISTRUZIONI PER L'USO  
GEBRAUCHSANWEISUNG  
GEBRUIKSAANWIJZING

Manufacturer:  
**DENTSPLY GAC International**

One CA Plaza, Suite 100  
Islandia, NY 11749 USA

Tel: 800-645-5530

Fax: 631-357-8793

**EC** **REP**

Emergo Europe  
Prinsessegracht 20  
2514 AP, The Hague  
The Netherlands

**CE**  
**0120**

Temperature  
Limitation

15° C (59° F) 30° C (86° F)

Keep away  
from sunlight

**R<sub>x</sub>**  
**ONLY**

Made in USA

#DFU-0110 Rev. 02 Issued 06/17

Ortho-Loc, no mix orthodontic bonding system is recommended for use with metal and plastic brackets. The system incorporates a conditioner, primer, and paste. Since an excess of any of these three ingredients will cause bracket flotation, care should be taken in following the measurements prescribed in the product instructions.

**INTENDED USE:**

Tooth Etch is used to surface condition enamel (and dentin where total etch technique is desired) prior to restoration with adhesive restorative materials.

**INDICATIONS FOR USE:**

Indicated for patients who will need metal and plastic orthodontic brackets to be bonded.

**CONTRAINDICATIONS:**

Tooth Etch is contraindicated for the use on dental pulp tissue.

**⚠️ WARNINGS:**

1. Tooth Etch contains phosphoric acid, which may cause burns. Avoid contact with oral tissues, eyes and skin. Do not take internally.
2. **Eye Contact:** Tooth Etch contains phosphoric acid, which may cause burns or may be irritating to eyes. Before using this product wear protecting glasses as well as covering the patient's eyes to protect from squirting the material inadvertently. In case of accidental contact with eyes, rinse eyes with plenty of water and seek medical attention.
3. **Skin Contact:** Tooth Etch contains phosphoric acid, which may cause burns or skin sensitization in susceptible individuals. If contact with skin occurs immediately wipe off and flush with generous amounts of water, then wash well with soap and water after contact. If skin rash, irritation, sensitization or other allergic reaction occurs, seek medical attention immediately.
4. **Oral Mucosa Contact:** Avoid contact with oral soft tissue. If accidental contact occurs, flush with plenty of water and expectorate water. If sensitization of mucosa persists, seek medical attention immediately.
5. **Ingestion:** Do not swallow or take internally. If accidental swallowing occurs, drink lots of water. If nausea or illness develops, seek medical attention immediately. Contact regional Poison Control Center immediately.
6. Tooth Etch should not be used with patients who have a history of severe allergic reaction to any of the components.

**PRECAUTIONS:**

1. This product is intended to be used only as specifically outlined in the *Directions for Use*. Any use of this product inconsistent with the *Directions for Use* is at the discretion and sole responsibility of the practitioner.
2. Wear suitable protective eyewear, clothing and gloves. Protective eyewear is recommended for patients.
3. Replace original cap of Tooth Etch bottles tightly after each use to avoid evaporation. Discard dispensing tips or compules after use, as dispensing tips may clog if etch is allowed to dry inside.
4. Manufacturer supplies the appropriate dispensing syringe tip for the Tooth Etch. This tip is the only tip that should be utilized for placement of gel.
5. For use on full coverage vital crown preparations see *Warnings* and *Step-by-Step instructions*.
6. The efficacy of Tooth Etch as an etchant/cleanser of indirect restorations has not been demonstrated.
7. Tooth Etch in syringes should extrude easily: **DO NOT USE EXCESSIVE FORCE**. Excessive pressure may result in unanticipated extrusion of the material.
8. For full coverage vital crown preparations, etch remaining enamel only. Etching of full coverage dentin surfaces is not recommended to minimize the possibility of postoperative sensitivity.

**ADVERSE REACTIONS:**

1. Product may irritate the eyes and skin. Eye Contact: Irritation and possible cornea damage. Skin Contact: Irritation or possible allergic response. Reddish rashes may be seen on the skin. Mucosa Membranes: Inflammation, edema, sloughing. See *Warnings*.
2. Product may cause serious health effects if ingested. See *Warnings*.
3. Product may cause pulpal effects. See *Contraindications* and *Warnings*.
4. Inhalation of vapors may cause varying degrees of damage to the affected tissue and also increased susceptibility to respiratory illness.
5. The following medical conditions are generally aggravated by exposure: Individuals with pre-existing skin disorders, eye problems or impaired liver, kidney and respiratory functions.

**STEP-BY-STEP INSTRUCTIONS:**

Bonding effectiveness will be enhanced if these step-by-step instructions are followed:

**Tooth Preparation:**

1. **Prophy** – Using a plain pumice, completely prophy the enamel surfaces of the teeth selected for bonding. Do not use any prophy pastes containing oil or fluoride. Rinse the teeth thoroughly and dry them with oil-free, no moisture compressed air.
2. **Etching tooth Surface**
  - a. Apply Tooth Etch to clean enamel. Allow etch to remain in place for 15 seconds. Where total etch technique is desired, begin with application of etchant to enamel first, then flowing onto dentin. Allow etchant to remain in place for a total of 15 seconds for initial enamel contact.
  - b. Rinse etched areas thoroughly with water (approx. 10-15 seconds).
  - c. If only enamel has been etched, air dry with clean, oil-water-free compressed air for 15 seconds. Properly etched enamel will appear dull, frosty white. NOTE: Protect exposed dentin from air to prevent desiccation.
  - d. If total etch technique has been employed, blot dry etched areas with a moist cotton pellet. Do not rub tooth surface while drying. Properly etched dentin should appear glistening, without pooling of moisture.
  - e. Proceed immediately to bonding technique desired.

Properly etched surfaces must keep free from contamination by oral fluids. If salivary contamination occurs, thoroughly clean with forceful water spray (free of oil contamination), blot dry and repeat etching procedure on enamel only for 10 seconds, rinse as in step 2 above and dry as in step 3 above.

3. **Rinse & Dry** – Rinse the teeth thoroughly with water. Do not allow the patient to rinse or contaminate enamel surfaces. Re-isolate the teeth and dry them completely. The conditioned areas should appear frosty white. If they are not, re-etch for fifteen seconds.

**Bonding Procedure:**

Apply a thin coat of primer to the surface of each etched, dry tooth to which the bracket is to be bonded, and to the underside of each bracket base. (Note: Bonding problems arise most often because too much primer has been used. Less is best.)

**Dispensing from Syringe**

Apply a small quantity of bonding paste from the Ortho-Loc syringe to the bracket base.

**Dispensing from Compules**

1. Remove tip of compule with cutter.
2. Insert compule in compule gun and apply gentle pressure.

**Warning:** Only use compule gun designed specifically for Ortho-Loc compules. If compules do not fit properly, the adhesive could burst from compules.

3. Apply adhesive to bracket base

Within 25 seconds, position the bracket with the desired position, and press the bracket firmly onto the tooth surface. Remove excess adhesive after positioning each bracket.

After five minutes, an active arch wire can be inserted.

**Disinfections of Syringe and/or Bottle**

The syringe and/or bottle if exposed to spatter or spray of body fluids, contaminated hands, or oral tissue, should be disinfected with hospital-level disinfectant. Acceptable disinfectants are EPA-registered as tuberculocidal. Iodophors, sodium hypochlorite (5.25%), chlorine dioxide and dual or synergized quaternaries are approved disinfectants. Disinfect the syringe and/or bottle by spraying the syringe and/or bottle with any recommended hospital level disinfectant except neutral glutaraldehyde for the contact time recommended by the disinfectant manufacturer for optimum results. Some phenolic-based agents and iodophor-based products may cause surface staining. Agents containing organic solvents, such as alcohol, may tend to dissolve the plastic. The disinfectant manufacturer's directions should be followed properly for optimum results. Water-based disinfectant solutions are preferred.

**Storage and Shelf Life:**

1. Store syringe with original cap only. Do not store syringe with dispensing tip in place, as this may harden the material making extrusion difficult (see *Warnings*). Temperature Limitation: 15°C/59°F - 30°C/86°F. Do not freeze. Do not use after expiration date.
2. Use only in well ventilated area.
3. Keep away from direct sunlight.

**Reorder Information:**

Ortho-Loc Bracket Bonding Kit (7 gr. syringe)	KIT52-500-00
Ortho-Loc Kit with Fluoride	KIT52-599-00
Ortho-Loc Bracket Bonding Kit (14 gr. jar)	KIT52-570-00
Ortho-Loc Bracket Bonding Kit (15 compules)	KIT52-590-00
Ortho-Loc Syringes, 7 gr.	52-507-00
Ortho-Loc Primer, 16cc	52-508-00
Ortho-Loc Primer with Fluoride, 16cc	52-508-10
Ortho-Loc Paste, 14 gr.	52-510-00
Ortho-Loc No-Mix Compules (pk of 15)	52-580-00
Ortho-Loc Tooth Conditioner, 9 gr.	51-000-09
Mixing Stick	51-000-05
Mixing Pad	51-000-07
Micro Brush for Etch/Seal, silver (pk of 100)	17-000-13

*Note: Product may not be available for sale in your country. Please contact your local representative for more information.*

**Lot Number and Expiration Date**

1. Do not use after expiration date. ISO standard uses: "YYYY/MM"
2. The following numbers should be quoted in all correspondences.

-Reorder number

-Lot number on the syringe, bottle or compule

-Expiration date

**WARRANTY AND LIMITED REMEDY**

Manufacturer warrants that this product is free from defects in materials and manufacture.

Manufacturer's sole obligation and customer's sole remedy in the event of a purchased defect shall be limited to replacement merchandise or refund of the purchase price.

All implied warranties, including implied warranties of merchantability or fitness for any particular purpose or use are disclaimed.

**LIMITATION OF LIABILITY**

Except where prohibited by law, Manufacturer will not be liable for any loss or damages arising from this product, whether direct, indirect, special, incidental, or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability. This limitation does not apply to third party personal injury claims.

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