# SpaceFile® Interproximal Reduction Files

DIRECTIONS FOR USE  
INSTRUCCIONES DE USO  
INSTRUÇÕES DE UTILIZAÇÃO  
MODE D'EMPLOI  
ISTRUZIONI PER L'USO  
GEBRAUCHSANWEISUNG  
GEBRUIKSAANWIJZING

 **Manufacturer:**  
**Dentsply International Raintree Essix**  
7290 26th Court East  
Sarasota, FL 34243 USA  
Tel: 800.883.8733  
www.essix.com

**Distributor:**  
**Dentsply GAC International**  
One CA Plaza, Suite 100  
Islandia, NY 11749 USA  
Tel: 800-645-5530  
Fax: 631-357-8793  
www.dentsplygac.com


**R<sub>x</sub>**  
ONLY

**EC REP**

**CE**  
0120

Emergo Europe  
Prinsessegracht 20  
2514 AP, The Hague  
The Netherlands

©Dentsply Sirona #DFU-0114 RTE-039-17 Rev.4 3/17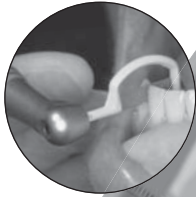

Use three ways:  
hand-held, with  
removable grip, or in  
optional air-driven  
contra-angle  
handpiece.

Contra-angle handpiece  
sold separately.

# SpaceFile® Interproximal Reduction Files

ENG

Caution: U.S. Federal law restricts this device to sale by or on the order of a dentist.

**Prior to use with air-driven handpiece, please read the “Directions For Use” that was included with the Interproximal Reduction System (IRS) handpiece, sold separately.** Do not apply excessive pressure on strips, which may lead to premature fracture of the plastic handle.

## INTENDED USE/INDICATIONS FOR USE:

Intended for use as a method for achieving interproximal reduction of teeth, including joined enamel, composite, amalgam, porcelain or gold.

## CONTRAINDICATIONS:

The IRS is contraindicated for use with patients if it has not been sterilized as recommended in the Directions For Use.

The IRS is contraindicated for use in patients who:

- Are hypersensitive,
- Don't accept stripping,
- Have low enamel density (hypoplasia) or a high caries index,
- Display poor hygiene,
- Have rectangular-shaped teeth, and/or,
- Display a high degree of crowding.

## ⚠ WARNINGS:

Files are made from stainless steel and contain 6-8% of nickel by weight. Nickel may cause allergic reactions in some patients.

## PRECAUTIONS:

Read the Directions for Use thoroughly and comply with them during use. Any use of this product inconsistent with these directions for use, is at the discretion and sole responsibility of the practitioner.

Perform a visual inspection of the respective SpaceFile® file. Do not use files that are damaged, bent, or broken.

Prior to use on a patient, observe the recommended sterilization instructions below.

Use care when installing File in Grip as Files may be sharp.

Grip is provided for manual use only and must not be used with the air-driven handpiece.

Lifetime of the files is dependent on physical usage; no expiry date is provided. It is recommended that files are only used a maximum of 20 times. If wear or damage becomes obvious, files should be discarded. Used files should be discarded safely according to standard biohazard waste procedures.

**ADVERSE REACTIONS:** Allergic Reactions


## STEP BY STEP INSTRUCTIONS:

SpaceFile® files can be used with the specified IRS handpiece or manually with or without the included Grip. If using the handpiece, refer to its Directions for Use.


Use the double-sided or single-sided File of appropriate grit in sequence as needed to create the desired space. A serrated File is also available to assist in breaking initial contact.

### To insert SpaceFile® File in Grip:


1. Place the shank of the SpaceFile® File into the hole on the Grip until resistance is felt. The opposite side of the Grip can be bent out of the way to facilitate this.
2. With the base of the SpaceFile® File resting on a firm surface, press down on the middle of the Grip until positive engagement is felt.
3. Push the end of the Grip without the hole towards the File to complete engagement of the File within the Grip.

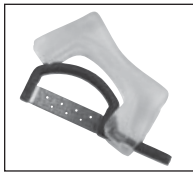**Figure 1:** Shank of File fully inserted into Grip hole

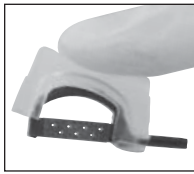**Figure 2:** Press down on mid-section of Grip until engagement is felt

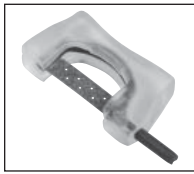**Figure 3:** File fully inserted in Grip

### To Remove SpaceFile® File from Grip:

1. Grasping the Grip at the end with the shank, pull the opposite end of the Grip upwards and away from the File.
2. Pull the File straight out of the hole in the Grip.

## CLEANING GUIDELINES:

### Scope

The SpaceFile® File and Grip should be both cleaned and then sterilized after each use. Refer to the Sterilization Guidelines for sterilization instructions. Cleaning should be completed immediately after use to avoid contaminants drying, thereby making cleaning more difficult.

### Manual Cleaning Procedure

1. Rinse under running cold water to remove gross contamination.
2. Keeping it immersed, brush thoroughly away from the body using an enzymatic cleaning/disinfecting agent validated for use on dental instruments. Follow the agent manufacturer's instructions.
3. Special care should be taken to clean crevices, holes and other hard-to-reach areas thoroughly.
4. Perform final thorough rinse and visual inspection to confirm the removal of debris. Repeat the cycle if needed.
5. Dry using a lint-free wipe or clean compressed air.
6. Inspect for damage or wear that could prevent proper function.

## STERILIZATION GUIDELINES:

### Scope

The SpaceFile® Interproximal Reduction Kit should be sterilized before and after each use.

### Packaging

1. Remove the inner tray from the outer case by pulling up on the tab. DO NOT sterilize the outer case.
2. Pack the Grip and/or SpaceFile® Files in the insert.
3. Insert in pouches validated for sterilization.

### Steam Sterilization Procedure

| Cycle Type | Minimum Sterilization Exposure Time | Minimum Sterilization Exposure Temperature | Minimum Dry Time |
|------------|-------------------------------------|--|------------------|
| Gravity    | 10 minutes                          | 134° (+3) C or 274° (+5.4) F               | 15 minutes       |

### Storage

The device should be stored in the sterilization pouch until required.

### Additional Information

These cleaning and sterilization processes have been validated as being capable of preparing the SpaceFile® Interproximal Reduction Kit for use. Any deviation from these instructions should be properly validated for effectiveness and potential adverse results.

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