FR INSTRUMENTS D'ORTHODONTIE

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EN ORTHODONTIC INSTRUMENT

INDICATIONS FOR USE

Orthodontic instruments are strictly intended for orthodontic treatment. Treatment includes the positioning and placing of brackets, wires, and associated appliances as well as any additional procedures defined by the appropriate clinician.

CONTRAINDICATIONS

None known

⚠ WARNINGS

- Not properly maintaining the instruments can lead to instrument failure or lack of precision cutting, bending, and/or movement.
- Not properly cleaning and sterilizing the instruments can lead to contamination and corrosion.
- For ligature wire cutters:
 - Do not cut ligature wires that have been braided
 - Do not cut wires exceeding the maximum cutting capacity of the instrument
 - Do not cut arch wire
 - Use for ligature wire only

PRECAUTIONS

- Orthodontic instruments are not intended for use outside of orthodontics and must be used by licensed orthodontists.
- Failure to clean the instrument before sterilization can compromise the disinfection or sterilization process.
- Always keep cutting pliers sharp.

ADVERSE REACTIONS

Orthodontic instruments contain nickel, which may cause allergic reactions.

STEP-BY-STEP INSTRUCTIONS

Cutting pliers display the maximum cutting capacity etched on the handle. To ensure warranty claims, cutting instruments should be sent for proper service and maintenance every six to ten months. Service consists of re-sharpening, assembly adjustment, and detailed cleaning. Maintenance will be free of charge for customers within their warranty period; shipping cost may apply. See Warranty and Limited Remedy for details.

FOR PROPER CLEANING:

Please ensure your equipment is validated to clean to these methods.

Before sterilization or high-level disinfection, instruments should be cleaned so that any debris is removed. Heavy duty gloves should be worn when handling contaminated instruments. Debris can be removed from an instrument either by scrubbing the instrument manually with a detergent, a disinfectant, or an enzymatic cleaner, or by using automated equipment (ultrasonic cleaner, washer-disinfecter) and appropriate agents. Automated equipment is most efficient due to increased productivity, effectiveness, and lowered exposure to body fluids.

Ultrasonic cleaning is not recommended for cutting instruments; solutions combined with vibrations are very caustic and could dull or damage cutter. Any torque or angling of the instrument is not recommended. Ultrasonic cleaning is acceptable for non-cutting instruments such as pliers, bender, and hand instruments.

DRY HEAT

Place on tray, sheet, or rack with jaws apart (open). Sterilize for 1 to 2 hours at 160°C, according to equipment manufacturer's instruction.

Note: Instruments may be bagged prior to sterilization.

AUTOCLAVE*

Place instrument on autoclave tray with jaws apart (open). Process for 6 to 12-minute cycle at 134°C, according to equipment manufacturer's instruction. BE CERTAIN UNIT USES DISTILLED WATER.

Note: Instruments may be bagged prior to sterilization.

MAINTENANCE

Failure to keep instrument dry may result in spotting. If you are experiencing brown or yellow staining, scrub with ISOPROPYL alcohol and steel wool (Type A-extra fine).

MAINTENANCE FOR HINGED INSTRUMENTS

Lubricate the joints and hinges of instruments after sterilizing to prevent seizing and corrosion, using NuJoint Lubricant (Part #97-977-15). Wipe off any excess lubricant. Do not overuse lubricants as this may stain or discolor the instrument.

WARRANTY AND LIMITED REMEDY

DENTSPLY GAC warrants that this product is free from defects in materials and manufacture.

DENTSPLY GAC's sole obligation and customer's sole remedy in the event of a claimed defect shall be limited to replacement of merchandise or refund of the purchase price.

ALL IMPLIED WARRANTIES, INCLUDING IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR ANY PARTICULAR PURPOSE OR USE ARE DISCLAIMED.

All instruments are guaranteed to be free from any manufacturing defects for the 5 year expected life of the instrument. Defects will be replaced at no charge. Cutting pliers carry an all-inclusive 1-year warranty that includes sharpening. To return a DENTSPLY GAC instrument sharpening or repair, the following procedures are to be followed:

- Instrument is to be sterilized and sealed in a zip-lock bag.
- An explanation of why the instrument is being returned must accompany each instrument.
- Doctor's name, address, phone number, Dentsply GAC account number, and email (if applicable) must accompany each instrument.
- Mail to OrthoDental International;
 - 280 Avenida Campillo, Suite J, Calexico, California 92231
 - Phone: 760-357-8070 Fax: 760-357-2910
- If the above procedures are not followed, the instrument (s) will be subject to return to doctor without the requested repair or sharpening being completed.

LIMITATION OF LIABILITY

Except where prohibited by law, DENTSPLY GAC will not be liable for any loss or damages arising from this product, whether direct, indirect, special, incidental, or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability. This limitation does not apply to third party personal injury claims.

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Manufacturer:
DENTSPLY GAC INTERNATIONAL
One CA Plaza, Suite 100 Islandia, NY 11749 USA
Phone: 800-645-5530 Fax: 631-357-8793
www.dentsplysirona.com/orthodontics

Rx
ONLY

Consult
Instructions
for Use

Made in Mexico

DFU-0121-A Rev. 02
Issue Date 05-17

Z-fold