Black

MAGENTA does not print

ROLL FOLD

245*

BACK

FRONT

Dentsply
Sirona

1 Per Package
1 Per confezione
1 Por paquete
1 Par paquet
1 Pro packing
1 per verpakking

ORTHODONTIC INSTRUMENT

ORTHODONTISCH INSTRUMENT
INSTRUMENTS D'ORTHODONTIE
INSTRUMENTE FÜR ORTHODONTIE
INSTRUMENTOS ORTODONTICOS
STRUMENTI ORTODONTICI

Made in Mexico

DFU-0121 Rev. 02
Issue Date 05-17

Emergo Europe Prinsessegracht 20
2514 AP The Hague, The Netherlands

IT STRUMENTI ORTODONTICI

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EN ORTHODONTIC INSTRUMENT

INDICATIONS FOR USE

Orthodontic instruments are strictly intended for orthodontic treatment. Treatment includes the positioning and placing of brackets, wires, and associated appliances as well as any additional procedures defined by the appropriate clinician.

CONTRAINDICATIONS

None known

WARNINGS

- Not properly maintaining the instruments can lead to instrument failure or lack of precision cutting, bending, and/or movement.
- Not properly cleaning and sterilizing the instruments can lead to contamination and corrosion.
- For ligature wire cutters:
 - Do not cut ligature wires that have been braided
 - Do not cut wires exceeding the maximum cutting capacity of the instrument
 - Do not cut arch wire
 - Use for ligature wire only

PRECAUTIONS

- Orthodontic instruments are not intended for use outside of orthodontics and must be used by licensed orthodontists.
- Failure to clean the instrument before sterilization can compromise the disinfection or sterilization process.
- Always keep cutting pliers sharp.

ADVERSE REACTIONS

Orthodontic instruments contain nickel, which may cause allergic reactions.

STEP-BY-STEP INSTRUCTIONS

Cutting pliers should be maximum cutting capacity etched on the handle. To ensure warranty claims, cutting instruments should be set for proper service and maintenance every six to ten months. Service consists of re-sharpening, assembly adjustment, and detailed cleaning. Maintenance will be free of charge for customers within their warranty period; shipping cost may apply. See Warranty and Limited Remedy for details.

FOR PROPER CLEANING:

Please ensure your equipment is validated to clean to these methods.

Before sterilization or high-level disinfection, instruments should be cleaned so that any debris is removed. Heavy duty gloves should be worn when handling contaminated instruments. Debris can be removed from an instrument either by scrubbing the instrument manually with a detergent, a disinfectant, or an enzymatic cleaner, or by using automated equipment (ultrasonic cleaner, washer-disinfector) and appropriate agents. Automated equipment is most effective to increase productivity, effectiveness, and lowered exposure to body fluids.

Ultrasonic cleaning is not recommended for cutting instruments; solutions combined with vibrations are very caustic and could dull or damage cutter. Any torque or angling of the instrument is not recommended. Ultrasonic cleaning is acceptable for non-cutting instruments such as pliers, bender, and hand instruments.

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