# Elation™ MB

## Orthodontic Brackets

DIRECTIONS FOR USE  
INSTRUCCIONES DE USO  
INSTRUÇÕES DE UTILIZAÇÃO  
MODE D'EMPLOI  
ISTRUZIONI PER L'USO  
GEBRAUCHSANWEISUNG  
GEBRUIKSAANWIJZING

Manufacturer:

**DENTSPLY GAC International**

One CA Plaza, Suite 100

Islandia, NY 11749 USA


Tel: 800-645-5530

Fax: 631-357-8793

[www.dentsplysirona.com/orthodontics](http://www.dentsplysirona.com/orthodontics)

Single  
Use Only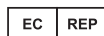

Emergo Europe  
Prinsessegracht 20  
2514 AP, The Hague  
The Netherlands

Made in Japan  
#DFU-0124 Rev. 01  
Issued 06/17

## Orthodontic Brackets

**INTENDED USE:**

Intended to be used in orthodontic treatment of natural teeth.

**INDICATIONS FOR USE:**

Indicated for patients requiring orthodontic treatment by movement of natural teeth.

**CONTRAINDICATIONS:**

Elation MB brackets should not be bonded to restorative or compromised enamel surfaces.

Elation MB brackets are contraindicated for patients with an allergy to Nickel.

**⚠ WARNINGS:**

Single use only.

Device contains nickel and may cause allergic reactions.

**PRECAUTIONS:**

For professional use only. U.S. Federal law restricts this device to sale by or on the order of a Dentist.

Bonding of brackets to compromised teeth (i.e., with large restorations, to peg laterals, or pre-existing pathological conditions) can increase the risk of tooth damage.

Do not use product for anything other than orthodontic purposes.

Instruct patients not to chew or bite hard substances such as hard candy, ice or carrots.

Bonding to porcelain crowns or facings may cause chipping or breakage of the crown or facing during treatment or debonding.

For positioning, hold the bracket in the mesiodistal direction. If the bracket is held in the dental axis direction, the clip may close.

When bonding, take precautions with excess bonding material, or it may adhere to the clip, adversely affecting the opening mechanism.

For a severe case of crowding, initially use thin round wire.

Do not reuse the product. Risks of reuse include infection and cross-contamination to patient and healthcare provider.

If deformation or deterioration is recognized, do not use. Contact manufacturer for assistance.

If an allergic reaction occurs, discontinue use immediately and direct the patient to consult a physician.

Elation MB brackets are capable of withstanding all normal torque requirements. However, care should be taken when making torquing activations. Large corrections with full size stainless steel wires may result in bracket failure and should be avoided.

**ADVERSE REACTIONS:**

Allergic reaction.

**ORTHODONTIC BRACKET IDENTIFICATION**

Elation MB brackets are easily identifiable by a color-coded dot on the disto-gingival tie wing:

UR & UL CENTRALS:	BLUE
UR & UL LATERALS:	PINK
UR & UL CUSPIDS	GREEN
UR & UL BICUSPIDS:	PURPLE
LOWER ANTERIORS:	ORANGE
LR & LL CUSPIDS:	LIGHT BLUE

**STEP-BY-STEP INSTRUCTIONS:****BRACKET BONDING**

- Bond the bracket to each tooth using an orthodontic adhesive. Refer to orthodontic adhesive's Directions for Use.
- When placing orthodontic Elation MB brackets, place the bracket using a sliding motion, occlusal to gingival, forcing excess adhesive to the incisal edge of the bracket for easier clean up. However, please note that excess sliding motion may increase the risk of bond failure because of the lack of adhesive under the bracket.

**BRACKET RE-BONDING**

If a spontaneous bond failure occurs, it may be necessary to rebond a bracket. The following steps are recommended:

- Carefully inspect the bracket for any damage.
- Remove contour cured adhesive on bracket base with a bur.
- Leave a thin layer of adhesive on the bracket base when removing any excess adhesive. An adhesive-to-adhesive bond is optimal when rebonding orthodontic Elation MB Brackets.
- If bracket has been contaminated (e.g. moisture), rinse the bracket in isopropyl alcohol and allow it to dry. It is not recommended to use Acetone, as this Acetone may cause plastics to lyse or branch.
- Prepare the tooth surface and bond the bracket using the procedure as described by the adhesive manufacturer.

**SUGGESTED DE-BONDING PROCEDURE**

For de-bonding, first fully scrape off the bonding material near the base; engage the bracket-tooth interface in the mesiodistal direction using a ligature cutter; and, finally, apply firm pressure.

**WARRANTY AND LIMITED REMEDY**

Manufacturer warrants that this product is free from defects in materials and manufacture.

Manufacturer's sole obligation and customer's sole remedy in the event of a claimed defect shall be limited to replacement merchandise or refund of the purchase price.

All implied warranties, including implied warranties of merchantability or fitness for any particular purpose or use are disclaimed.

**LIMITATION OF LIABILITY**

Except where prohibited by law, Manufacturer will not be liable for any loss or damages arising from this product, whether direct, indirect, special, incidental, or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability. This limitation does not apply to third party personal injury claims.

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