BioBiteCorrector® SP

DIRECTIONS FOR USE
INSTRUCCIONES DE USO
INSTRUÇÕES DE UTILIZAÇÃO
MODE D'EMPLOI
ISTRUZIONI PER L'USO
GEBRAUCHSANWEISUNG
GEBRUIKSAANWIJZING

Distributor:
DENTSPLY GAC International
One CA Plaza, Suite 100
Islandia, NY 11749 USA
T: 1-800-645-5530 In USA
F: 631-357-8793
www.dentsplysirona.com
www.bio-bite-corrector.com

Manufacturer:
Trinon Titanium GmbH
Augartenstr. 1
D-76137 Karlsruhe
GERMANY

Made in Germany
#DFU-0444 Rev. 03
Issued 09/17

The BioBiteCorrector SP (BBC) is a fixed Class II appliance working with a coil spring.

INTENDED USE:

Intended to be used in orthodontic treatment of natural teeth.

INDICATIONS FOR USE:

- Correction of Class II malocclusions and functional mandibular midline shifts
- Use as anchorage appliance for closing a gap in the posterior teeth of the lower jaw while simultaneously correcting a Class II malocclusion
- Avoidance of dysgnathic surgeries

CONTRAINDICATIONS:

- Severe protrusion of lower incisors

MATERIAL:

- The BioBiteCorrector is manufactured of titanium grade 5 and the spring inside of stainless steel.
- The xing screws and the distance spacers are manufactured of titanium
- The screwdriver is manufactured of stainless steel (sterilizable and re-usable)
- The BioBiteCorrector is determined for single use only.

THE FIXATION OF THE BBC:

Bands on the upper molars are not required to fix the BioBiteCorrector

1. PRECONDITIONS

- Precondition for the integration of the BioBiteCorrector is a multi-bracket appliance with a steel archwire with a minimum size of .017" x .025" in the upper and lower jaws.
- In the lower jaw the steel archwire must have a cinch back to avoid a proclination of the incisors. For the cinch back we recommend a Twister-Instrument. With the Twister you can bend the arch very simply without risking to remove the bracket from the tooth.

2. CHOICE OF SIZE

- BBC SP is delivered only in one size.

3. FIXATION OF THE BBC

- The BBC is already pre-assembled. The appliance is distinguished in the left and right sides. First the BBC is fixed in the upper jaw and afterwards in the lower jaw.

4. FIXATION IN THE UPPER JAW

- The upper jaw connecting element is fixed between the first molar and second premolar. The connecting element should have no contact with the bracket of the first molar.
The screwdriver should be assured with dental floss to avoid aspiration or swallowing.
- Slide the body on the archwire from lateral and then fix the screw.

5. FIXATION IN THE LOWER JAW

- After fixing the upper jaw connecting element, the lower jaw connecting element must be fixed between canine and first premolar. Slide the body on the archwire from lateral and then fix the screw. The connecting element should be situated as distal as possible mesially to the bracket of the first premolar. The connecting element must not have any contact with the canine bracket at all. Fix the other side in the same way.

6. ADJUSTMENT IN NEUTRAL OCCLUSION

- After both sides are xed, guide the lower jaw back as far as possible. If there is still distal occlusion the lower jaw can be shifted forward by using distance spacers. These spacers can be crimped on the appliance, whereby the lower jaw can be shifted into neutral occlusion.
- The distance spacers are attached to the thinnest telescope pipe. Fine corrections can be carried out by shifting the connecting element.

LIABILITY

- The product range description is included on the product website.
- There is no liability for unsatisfying results. The appliance must only be used by a dentist or an orthodontist. The dentist or orthodontist is responsible for the use himself.

SAFETY INSTRUCTIONS

- The firmness of the screws must be checked by the doctor in regular intervals to recognize loosened screws and to prevent a loss of the appliance with the danger of swallowing or aspirating the appliance or parts of it. The patient must be instructed for self-control.
- The average duration of treatment with the BioBiteCorrector is approximately 4-7 months.

CLEANING AND DISINFECTING

Suitable measures for cleaning, disinfection if necessary sterilization must be carried out with a validated method before using the product. Detailed instructions on processing are provide separately and can be requested.

Explanation of symbols used on the label

Instructions enclosed

Consult instructions for use

Manufacturer

European Conformity with identification number of the Notified Body

Only with medical prescription

SINGLE USE ONLY

Do not reuse

Batch code/number

Catalog number

DataMatrix code with catalog number and Batch number

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