37% Phosphoric Acid Tooth Etch

ENG

DIRECTIONS FOR USE-ENGLISH

INTENDED USE:

Tooth Etch is used to surface condition enamel (and dentin where total etch technique is desired) prior to restoration with adhesive restorative materials.

INDICATIONS FOR USE:

Indicated for patients who will need metal and plastic orthodontic brackets to be bonded.

COMPOSITION:

Water, 37% Phosphoric acid, Silicon Dioxide, Surfactants, colorant

CONTRAINDICATIONS:

Tooth Etch is contraindicated for the use on dental pulp tissue.

⚠️ WARNINGS:

1. Tooth Etch contains phosphoric acid, which may cause burns. Avoid contact with oral tissues, eyes and skin. Do not take internally.
2. **Eye Contact:** Tooth Etch contains phosphoric acid, which may cause burns or may be irritating to eyes. Before using this product wear protecting glasses as well as covering the patient's eyes to protect from squirting the material inadvertently. In case of accidental contact with eyes, rinse eyes with plenty of water and seek medical attention.
3. **Skin Contact:** Tooth Etch contains phosphoric acid, which may cause burns or skin sensitization in susceptible individuals. If contact with skin occurs immediately wipe off and flush with generous amounts of water, then wash well with soap and water after contact. If skin rash, irritation, sensitization or other allergic reaction occurs, seek medical attention immediately.
4. **Oral Mucosa Contact:** Avoid contact with oral soft tissue. If accidental contact occurs, flush with plenty of water and expectorate water. If sensitization of mucosa persists, seek medical attention immediately.
5. **Ingestion:** Do not swallow or take internally. If accidental swallowing occurs, drink lots of water. If nausea or illness develops, seek medical attention immediately. Contact regional Poison Control Center immediately.
6. Tooth Etch should not be used with patients who have a history of severe allergic reaction to any of the components.

PRECAUTIONS:

1. This product is intended to be used only as specifically outlined in the *Directions for Use*. Any use of this product inconsistent with the *Directions for Use* is at the discretion and sole responsibility of the practitioner.
2. Wear suitable protective eyewear, clothing and gloves. Protective eyewear is recommended for patients.
3. Replace original cap of Tooth Etch bottles tightly after each use to avoid evaporation. Discard dispensing tips or compules after use, as dispensing tips may clog if etch is allowed to dry inside.
4. Manufacturer supplies the appropriate dispensing syringe tip for the Tooth Etch. This tip is the only tip that should be utilized for placement of gel.
5. For use on full coverage vital crown preparations see *Warnings* and *Step-by-Step instructions*.
6. The efficacy of Tooth Etch as an etchant/cleanser of indirect restorations has not been demonstrated.
7. Tooth Etch in syringes should extrude easily: **DO NOT USE EXCESSIVE FORCE**. Excessive pressure may result in unanticipated extrusion of the material.
8. For full coverage vital crown preparations, etch remaining enamel only. Etching of full coverage dentin surfaces is not recommended to minimize the possibility of postoperative sensitivity.

ADVERSE REACTIONS:

1. Product may irritate the eyes and skin. Eye Contact: Irritation and possible cornea damage. Skin Contact: Irritation or possible allergic response. Reddish rashes may be seen on the skin. Mucosa Membranes: Inflammation, edema, sloughing. See *Warnings*.
2. Product may cause serious health effects if ingested. See *Warnings*.
3. Product may cause pulpal effects. See *Contraindications* and *Warnings*.
4. Inhalation of vapors may cause varying degrees of damage to the affected tissue and also increased susceptibility to respiratory illness.
5. The following medical conditions are generally aggravated by exposure: Individuals with pre-existing skin disorders, eye problems or impaired liver, kidney and respiratory functions.

STEP-BY-STEP INSTRUCTIONS:

I. Cleaning

Clean un-instrumented enamel with either a rubber cup and water/fluoride, pumice or cleaning paste. Wash thoroughly with clean, oil-free water spray.

II. Pulp Protection

In deep cavities, cover the dentin close to the pulp (less than 1 mm remaining) with hard setting calcium hydroxide liner leaving the rest of the cavity surface free for bonding.

III. Application

Dispensing From Syringe

1. Remove cap from the end of the syringe. To assure free flow of material from syringe, express a small amount onto a mixing pad.
2. Attach disposable dispensing tip to the end of the syringe. Turn tip clockwise until fully seated. Tug on tip to assure locking into syringe collar.
3. Dispensing tip may be bent for easy access. Gentle curving of the tip is preferable to an acute bend. Bending may occlude the tip. Be sure to check that this has not occurred by extruding a small amount through tip.
4. Gel should flow freely with GENTLE PRESSURE. **DO NOT USE EXCESSIVE FORCE**. If not, remove syringe from patient field and check for obstruction.
5. Apply gel through syringe dispensing tip to mixing pad. Using a clean, soft brush, apply etch to surfaces to be etched.

Dispensing From Bottle

1. Dispense desired amount onto a mixing pad or dampen dish. To prevent the gel from drying out, dispense gel just prior to placement procedure. Stirring will reduce viscosity.
2. Using a clean, soft brush, apply etch to surfaces to be etched.

Dispensing From Compules

1. Remove tip of compule with cutter.
2. Apply small amount of etchant onto a mixing pad. To prevent the gel from drying out, dispense gel just prior to placement procedure. Stirring will reduce viscosity.
3. Using a clean, soft brush, apply etch to surfaces to be etched.

IV. Etching Tooth Surface

1. Apply Tooth Etch to clean enamel. Allow etch to remain in place for 15 seconds. Where total etch technique is desired, begin with application of etchant to enamel first, then flowing onto dentin. Allow etchant to remain in place for a total of 15 seconds for initial enamel contact.
2. Rinse etched areas thoroughly with water (approx. 10-15 seconds).
3. If only enamel has been etched, air dry with clean, oil-water-free compressed air for 15 seconds. Properly etched enamel will appear dull, frosty white. NOTE: Protect exposed dentin from air to prevent desiccation.
4. If total etch technique has been employed, blot dry etched areas with a moist cotton pellet. Do not rub tooth surface while drying. Properly etched dentin should appear glistening, without pooling of moisture.
5. Proceed immediately to bonding technique desired.

Properly etched surfaces must keep free from contamination by oral fluids. If salivary contamination occurs, thoroughly clean with forceful water spray (free of oil contamination), blot dry and repeat etching procedure on enamel only for 10 seconds, rinse as in step 2 above and dry as in step 3 above.

Disinfections of Syringe and/or Bottle

The syringe and/or bottle if exposed to spatter or spray of body fluids, contaminated hands, or oral tissue, should be disinfected with hospital-level disinfectant. Acceptable disinfectants are EPA-registered as tuberculocidal. Iodophors, sodium hypochlorite (5.25%), chlorine dioxide and dual or synergistic quaternaries are approved disinfectants. Disinfect the syringe and/or bottle by spraying the syringe and/or bottle with any recommended hospital level disinfectant except neutral glutaraldehyde for the contact time recommended by the disinfectant manufacturer for optimum results. Some phenolic-based agents and iodophor-based products may cause surface dissolving. Agents containing iodophor solvents, such as alcohol, may tend to dissolve the plastic. The disinfectant manufacturer's directions should be followed properly for optimum results. Water-based disinfectant solutions are preferred.

Storage and Shelf Life:

1. Store syringe with original cap only. Do not store syringe with dispensing tip in place, as this may harden the material making extrusion difficult (see *Warnings*). Temperature Limitation: 15°C/59°F - 30°C/83°F. Do not freeze. Do not use after expiration date.
2. Use only in well ventilated area.
3. Keep away from direct sunlight.

Lot Number and Expiration Date

1. Do not use after expiration date. ISO standard uses: "YYYY/MM"
2. The following numbers should be quoted in all correspondences.

-Reorder number

-Lot number on the syringe, bottle or compule

-Expiration date

WARRANTY AND LIMITED REMEDY

Manufacturer warrants that this product is free from defects in materials and manufacture.

Manufacturer's sole obligation and customer's sole remedy in the event of a claimed defect shall be limited to replacement merchandise or refund of the purchase price.

All implied warranties, including implied warranties of merchantability or fitness for any particular purpose or use are disclaimed.

LIMITATION OF LIABILITY

Except where prohibited by law, Manufacturer will not be liable for any loss or damages arising from this product, whether direct, indirect, special, incidental, or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability. This limitation does not apply to third party personal injury claims.

Manufacturer:
DENTSPLY GAC International

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