NeoBond®

Bracket Adhesive

NeoBand®

Light Cure Band Cement

DIRECTIONS FOR USE
INSTRUCCIONES DE USO
MODE D'EMPLOI
ISTRUZIONI PER L'USO
GEBRAUCHSANWEISUNG
GEBRUIKSAANWIJZING

DIRECTIONS FOR USE

ENG

NeoBond® Bracket Adhesive and NeoBand® Light Cure Band Cement

INTENDED USE:

Bonding of orthodontic brackets and bands to natural and artificial tooth surfaces.

INDICATIONS FOR USE:

Indicated for patients who will need metal and ceramic orthodontic brackets and bands to be bonded.

CONTRAINDICATIONS:

- NeoBond Bracket Adhesive and NeoBand Light Cure Band Cement are contraindicated for use with patients who have a history of severe allergic reactions to methacrylate resins or any of the other components.
- NeoBond Bracket Adhesive and NeoBand Light Cure Band Cement are contraindicated for use as a restorative material.
- NeoBond Bracket Adhesive and NeoBand Light Cure Band Cement are contraindicated for bonding metal crowns.
- NeoBond Tooth Etchant is contraindicated for use on soft tissues including oral mucosa, eyes, and dental pulp tissue.

WARNINGS:

- NeoBond adhesive system components contain polymerizable methacrylate monomers. Avoid prolonged or repeated contact with skin and eyes. Avoid prolonged inhalation. Do not take internally. If skin sensitization or other reactions occur, discontinue use.
- NeoBond Tooth Etch contains phosphoric acid, which may cause burns. Avoid contact with oral tissues, eyes and skin. Do not take internally.

- **Eye Contact:** Tooth Etch contains phosphoric acid, which may cause burns or may be irritating to eyes. Before using this product wear protecting glasses as well as covering the patient's eyes to protect from squirting the material inadvertently. In case of accidental contact with eyes, rinse eyes with plenty of water and seek medical attention.
- **Skin Contact:** Tooth Etch contains phosphoric acid, which may cause burns or skin sensitization in susceptible individuals. If contact with skin occurs immediately wipe off and flush with generous amounts of water, then wash well with soap and water after contact. If skin rash, irritation, sensitization or other allergic reaction occurs, seek medical attention immediately.
- **Oral Mucosa Contact:** Avoid contact with oral soft tissue. If accidental contact occurs, flush with plenty of water and expectorate water. If sensitization of mucosa persists, seek medical attention immediately.
- **Ingestion:** Do not swallow or take internally. If accidental swallowing occurs, drink lots of water. If nausea or illness develops, seek medical attention immediately. Contact regional Poison Control Center immediately.
- In case of prolonged inhalation, give oxygen or artificial respiration if necessary.

PRECAUTIONS:

- For professional use only. U.S. Federal law restricts this device to sale by or on the order of a Dentist.
- NeoBand Light Cure Band Cement, NeoBond Primer, Bracket Adhesive, and Tooth Etchant are to be used only as specifically outlined in these directions for use. Any use of these products inconsistent with these directions for use is at the discretion of and sole responsibility of the practitioner.
- Curing for less than the specified time can reduce the bond strength.
- Use only in well ventilated area.
- Wear suitable protective eyewear clothing and gloves. Protective eyewear is recommended for patients when the tooth etchant is used.

- Manufacturer supplies the appropriate dispensing syringe tip for the Tooth Etch. This tip is the only tip that should be utilized for placement of gel.
- NeoBond Tooth Etchant should extrude easily. **DO NOT USE EXCESSIVE FORCE.** Excessive pressure may result in unanticipated extrusion of the material.
- If a serious incident or adverse reaction occurs when using this device, please report it to the manufacturer and the applicable regulatory authority in the country or region where the user and/or patient is located.

ADVERSE REACTIONS:

NeoBand Light Cure Band Cement, NeoBond Primer, and Bracket Adhesive contain polymerizable methacrylate monomers, which can cause skin sensitization (allergic contact dermatitis) in susceptible individuals. Avoid prolonged or repeated contact with skin or soft tissues and eyes. Avoid prolonged inhalation.

NeoBond Tooth Etchant may irritate the eyes and skin. It may:

- Irritate and cause corneal damage.
- Irritate the skin and cause allergic response and red rashes.
- Cause inflammation, edema, and sloughing of mucous membranes.
- Cause serious health effects if ingested.
- Cause pulpal effects if placed on the pulp.
- Do not use with patients who have a history of severe allergic reaction to any of the components.
- Inhalation of vapors may cause varying degrees of damage to the affected tissue and also increased susceptibility to respiratory illness. Medical conditions (pre-existing skin disorders, eye problems or impaired liver, kidney and respiratory functions) may be aggravated by exposure to the tooth etchant.


Please refer to the Warnings and Precautions.

STEP-BY-STEP INSTRUCTIONS:

1. Clean teeth with pumice or a non-oil containing prophylaxis paste, using a rotating bristle brush or rubber cup.
2. Rinse, isolate, and thoroughly air-dry (clean, oil-and-water-free compressed air) the surface area to be bonded.
3. When cementing bands, apply a small amount of NeoBand Light Cure Band Cement onto the tooth-bearing surfaces of the band and continue to step 7. For bracket bonding continue to step 4.
4. Etch the enamel with NeoBond Tooth Etchant. Apply the etchant and wait 20 seconds, then rinse thoroughly (10-15 seconds). Dry with a clean, oil and water free compressed air for 15 seconds. Properly etched enamel will appear dull frosty white. Surfaces must be kept free of contamination by oral fluids. If salivary contamination occurs, thoroughly clean with forceful water spray (free of contamination), blot dry, and repeat etching on enamel for only 10 seconds, rinse, and dry.

NOTE: For bonding to porcelain crowns, please follow these directions. A porcelain etchant is NOT provided in this kit. Please observe precautions for intra-oral use of porcelain etchant (hydrofluoric acid) before this step! When bonding to porcelain, the bonded porcelain surface should be etched with dilute hydrofluoric acid, rinsed, thoroughly dried. The porcelain surface should then be primed with a porcelain silane. Caution - some silanes are flammable! After removing residual silane with a vigorous blast of oilfree air, continue with steps 5-9.



NOTE: When bonding to amalgam, roughen the amalgam surface with a diamond bur or with a micro-etcher, before applying the NeoBond Tooth Etchant.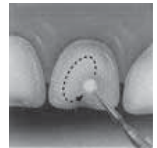

5. Apply a thin uniform layer of Primer onto the etched enamel surface. Gently spread the Primer by use of air. Do not rinse! If desired, using a conventional halogen curing light or an LED light, polymerize the primer. Curing the primer is not required. The primed surfaces should have a glossy appearance.

NOTE: Do not remove Primer with the high power suction!

6. Apply a small amount of NeoBond Bracket Adhesive onto all tooth-bearing surfaces of the bracket. The adhesive should be added to one bracket at a time.
- 6A. Apply the NeoBand Light Cure Band Cement material, for composite buildup, to avoid undesired occlusal contact with the appliances in deep bite cases.
7. Immediately seat the bracket or band onto the tooth with a slight rotating motion. A small amount of adhesive should exude around the entire bracket or band periphery. Final adjustment must be made before the adhesive begins to cure from the ambient light (approximately 60 seconds).
8. NeoBond Bracket Adhesive and NeoBand Light Cure Band Cement can be easily wiped away with a cotton roll before it starts setting, or can be removed with a scaler after it has set.
9. Using a conventional halogen curing light or a LED light, polymerize entire bonded surface for 20 seconds from the facial, mesial, and distal directions.

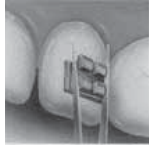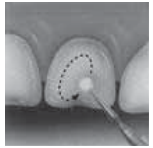DISINFECTION OF SYRINGE AND/OR BOTTLE

The syringe and/or bottle if exposed to spatter or spray of body fluids or that may have been touched by contaminated hands, or oral tissues, should be disinfected with a hospital-level disinfectant. Acceptable disinfectants are EPA registered as tuberculocidal, iodophors, sodium hypochlorite (5.25%), chlorine dioxide and dual or synergized quaternaries are approved disinfectants. Disinfect the syringe and/or bottle by spraying the syringe and/or bottle with any recommended hospital-level disinfectant except neutral glutaraldehyde for the contact time recommended by the disinfectant manufacturer for optimum results. Some phenolic-based agents and iodophor-based products may cause surface staining. Agents containing organic solvents, such as alcohol, may tend to dissolve the plastic. The disinfectant manufacturer's directions should be followed properly for optimum results. Water-based disinfectant solutions are preferred.

STORAGE AND SHELF LIFE

1. Replace the cap of the tooth etchant tightly after each use to avoid evaporation. Discard dispensing tips after use because dispensing tips may clog if the gel is allowed to dry inside the tip.
2. Store between 2-24°C (35.6-75°F). For optimum freshness, keep refrigerated. Please note that refrigeration will increase viscosity, making syringing more difficult, and will retard setting time. Do not freeze.
3. Keep away from direct sunlight.
4. Do not use after expiration date.

SAFE DISPOSAL:

Dispose of used product in accordance with national and local regulations.

It may be appropriate to consider used dispensing tips as contaminated and dispose of them as biohazardous waste.

Always follow Universal Precautions when in contact with patients.

WARRANTY AND LIMITED REMEDY

Manufacturer warrants that this product is free from defects in materials and manufacture.

Manufacturer's sole obligation and customer's sole remedy in the event of a claimed defect shall be limited to replacement merchandise or refund of the purchase price.

All implied warranties, including implied warranties of merchantability or fitness for any particular purpose or use are disclaimed.

LIMITATION OF LIABILITY

Except where prohibited by law, Manufacturer will not be liable for any loss or damages arising from this product, whether direct, indirect, special, incidental, or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability. This limitation does not apply to third party personal injury claims.

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