SureSmile® Aligner

Doctor Directions For Use

These are custom-made aligners
for an individual patient.

EN	Directions For Use
DE	Gebrauchsanweisung
DK	Instruktioner til Brug
ES	Instrucciones de Uso
FR	Mode d'emploi
IT	Istruzioni d'uso
NL	Gebruiksaanwijzing
NO	Instruksjon for Bruk
PT	Instruções de Uso

suresmile.com

For Professional Use Only – Rx Only

Device Description

SureSmile® Aligners are fabricated from clear thin thermoformed plastic in a sequential series to reposition the teeth progressively. Minor, incremental changes in tooth positions in each subsequent aligner deliver corrective forces to straighten the teeth.

Indications for Use

SureSmile® Aligners are indicated for the treatment of tooth malocclusion in patients with permanent dentition.

Contraindications

SureSmile® Aligners are contraindicated for use in patients with active periodontal disease.

Warnings

- In rare instances, some patients may be allergic to plastic. Patients with a history of allergic reactions to plastics should not use this product.
- Do not use damaged or broken aligners, as their parts can be accidentally swallowed or aspirated, and may be harmful.

Precautions for Doctors

- Do not sterilize SureSmile® Aligners.
- When evaluating the use of interproximal reduction to create space for tooth movement, you should consider all associated risks.
- Treatment with SureSmile® Aligners is an orthodontic therapy and you should consider all potential side effects associated with such therapy.
- The health of the bone and gums supporting the teeth may be adversely affected.
- Certain medical conditions or medications can affect orthodontic treatment outcomes.
- Orthodontic treatment may shorten the teeth roots. This may reduce the longevity of the teeth.
- Missing, erupting or unusually shaped teeth may affect treatment outcomes.
- To help prevent supra-eruption, the aligner should cover all teeth.
- Existing dental restorations such as crowns or bridges, may require recementation or, in some cases, replacement.
- Aligners are not effective for moving dental implants.
- You may need to use skeletal expanders, bonded auxiliaries, elastics and other fixed and/or removable orthodontic appliances for more complicated treatment plans where aligners alone may not be adequate to achieve the desired outcome.

EN-02

- You may need to bond attachments temporarily to one or more teeth during treatment to facilitate aligner retention or tooth movement.
- Replace attachments if they become loose.
- Remove attachments at the end of treatment.
- You may need to adjust the patient's bite at the end of treatment.
- Joint pain, headaches, or ear problems may result due to temporomandibular joint issues. This rarely occurs.
- Supplemental treatment may be required for severe open bite/overjet or skeletally narrow jaw.
- To correct severe crowding or jaw imbalances, oral surgery could be required. If so, consider risks associated with surgery before starting aligner treatment.

Precautions for Patients

- The patient may experience tenderness or temporary discomfort at initial aligner placement and after switching to each new aligner.
- The aligner may initially affect speech and may result in a lisp, usually disappearing within two weeks.
- The patient may experience a temporary increase in salivation or dryness of mouth.
- The aligners may scratch or irritate the patient's lips, cheeks or gums.
- The patient should not wear aligners when drinking hot, sweet, or colored liquids.
- If patient does not brush and floss correctly during treatment, or if they consume sugary foods or beverages while wearing aligners, they may be subject to tooth decay, periodontal disease and permanent markings from decalcification and staining.
- Lack of compliance in wearing the aligners for the prescribed number of hours per day or not using the aligners as directed can increase treatment time and affect desired results.
- The patient's teeth may shift position after treatment. To help prevent this, the patient should wear retainers.

Adverse Reactions

Allergic reactions, to plastics.

Step-by-Step Instructions

1. Rinse new aligners with cold water before inserting in the patient's mouth.
2. When an extraction is part of treatment, retain the space until the area heals. Add pontics to the aligners, if applicable. Wait until after an extraction site has completely healed before bonding attachments.

EN-03

3. Seat aligners on the patient's teeth and check for soft tissue impingement. Trim the aligners as needed. Polish the aligners after trimming.
4. Complete any attachment placement and perform any necessary interproximal reduction (IPR) procedures.
5. Review the SureSmile® Aligner Patient Instructions with the patient and go over the following points:
 - Each aligner is engraved with the first four letters of patient's first name. An "U" or an "L" denotes whether the aligner is for the upper or lower arch, followed by another number indicating the treatment stage.
 - Aligners should be worn 20 to 22 hours daily. Each set of aligners should be replaced after approximately 2 weeks of use (i.e., about 300 hours per aligner).
6. Dispense the next sets of aligners. Make sure you distribute enough aligners to last until the next appointment.
7. Provide the patient with both storage cases. One case is for the current set of aligners; the other case is for the previous set just worn.
8. At each appointment, check teeth for tight interproximal contacts at the appropriate aligner stage and check the fit of the aligners.
9. If the patient needs additional aligners, you may need to take a new scan.
10. Recommend that the patient wears retainers after treatment.

Daily Care and Maintenance of Aligners

Retainer Brite® products are recommended to clean SureSmile® aligners safely and effectively. Retainer Brite® product samples are provided with each set of aligners to assist patient with cleaning and hygiene.

1. Patients should clean their aligners before each insertion.
2. Patients should NOT use toothpaste or soak aligners in mouthwash, as this can scratch and dull the plastic, adversely affecting the aligner's clarity.
3. Patients should follow the specific directions for use provided with each Retainer Brite® product.
4. Patients should never put Retainer Brite® Cleaning Tablets or tablet solution in their mouth.

Proper Oral Hygiene

1. Patients should remove aligners when eating or drinking any beverage other than cool water.
2. Patients should brush and floss their teeth after each meal or snack before reinserting their aligners.
3. We recommend regular dental checkups and cleaning for the continued health of a patient's teeth and gums.

If you have additional questions after reading these directions, please contact Customer Service at 888.672.6387 (US/CA) or +800 6655 1234 (EU/AU/NZ).

Symbol Glossary		
Symbol and Standard* Reference No.	Title of Symbol	Description of Symbol
 5.4.4	Caution	Indicates the need for the user to consult the instructions for use for important cautionary information such as warnings and precautions that cannot, for a variety of reasons, be presented on the medical device itself
 5.4.3	Consult Instructions for Use	Indicates the need for the user to consult the instructions for use.
* Source: ISO 15223-1:2012-07-01, Medical devices — Symbols to be used with medical device labels, labelling and information to be supplied — Part 1: General requirements		
Rx only **	Prescription Use	Caution - US Federal law restricts this device to sale by or on an order of a dentist.
**81 FR 38911 Use of Symbols in Labeling. Final Rule		

Manufacturer:
[Corporate Headquarters](#)
OraMetrix, Inc.
 2350 Campbell Creek Blvd.
 Suite 400
 Richardson, Texas 75082 USA
 Phone +1 972 728 5500

EC	REP
----	-----

[Authorized EU Representative](#)
OraMetrix GmbH
 Rungestr. 19
 10179 Berlin Germany
 Phone +49 30 2430 910

[Australian Sponsor](#)
OraMetrix Pty. Limited
[Suite 9, Level 6](#)
 10 Help Street
 Chatswood NSW 2067 Australia
 Phone +61 2 8035 5400

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©2018 Dentsply Sirona