Ortho-LipGuard™ Strips

DIRECTIONS FOR USE

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MODE D'EMPLOI

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Manufacturer:

Dentsply International Raintree Essix

7290 26th Court East

Sarasota, FL 34243 USA

Tel: 800.883.8733

www.essix.com

Emergo Europe
Prinsessegracht 20
2514 AP, The Hague
The Netherlands

**SINGLE
USE ONLY**

Ortho-LipGuard™ is not a trademark of
Dentsply Sirona.

#DFU-OLG RTE-130-17 Rev. 4 6/17

Ortho-LipGuard™ Strips

ENG

Indications for Use:

Provides temporary relief of direct pressure to the lips caused by orthodontic appliances.

Contraindications:

Not designed to provide protection during contact sports.

**Warnings:**

Hot water may cause severe burns. Use caution when forming this device.

Precautions:

Always check with your orthodontist to make sure the Ortho-LipGuard™ Strip will not interfere with other appliances.

Continuous wear may delay orthodontic treatment, so limit use to several hours per day.

If under age 18, have an adult assist with forming of this product.

Review all steps before forming the Ortho-LipGuard™ Strip.

Single Use Only. Risks of reuse include infection and cross-contamination to patient and healthcare provider as well as other wear-related irregularities.

Adverse Reactions:

None Known.

Step By Step Instructions:

Collect one of each of the following:

- Ortho-LipGuard™ Strip
- Container of cold water
- Container of boiling water
- Utensil (for example, a plastic fork)
- Clock or timer indicating seconds

The next steps must be performed quickly to prevent strip from setting before a comfortable impression of your braces is made.

1. Submerge the strip in boiling hot water for 45 seconds.
2. Using utensil, remove strip from hot water and quickly submerge it in cold water for 3 seconds.
3. Using utensil, remove strip from cold water.
4. Hold curved portion of strip with index fingers and thumbs of both hands. Press and mold the flat portion of the strip over your braces, across the front of your 4 to 6 middle teeth. The goal is to create a smooth, contoured and comfortable surface that protects the inside of your lips from the sharp edges of your braces. Using a mirror may be helpful during this step.
5. Allow the strip to cool in place for approximately 45 seconds.
6. Remove the strip and repeat the process as necessary to create a molded strip for your upper and lower teeth. It may take several attempts to perfect the fit. This process should be repeated every few weeks as your teeth move.

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