USER MANUAL

The Essix[®] Machine

Vacuum Thermoforming Machine

Item #: 85000, 85220 & 85220-CE

ESSIX® MACHINE USER MANUAL

ENG

Thank you for your purchase of the Essix® Vacuum Machine.

Please read the important notes and the entire instruction manual before use.

The Essix® Machine utilizes an electrically driven vacuum motor to fabricate all types of orthodontic & dental appliances more accurately than traditional vacuum forming machines. By incorporating a two stage motor, it helps produce maximum adaptation, ease of use and professional results.

Please read the entire instruction manual before operating the Essix® Machine. You can learn correct operating procedures by carefully reviewing the following pages, however, mastering vacuum forming techniques may require a longer period of time. Practice with appropriate materials is recommended. Also, pay close attention to all "Note" and "Caution" statements included in each section.

UNPACKING AND SETUP

The Essix® Machine requires minimal preparation for use. A few simple steps are necessary before operating the unit:

1. **Handles:** Attach the two enclosed "tear drop" shaped knobs into the threaded holes located at the rear of the hinged frame, attach the remaining knob on the front of the hinged frame according to your preference. Install the heating unit handle into the hole centered in the front of the heater unit.
2. **Placement:** The Essix® Machine should be placed on an open level counter, within 4' of a standard properly grounded electrical outlet.

CAUTION: The unit should not be placed closer than 6" from walls, cabinets or other equipment while in operation because of the temperature emitted from the heater. Under no circumstances should this unit be operated while located under or in a cabinet. Turn off the machine when not in use. Do not place any items under the heating element.

3. **Electric Cord:** Plug the female end of the cord into the electrical inlet located on the back of the unit and the male end into a properly grounded electrical outlet. NOTE: The unit's power cord should not be placed near the heating unit or any other hot surface.

PARTS IDENTIFICATION

- | | |
|------------------------|-------------------------|
| 1. Heating Unit | 7. Vacuum Plate |
| 2. Heating Unit Handle | 8. Base Casting |
| 3. Vertical Post | 9. Heater Pilot Light |
| 4. Hinged Frame Boss | 10. Heater/Motor Switch |
| 5. Hinged Frame | 11. Tear Drop Knob |
| 6. Frame Latch Knob | |

Technical Specifications:

Single Phase 110 Volt: 115V- 50/60Hz 1475W 18#’s

Single Phase 220 Volt: CE 230V- 50/60Hz 1475W 18#’s

WARRANTY

The Essix® Machine is warrantied for its designated use, in accordance with the instructions, against original defects in material and workmanship for a period of 5 years from the date of purchase.

This warranty extends only to the original purchaser.

MODEL PREPARATION GUIDELINES

BASE - The base of the model should be as slim as practical - no more than 3/4" from the base to the incisal edge. Excessive base can result in thinning of the material and/or decreased details.

STONE - Quality stone or gypsum that produces a hard, dense model is suggested. Softer materials may be dusty. This dust may settle into the finished appliance.

UNDERCUTS - The trimmed sides should be flat and cut at a 90° angle. Undercuts and angles that cut into the base will create a vacuum lock and make it difficult to remove the appliance. If possible, it is best to have the base flare out by a few degrees. This is completed during the trimming stage.

INSTRUCTIONS FOR BASIC OPERATION

- 1. Swing the heating unit fully to the rear and turn on the heating element switch (Fig. 1). It will require approximately three minutes to preheat. Preheating the unit before use is especially important when using materials .040" or thinner. These materials heat very quickly and it is best to have the heating element temperature stabilized prior to use.
- 2. Place the cast (stone, gypsum, etc.) inside the Vacuum Maximizer® on the vacuum plate (Fig. 2). The vacuum machine will accommodate 1 full arch Essix trimmed cast.
- 3. Raise the hinged frame. Open the hinged frame using the handle provided and place a sheet of Essix® plastic within the frame (Fig. 3). Center it carefully.
- 4. Close the frame and secure with the frame latch (Fig. 4). Swing the heating unit into position squarely over the material (Fig. 5).

CAUTION: Move heating unit only by the handle provided. Do not leave the machine unattended while heating plastic.

- 5. The plastic heats quickly and will be useless if allowed to drop excessively. In extreme cases, the plastic may be absorbed into the vacuum motor causing damage to the motor.

IMPORTANT INFORMATION: See the Essix® Plastic instructions for the specific type of plastic you are using. Instructions will indicate heating times, sag depth and what to look for when thermoforming with a specific type of Essix® Plastic.

- 6. Observe the plastic as it heats. Follow directions for the Essix® Plastic you are using. Watch for sag depth if plastic specifies (Fig. 6).
- 7. Turn on vacuum, then firmly lower the plastic over model (Fig. 7).
- 8. Push plastic into the under cuts of model with the Essix Accentuator® tool (Fig. 8).
- 9. Immediately spray with Essix Freeze Spray® Coolant (Fig. 9). **This is an essential step.** Not only does it cool the plastic, prohibiting further thinning, but it "shrinks" the material around the cast, realizing a tighter fit. Let the vacuum run until the plastic encased model is cool. Open the release handle and remove the thermoformed plastic from the vacuum machine.
- 10. To remove the cast from the adaptation, first cut away the flat surplus plastic. Trim as close to the cast as possible to facilitate handling. Universal Shears are recommended.
- 11. For hard, thin plastics (.040 or less thickness) use the Essix® Lab Knife (or any grinding wheel suitable for trimming excess acrylic) to cut through the heel of the cast enclosed in plastic. Using the cast removing instrument, lift the plastic off of the model. For other plastics, a wheel saw or Precision Heat Knife are recommended for other trimming techniques.
- 12. Final, detailed trimming with scissors and polishing brushes will produce a smooth, professional, finished appliance.

TROUBLESHOOTING GUIDE

1. **Insufficient detail:** This is often caused by improper heating or vacuum time. If your adaptation does not show enough detail, allow the material to heat longer prior to forming and/or increase vacuum time. When using Essix® plastics, make sure to follow the plastic instructions during the cooling phase of thermoforming.
2. **Material thin on incisal edge:** First, check the height of the cast, it should be trimmed to no more than 3/4". Second, do not center the appliance on the base plate. The anterior section should always be offset.
3. **Material creasing or tearing:** This is a result of excessive heat. Reduce the heating time. It is also important to stabilize the heater temperature by pre-heating the unit for approximately 3 minutes prior to forming.
4. **Material sticking to the model:** Spray the model with Trim-Rite® Release Agent prior to forming.
5. **Material stuck in vacuum holes:** This can occur if the material is overheated and pulled into and through the vacuum holes. Allow materials to cool completely. Attempt to pull the material from the platform. If the material is not protruding through the vacuum holes, gently pry the material off.
6. **Heater does not function:** Ensure power cord is properly connected to the wall outlet and rear of Essix® Machine. The heater is always functional when the pilot light is on. If the pilot light does not illuminate when turned "on", disconnect the power cord from the wall outlet and rear and check the fuse. Replace if necessary. (See Cleaning and Maintenance – for fuse replacement instructions).

If the Essix® Machine remains non-operational, please call Customer Service for further assistance at: (800) 883-8733.

CLEANING AND MAINTENANCE

1. A slightly damp cloth can be used as necessary to wipe down the unit. No other cleaning should be required.
2. Periodically inspect the power cord for wear (ie., fraying, etc.) and check the plug to ensure that the connectors are fully seated.
3. **Fuse Replacement** – The power inlet on the back of the Essix® Machine incorporates a fuse. The fuse is rated for proper operation under normal conditions. If the fuse needs to be replaced, an extra fuse is included for your convenience. It is located in the fuse holder. First, remove the cord assembly from the unit by unplugging from the electrical outlet and from the unit itself. Use a flat bladed screwdriver to gently pry the fuse holder from the inlet at the area indicated by the fuse symbol. Remove the fuse and replace it with the one located in the holder. It is important to obtain spare fuses of the same type for replacement (15 Amp for the 110V and 6.3 Amp for the 220V unit). A different type of fuse may cause failure of the unit and will void warranty.
4. Only the finest quality materials are incorporated in the Essix® Machine. However, after years of use, the heater or some other part may need replacement. If a part requires replacement, please contact Customer Service at (800) 883-8733.

WARRANTY TERMS AND CONDITIONS

The Essix® Machine is warranted for its designated use, in accordance with the instructions, against original defects in material and workmanship for a period of 5 years* from the date of purchase. This warranty extends only to the original purchaser. Gaskets, fuse and power cord are excluded from warranted parts.

*Warranty void if unit is tampered with or if there is evidence of tampering.

NOTE: If the equipment is not used in the manner specified by the manufacturer, the protection provided by the equipment may be impaired. Gaskets, fuses, and electrical cord are excluded from warranty guidelines.

REPAIRS

Contact Dentsply International Raintree Essix for repair procedures.
For troubleshooting or specific questions, please contact:

Dentsply International Raintree Essix
7290 26th Court East
Sarasota, FL 34243 USA

For Technical Assistance: Phone: (800) 883-8733

Do NOT pack the Essix® Machine in Styrofoam chip material for shipping. The material may lodge into the heater mechanism and become a fire hazard. Remove handles.

The heater in the vacuum forming machine may absorb humidity or moisture when stored for a period of time. This may penetrate to the internal connectors and cause the unit to short circuit your GFCI outlet. If this occurs, we recommend using the machine on a non-GFCI outlet for approximately twelve cycles. This will enable the moisture to dissipate.

The situation is rare and is not cause for an alarm. Should you wish to discuss it further, please contact the Customer Service Center at: (800) 883-8733.

The Essix® Machine is intended for indoor use in a standard dental laboratory environment.

Ambient operating temperature:	5-40C.
Relative Humidity:	Max 80%
Altitude:	Maximum altitude 2,000 m
Installation Category:	nominally 11
Pollution Degree:	nominally 2
Main supply fluctuations not to exceed ± 10% of the nominal voltage	

THE ESSIX® MACHINE SYMBOL KEY

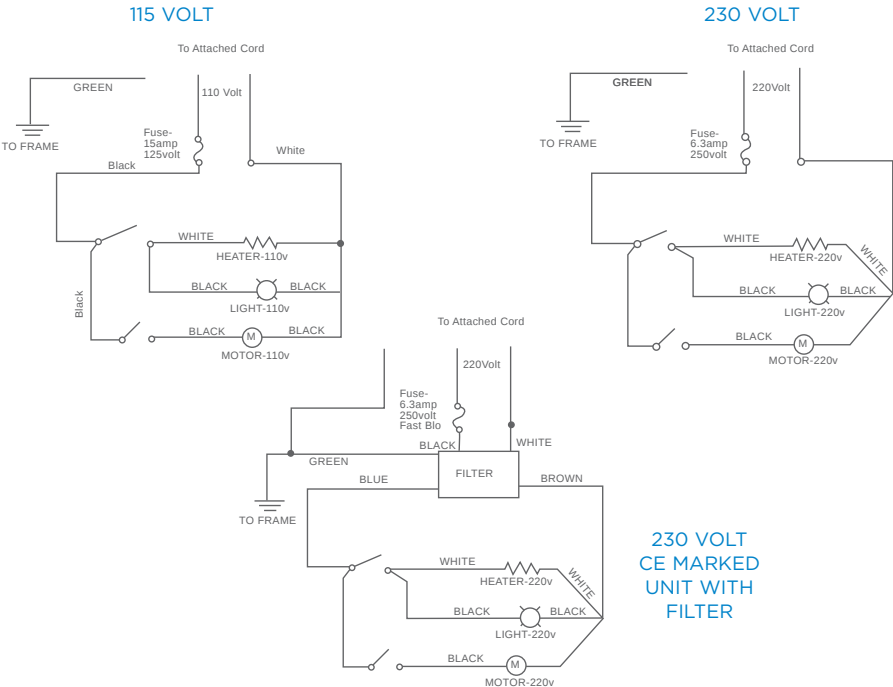
	Fuse		On		Attention		Heater
	Ground		Off		Vacuum		

THE ESSIX® MACHINE TECHNICAL SPECIFICATIONS

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